

Installation Instruction

(The image is blue item for the reference)

安装示意图：



1. 折叠完毕状态



2. 前后拉开底部框体



3. 将靠背扶手插入管内
锁定卡扣



4. 完成安装

Installation
Diagram:

1. Fully folded state
2. Pull the bottom frame apart front and back
3. Insert the backrest and armrests into the tubes, and then lock the clips
4. Installation complete

折叠示意图：



1. 将产品那起来前后对折



2. 折叠完成

Folding
Diagram:


1. Lift the product and fold it in half front to back
2. Folding complete

细节须知：

1. 高度调节依靠调整四个套管,如下图所示,按下弹珠,抽动套管到不同的孔位,对应不同的档位。



2. 扶手可翻转设计,方便清洗侧身

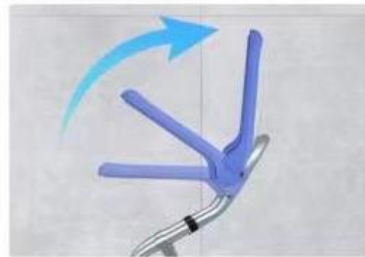

3. 防寒垫请撕开背面的保护膜,贴上去,而不是直接放在座板上。



安全须知：

1. 请在安装洗澡椅以前先仔细阅读并目理解本说明书。如果您无法理解说明书示例说明,请和当地经销商联系。确保理解后,再行安装。
2. 使用前请确认地面平坦,洗澡椅框体四脚平稳接触地面。
3. 使用前确保调节高度的弹珠顺利弹出到位,同时也在同一档位。
4. 安装以前请仔细检查框体和每一个配件是否有运输破损。一旦发现任何破损,请停止安装,并和我们的经销商联络。等待他们的指导。
5. 请仔细检查防滑脚垫是否有撕裂或者其他破损,一旦发现,请及时联系当地经销商更换配件。
6. 身体有残疾的使用者请在家人帮扶下安装使用。

Important Details:

1. Height adjustment is achieved by manipulating the four sleeves as shown below. Press the ball bearing and slide the sleeve into different holes to select various height settings.
2. The armrests feature a flip-up design for easy cleaning of the sides.
3. For the cold-weather pad, peel off the protective film on the back and attach it directly to the seat board instead of placing it on top. Im on the back and attach it directly to the seat board instead of placing it on top.

Safety Precautions:

1. Thoroughly read and understand this manual before installing the bath chair.
2. Ensure the floor is level and all four legs of the bath chair frame make stable contact with the ground before use.
3. Verify that the height adjustment ball bearings are fully engaged and aligned in the same position before use.
4. Inspect the frame and all components for shipping damage before installation. Cease assembly if any damage is detected.
5. Carefully inspect the non-slip footpads for tears or other damage. If any is found, contact us immediately for replacement parts.
6. Users with physical disabilities should install and use this product with assistance from family members.