


**SAVE THESE INSTRUCTIONS**

## **OPERATION INSTRUCTIONS**

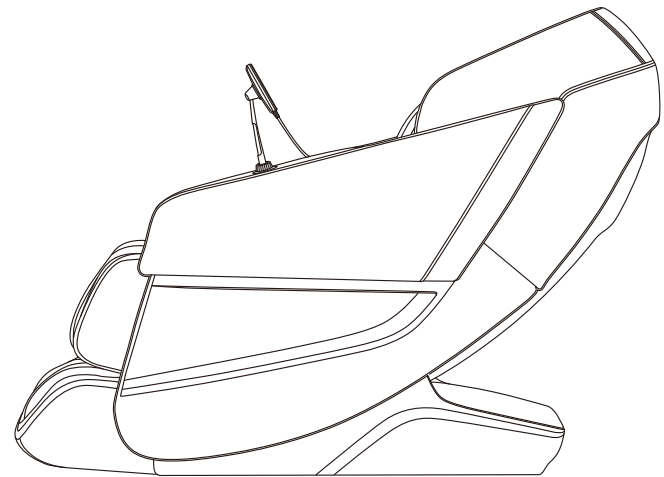

Thanks for purchasing this product. Please read this manual instruction carefully in order to know how to use this product correctly before operation. Please pay more attention to some important safety warning and keep this manual book properly for future use.

CONTENTS

<b>Before use</b>	
Safety precautions	02-05
Installation instructions	06
Place massage chair	07-08
Before use	09
Product structure	10-15



<b>Method of Usage</b>	
Operation Instructions	16-26
End of Massage	26

<b>Maintain</b>	
Cleaning and Maintenance	27


<b>Important Matters</b>	
Troubleshooting	28
Product Specifications	29

## Safety Precautions (Must be observed)


- Please read all relevant instructions carefully before use.
- The precautions indicated here will help you to use this product safely and correctly.
- In order to clearly indicate the magnitude of the harm and damage, precautions will indicate that the wrong use of the possible damage is divided into **"Warning"**, **"Attention"**. Both are important aspects of safety, so be sure to follow them.

 <b>Warning</b>	Indicates the contents of a matter that may result in serious injury or death if used incorrectly.
 <b>Attention</b>	Indicates matters that may cause minor injury or property damage if used incorrectly.


- Please pay attention to the symbols before each safety item


 <b>Prohibit</b>	Absolute prohibition
---	----------------------

### The following populations should not use this product


 <b>Prohibit</b>	<ul style="list-style-type: none"> <li>■ This product is not intended for use by persons (including children) with physical disabilities, reduced sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning the use of this product by the personnel responsible for their safety. Children should be supervised to ensure that they do not play with this product.</li> <li>■ Patients with osteoporosis.</li> <li>■ Patient with serious heart disease and those who wear implantable medical devices such as pacemakers and defibrillators that are susceptible to electromagnetic waves.</li> <li>■ People who have a fever.</li> <li>■ Pregnant women or women on their menstrual period.</li> <li>■ Patient who are prohibited from receiving massage on medical advice because of blood clots, aneurysms, severe varicose veins, or other circulatory diseases. For two weeks after the occurrence of the clot, any massage on the body part where the clot has occurred is prohibited.</li> <li>■ Patients who are injured or have a skin disease.</li> <li>■ When unsupervised, any children are under the age of 14 and those unconscious are not allowed to use this product.</li> <li>■ People who are told by their physicians to have rest or who feel unwell.</li> <li>■ Patient with abnormal spine, scoliosis or other diseases are prohibited from receiving massage with this machine.</li> <li>■ Patient with reduced mobility, weakness, communication difficulties or inability to take care of themselves should use this machine under the supervision of those responsible for their safety.</li> <li>■ People whose bodies are wet are forbidden to use this product.</li> <li>■ This product has a hot surface. Persons insensitive to heat must pay attention when using this product.</li> </ul>
---	--


## Safety Precautions (Must be observed)


Important Safety Warning	
 <b>Warnings</b>	<ul style="list-style-type: none"> <li>■ Do not allow children to touch any movable part of this product.</li> <li>■ Use the well-grounded power supply unit suitable for this product.</li> <li>■ Pull out the plug after use or before cleaning to avoid any physical injury or damage to the product.</li> <li>■ Operate this product as instructed in this manual.</li> <li>■ Do not use any parts or accessories which are not recommended.</li> <li>■ Do not use this product outdoors.</li> <li>■ Do not use near open flames such as heaters, and do not use while smoking.</li> <li>■ Do not place heating devices such as electric blankets on this machine.</li> <li>■ Do not sit on the backrest, calf rest, shoulder, armrest or stand on a seat</li> <li>■ Please read the Operation Instructions carefully before operation.</li> <li>■ Never use this product for any other purpose not listed herein.</li> <li>■ 20 minutes of usage each time is recommended.</li> <li>■ Do not use this product if the cover or leather is broken to avoid electric shock or injury.</li> <li>■ Do not drop anything into this product.</li> <li>■ Never fall asleep while using this product.</li> <li>■ Do not use this product if you are drunk or feel unwell.</li> <li>■ Do not use this product within one hour after a meal.</li> <li>■ Do not use this product with excessive force to avoid any injury.</li> <li>■ Do not use for purposes other than massage.</li> <li>■ If the power cord is broken, in order to avoid danger, it must be replaced by the manufacturer, maintenance department or other similar department.</li> </ul>

Safety Precautions	
 <b>Attention</b>	<ul style="list-style-type: none"> <li>■ Check whether voltage is consistent with the specification of this product.</li> <li>■ Never insert or pull out the plug with a wet hand.</li> <li>■ Do not let water flow into this product to avoid electric shock or cause damage to this product.</li> <li>■ Do not pull the power cord when plugging or unplugging it. Rough handling is forbidden.</li> <li>■ Do not damage the wire or modify the circuit of this product.</li> <li>■ Do not clean live parts of this product with a wet cloth such as a power switch and plug.</li> <li>■ Be away from this product in case of power failure to avoid any injury if power is restored suddenly.</li> <li>■ Stop using this product at once while it works abnormal, and consult online customer service.</li> <li>■ Stop using this product if you feel unwell, and consult your health care practitioner.</li> </ul>


## Safety Precautions (Must be observed)

Recommended Storage Condition	
 <b>Attention</b>	<ul style="list-style-type: none"> <li>■ Storage room temperature 41°F-104°F.</li> <li>■ Relative humidity of 20% to 80%, free of corrosive gases and well-ventilated room.</li> </ul>





Product Service and Maintenance	
 <b>Attention</b>	<ul style="list-style-type: none"> <li>■ If you encounter any fault, please contact online customer service first. No user should disassemble or maintain it without permission.</li> <li>■ Do not forget to turn off the main power switch after usage.</li> <li>■ Do not use this product if the power socket becomes loose.</li> <li>■ If this product will be left unused for a long period of time, it is necessary to roll up the power cord of this product and store this product in a dry and dust-free environment.</li> <li>■ Do not store this product at a high temperature or near open flame.</li> <li>■ Avoid prolonged exposure to direct sunshine.</li> <li>■ Please clean this product with dry cloth. Never use thinner, benzene or alcohol.</li> <li>■ The mechanical components of this product are specially designed and manufactured, so no special maintenance is needed.</li> <li>■ Do not use a sharp object to stab at this product.</li> <li>■ Do not roll or pull this product on any uneven ground. Instead, it should be lifted before moving.</li> <li>■ Please use this product intermittently. Do not keep it working continuously for a long period of time.</li> </ul>

Operating Environment	
 <b>Attention</b>	<ul style="list-style-type: none"> <li>■ Do not use this product in a very moist environment such as the bathroom.</li> <li>■ Stop using this product immediately when the ambient temperature changes sharply.</li> <li>■ Do not use this product in any heavily-dusty or caustic environment.</li> <li>■ Do not use this product in a small space or a place without good ventilation.</li> </ul>

## Safety Precautions (Must be observed)



Solutions to Ordinary Malfunctions	
 <b>Attention</b>	<ul style="list-style-type: none"> <li>■ If the motor sounds during operation of this product, this is the normal operation sound.</li> <li>■ If the controller cannot work normally, check whether the plug and the power socket are connected reliably, and whether the power switch is turned on.</li> <li>■ If the rated working time is over, this product will automatically activate the power-off button; If this product keeps operating for a long period of time, the temperature protector will stop this product automatically. The product can be used again after half an hour.</li> </ul>
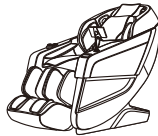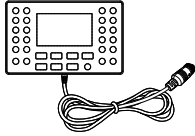
# Installation instructions

PARTS			
Ax1	Bx1	Cx1	Dx1
			





## Installation instructions of remote controller bracket

The thread on the support knob of the controller support assembly corresponds to the thread of the support mounting seat on the armrest(Figure 1),then turn clockwise,tighten and fix it(Figure 2).

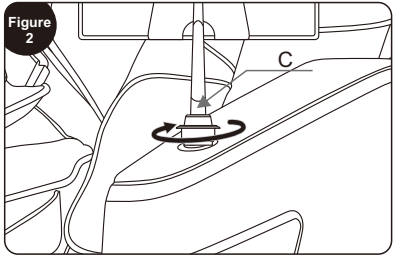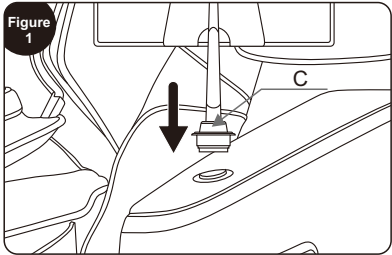

## Installation instructions for the aviation plug of the manual controller

**1.Fixed aviation plug installation method:** Align the notch of the manual controller aviation plug (Figure 1) with the raised position of the aviation plug (Figure 2) on the steel frame, then insert it (Figure 3), tighten it, and then lock the nut clockwise (Figure 4).

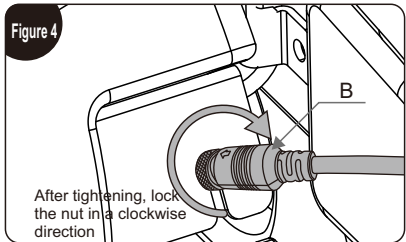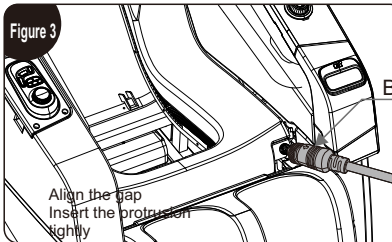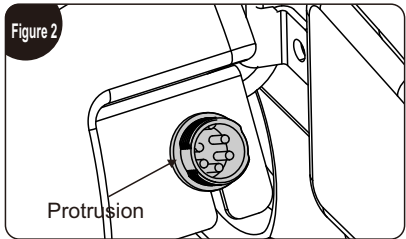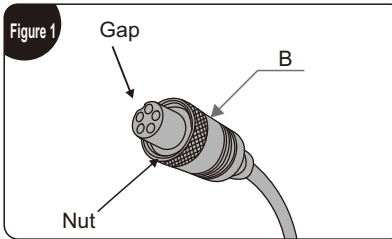


# Massage Chair Placement

## Installation placement

**Ensure enough space for the massage chair to tilt by at least 2.5cm.**

Do not expose the massage chair to direct sunlight and high temperature environment. If the massage chair is used in the specified position for a long time, it is recommended to lay a soft cushion. The size of the cushion should be enough to cover the scope of the machine to contact the floor and the scope of the foot to contact the floor.

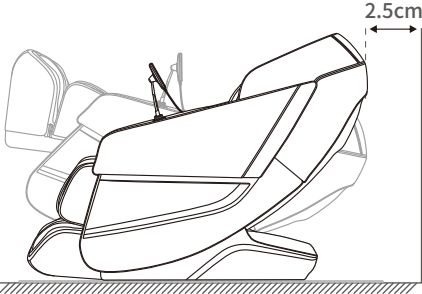


2.5cm

**Attention**

## Product Handling Instructions

**Hold the bottom sides of the frame, then lift and move. Because the machine is heavy, please be careful when handling it to avoid back strain. (This machine must be carried by two or more people)**

- Please lift the massage chair to move on the vulnerable floor such as wooden floor
- When moving on a wooden floor, it is recommended to lift it to avoid damage to the floor by casters. Lift and move to the designated position and put down, pay attention to avoid injury, and be careful of the foot is crushed. Do not loosen the unit until it is completely down, as the floor may be damaged, so it is recommended to lay a mat on the floor.

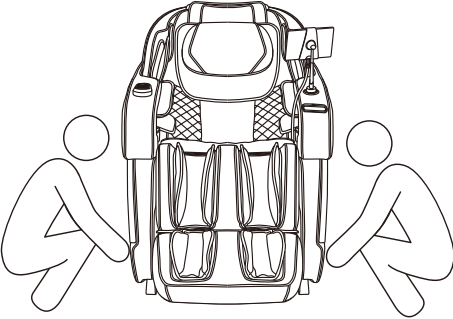


**Attention**

## Massage Chair Placement

### Move method

**Use the caster to move the massage chair. As illustrated, tilt the chair down and back about 45° to move. Do not press down too much to avoid the chair completely tipping and causing product damage.**



1. Power off correctly before moving, so that the backrest and calf mechanism are completely reset and the power supply is off.
2. Do not place things on the product when moving, and do not allow people, pets or debris to stay on the product.
3. When moving with casters, do not cross barriers with a drop greater than 2mm or gaps greater than 5mm.
4. When using casters to move the product, do not support a single continuous movement of more than 50 meters, please make the product slowly and uniformly move.

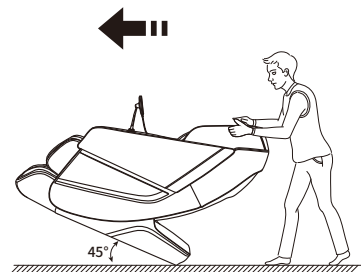

**Attention**

### Instructions for Removal and Use of Head Cushion and Back Cushion

The use of the head cushion can reduce the intensity of kneading massage on the neck and shoulders. You can decide whether to use the head cushion according to your needs (the head cushion is recommended). The backrest and the back cushion are connected by a zipper (1). The head cushion and the back cushion are connected by thread gluing (2).

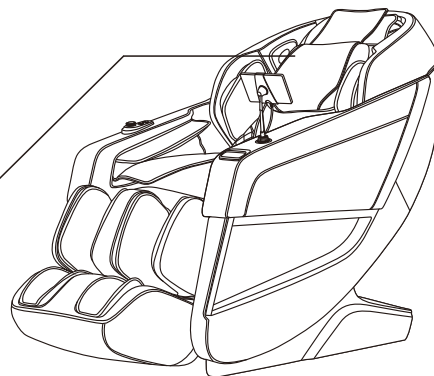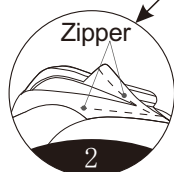

**Attention**


## Before use

### Check Around



**Attention**

Make sure there are no people, pets, or other objects around the machine.

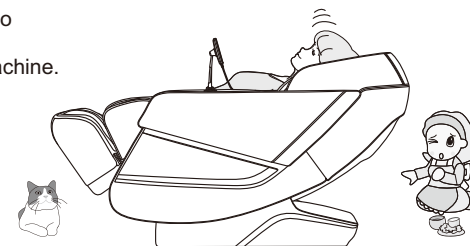


### Make sure there are no objects between the gaps



**Attention**


Before massage, please make sure that there is no objects in the calf rest or foot rest. Do not put your hands, feet, or head into the area between the calf mechanism and the body machine, or between the armrest and the capsule. Do not immerse your hands, feet, or head under the fabric or leather cover in the massage area.

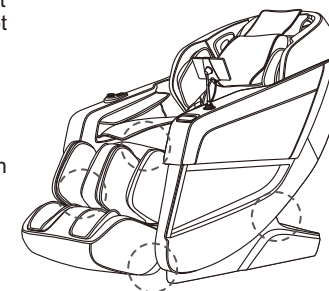


### Turn on the power supply of this product




**Attention**

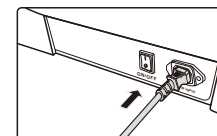

Schematic diagram of plug in

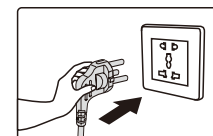

Schematic diagram of power on

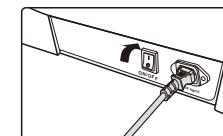

Turn on the power switch




**Warnings**

- Before use, check whether the power cable and power plug are damaged.
- Please supply power to this product according to the power supply requirements of this product.
- It is forbidden to use power supply other than the nominal power supply of this product.
- Do not connect any power conversion device to power this product without authorization.

## Product Structure


### Part description

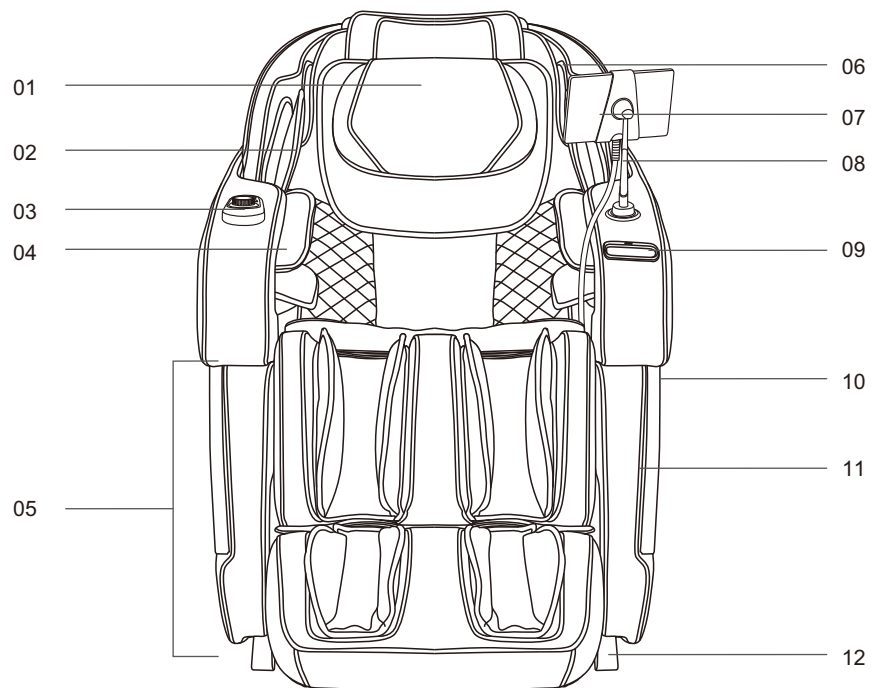

- |                           |                                   |
|---------------------------|-----------------------------------|
| 01.Head cushion           | 07.Remote controller              |
| 02.Shoulder airbag        | 08.Remote controller bracket      |
| 03.Short cut key          | 09.Placing slot/USB charging port |
| 04.Arm airbag             | 10.Mood lighting                  |
| 05.Calf massage mechanism | 11.Armrest                        |
| 06.Speaker cover          | 12.Side cover                     |

## Product Structure


### Schematic diagram of function

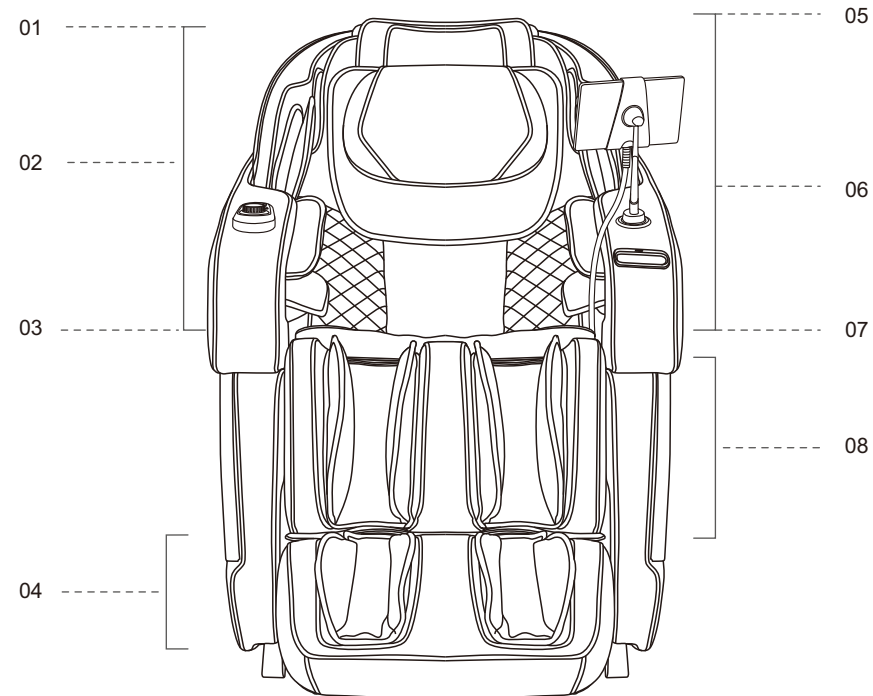

- |                                     |                                 |
|-------------------------------------|---------------------------------|
| 01.Head                             | 05.Both sides of the head       |
| • Voice control                     | • 3D digital audio              |
| • Built-in voice control head       | 06.From head and neck to bottom |
| 02.Shoulder of two sides            | • Back roller massage           |
| • Airbag massage                    | 07.Backrest                     |
| 03.The arm of both sides            | • Heating function              |
| • Airbag massage                    | 08.Calf rest                    |
| 04.Sole of the foot                 | • Heating function              |
| • foot roller massage               | • Airbag massage                |
| • Airbag massage                    |                                 |
| • Scraping massage before and after |                                 |

## Product Structure


Instruction of armrest short cut key

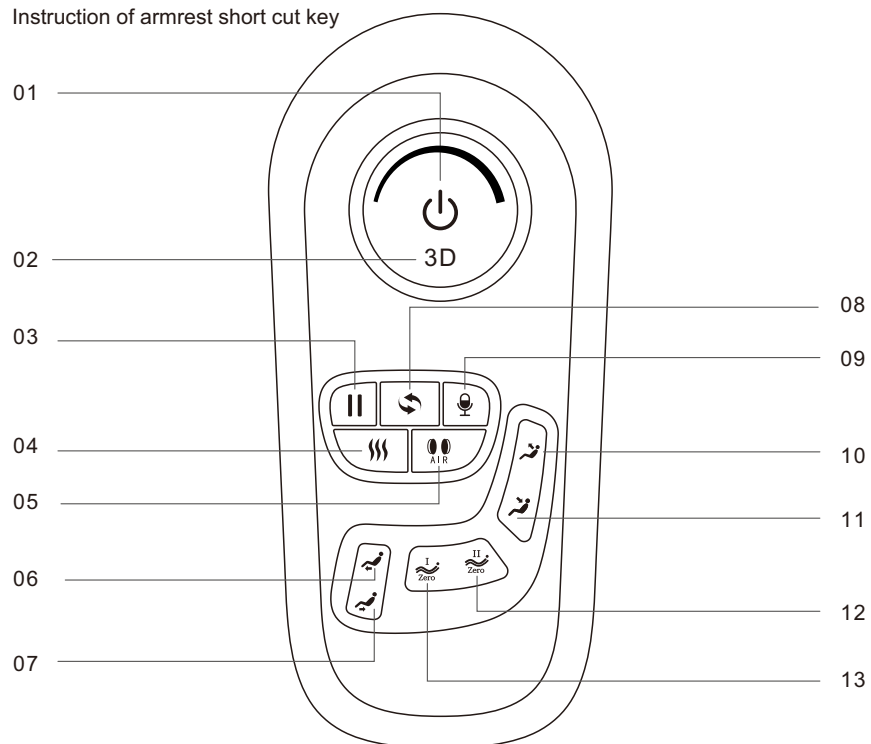

- 01.Switch button: Turn on or off the massage chair.
- 02.3D intensity: Adjust the massage intensity (rotation to the left reduces intensity, rotation to the right increases intensity).
- 03.Pause button: Pause massage or exit pause massage.
- 04.Heating button: Turn on or off the back or calf heating function.
- 05.Pressure button: Press this key to open or close the pressure massage function.
- 06.The calf mechanism rise key: Upward adjustment of the calf angle.
- 07.The calf mechanism down key: Downward adjustment of calf angle.
- 08.Automatic button: Select the automatic program (all automatic programs in order to cycle).
- 09.Voice control button: Short press to wake up the voice assistant, long press to turn on or off the voice control.
- 10.Sitting up button: Adjust the angle of the backrest upward.
- 11.Sitting down button: Adjust the angle of the backrest downward.
- 12.Zero gravity II button: The Angle is adjusted to zero gravity II.
- 13.Zero gravity I button: The Angle is adjusted to zero gravity I.

## Product Structure

Left armrest

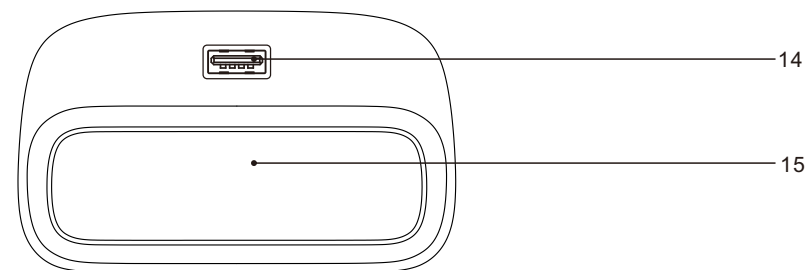


- 14.USB charging port: smart device charging.
- 15.Placing slot: mobile phone placing slot.




## Product structure

### The instruction of remote controller button

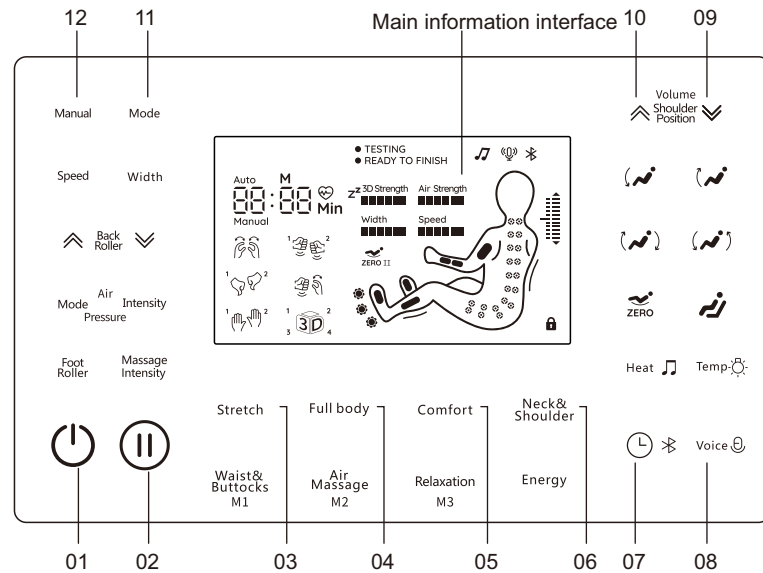

- 01.Switch button: Turn on or off the massage chair.
- 02.Pause button: pause massage or exit pause massage.
- 03.Stretch massage button: Open the stretch massage mode.
- 04.Full body massage button: Open the full body massage mode.
- 05.Comfort massage button: Open the comfort massage mode.
- 06.Neck & shoulder massage button: Open the neck & shoulder massage mode.
- 07.Timing/Bluetooth button: Short press to adjust the massage timing time; Press and hold for 3 seconds to turn Bluetooth on or off.
- 08.Voice wake up/switch button: Short press to wake up the voice assistant function, and long press for 3 seconds to turn on or off the voice control function.
- 09.Shoulder position/volume down button: Short press to adjust the shoulder position down; Press and hold to adjust the volume.
- 10.Shoulder position/volume up button: Short press to adjust the shoulder position up; Press and hold to increase the volume.
- 11.Mode button: Adjust five massage positions such as fixed point, partial point, full back, upper back and lower back.
- 12.Manual button: Turn on the manual mode, there are 11 modes can be adjusted.
- 13.Air pressure mode button: air pressure mode adjustment and open/close air pressure function.
14. Foot roller button: Turn on or off the foot roller massage function.
- 15.3D intensity button: Adjust the 3D intensity, it can be adjusted in 5 levels.
- 16.Air pressure intensity button: Adjust the pressure intensity, it can be adjusted in 5 levels

## Product structure

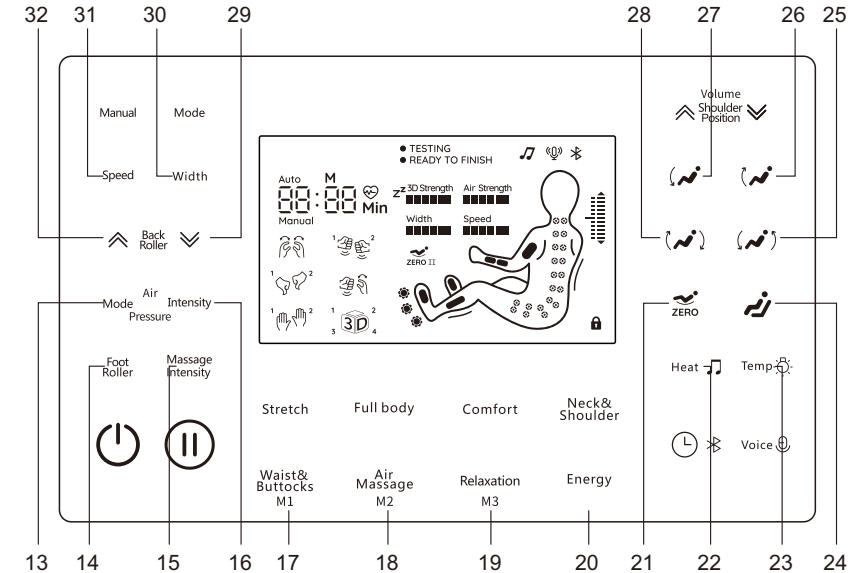


- 17.Waist & buttocks massage button: Short press to open the waist and hip massage mode, long press to open or close the intelligent storage M1.
- 18.Air massage button: Short press to open the full pressure massage mode, long press to open or close the intelligent storage M2.
- 19.Relaxation massage button: Short press to open relax massage mode, long press to open or close intelligent storage M3.
- 20.Energy button: Open the energy mode.
- 21.Zero gravity button: The key has two zero gravity mode/close zero gravity function.
- 22.Heating/background music button: Short press to adjust the heating part, open or close the heating function (back, calf); Press and hold for 3 seconds to open/close the background music function.
- 23.Temperature adjustment/mood lighting button: Short press to adjust the heating temperature , with two levels, long press 3 seconds to open or close the mood lighting.
- 24.Reset button: Press this key to adjust the angle to the initial position.
- 25.Sitting up button: Upward the angle of the massage chair.
- 26.Calf mechanism rise button: Calf mechanism upward adjustment.
- 27.Calf mechanism down button: Calf mechanism downward adjustment.
- 28.Sitting down button: Downward angle of the massage chair.
- 29.Down button: Massage roller downward adjustment.
- 30.Width button: Massage width adjustment, 5 levels.
- 31.Speed button: Massage speed adjustment, 5 levels.
- 32.Up button: Massage roller upward adjustment.

# Operation Instructions

## I.Start Massage

1.Press the power button on the remote controller or the armrest short cut key to power it on.

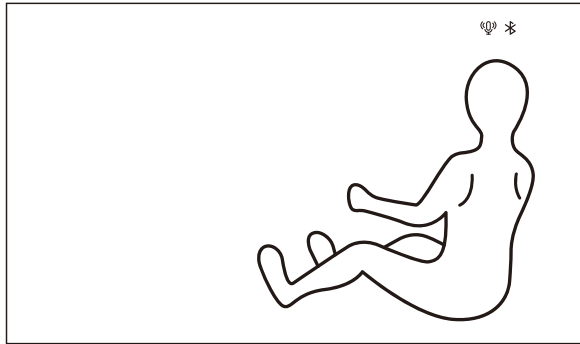


Power-on status diagram

- The massage chair will adjust the angle automatically after starting , then the human shoulder position will start to detect, at this time, please keep the body and sitting position and massage chair fit
- After the body shape detection is completed, the remote controller or voice prompt will prompt the user to adjust the shoulder position. If the massage roller stops at your shoulder position, you can directly start the massage without adjustment. If not, please use the up key and down key to adjust the massage roller to your shoulder position and then start the massage. If you do not operate for more than 10 seconds, the current position of the massage roller is directly default to the shoulder position and start massage.

Note: Press the corresponding key during startup or detection to switch to the default automatic program mode, press again to skip detection and directly enter the massage.



Shoulder position diagram

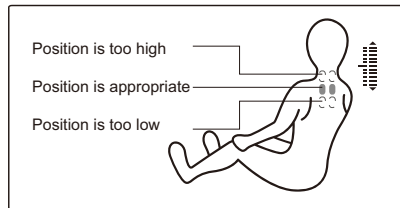

## II. Auto Mode(Please use this function after starting machine)

- Auto Mode: Select and click the area of automatic program to start to massage
- In the automatic program selection area, there are 8 automatic programs to choose from: Stretch massage, Full Body Massage, Comfort Massage, Neck&Shoulder massage, Waist&buttocks massage, Air massage, Relaxation massage, and Energy massage . Note: The automatic program has a background music function. After running the selected automatic program function, the background music will be automatically looped.

# Operation Instructions



Display diagram of remote auto program

Program name	Description	Display
Stretch	Using the calf frame and backrest frame to lift, stretch and pull the calf and waist, effectively relieve fatigue, relieve joint pain, restore vitality.	01
Full Body	Massage more comprehensive, a variety of techniques, all parts of the body for deep massage, relieve the whole body pain and fatigue, revitalize the body vitality.	02
Comfort	With SL guide rail and 3D movement, it can effectively stretch the back, stretch the body, effectively relieve muscle soreness, eliminate fatigue, achieve a better massage experience, and relieve physical and mental pressure	03
Neck&Shoulder	Massage for neck and shoulder, through the effective massage for cervical spine, scapula, relieve neck and shoulder pain, improve neck and shoulder	04
Waist&Buttocks	Mainly kneading,massage buttocks, and then through the tapping, kneading and tapping, deep massage buttocks and perineum, to relax the buttock muscles, play a role in lifting the buttock and slimming the waist, suitable for female massage, male prostate also has a maintenance role.	05
Air Massage	The whole air pressure massage is mainly, the air pressure action is more abundant, and the kneading and pressing of the air bag are used to promote blood circulation, reduce the pain of various body parts, and deeply relieve muscle fatigue.	06








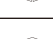


## Operation Instructions

Relaxation	Massage gently, massage the back and waist targeted massage, promote the back and waist blood circulation, dredge the meridians, relieve back and waist congestion. Suitable for people with back and waist pain.	01
Energy	It mainly massages the waist, shoulders and neck with kneading, tapping and kneading techniques, and deeply massages the whole body. The massage intensity is heavy, arouses the vitality of the human body and stimulates physical fitness. It is suitable for athletes, and plays a role in stimulating potential before the competition.	08

### III. Manual mode(Please use function after starting)

1. Massage technique: Select the manual mode button, and then select massage technique according to massage needs. There are 11 massage techniques to choose: stop, kneading, tapping 1, tapping 2, kneading&tapping, Shiatsu 1, shiatsu 2, shoulder lift, 3DI, 3DII, 3DIII, 3DIV keystroke cycle adjustment.



(1). Technique display

Function display	Massage roller adjustment	
Without technique	Width,intensity can be adjusted	<div>5 speeds can be adjusted</div> <div>5 widths can be adjusted</div> <div>5 intensity can be adjusted</div>
Kneading 	Speed,intensity can be adjusted	
Tapping I 	Speed,width and intensity can be adjusted	
Tapping II 	Speed,width and intensity can be adjusted	
Kneading&tapping 	Speed,intensity can be adjusted	
Shiatsu I 	Speed,width and intensity can be adjusted	
Shiatsu II 	Speed,width and intensity can be adjusted	
3D I 	Speed,intensity can be adjusted	
3D II 	Speed,intensity can be adjusted	
3D III 	Speed,width and intensity can be adjusted	
3D IV 	Speed,width and intensity can be adjusted	

## Operation Instructions



2. Massage speed: Select the **"speed"** button of the manual controller, and then select and adjust the massage speed according to the massage needs. The speed has a total of 1 to 5 optional, 1 level is the weakest and 5 level are the strongest, and the massage speed adjustment needs to be used in the following massage techniques: kneading, tapping I, tapping II, kneading&tapping, Shiatsu I, shiatsu II, 3DI, 3D II, 3D III, 3DIV.

(1). Speed Display

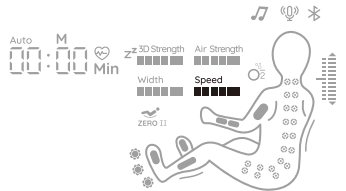Massage speed display	Massage Speed adjustment display
	<div>level 1 level 2 level 3 level 4 level 5</div> <div>  </div>



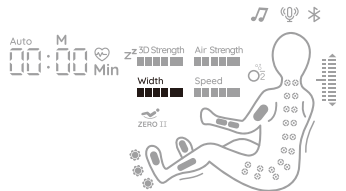


3. Massage width: Select the **"width"** button, and then adjust the massage width according to the massage needs. There are 1 to 5 widths to choose, 1 is the narrowest and 5 is the widest. Please use it in the following massage techniques: Without technique, tapping I, tapping II, Shiatsu I, shiatsu II, 3D III, 3DIV.

(1). Width display

Massage width display	Massage width adjustment display
	<div>level 1 level 2 level 3 level 4 level 5</div> <div>  </div>

4. Massage point: Select the **"Mode"** button of the remote controller, and then select the massage point according to the massage needs. There are 5 kinds of massage parts: fix, part, upper back, lower back and full back.


(1). Part display

Function		Description	
Fix		Set the position for a fix point massage	When selecting the fix and part point massage technique, the massage area can be adjusted by the up and down keys on both sides of the "back roller" on the remote controller.
Part		Small area round trip massage	
Full body		Round-trip massage of full back	
Upper body		Round-trip massage of upper back	
Lower body		Round-trip massage of lower back	

## Operation Instructions


5. Air pressure massage: Select the manual "air pressure" key, and then massage requirements to select the air pressure massage function. The air pressure mode has the upper body, lower body and full body pressure respectively, and the air pressure function is closed when all are closed.

(1). Air pressure display

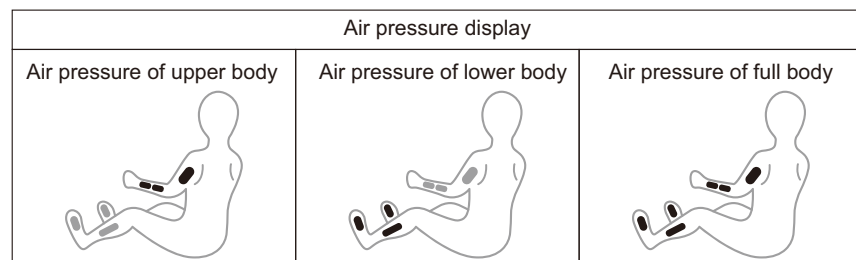

6. Pressure intensity: Select the "Intensity" button of the remote controller, and then adjust the pressure intensity according to the needs of the massage. The pressure intensity adjustment needs to adjust the pressure intensity after the pressure function is opened, and the pressure intensity can be adjusted in a total of 5 levels, 1 level indicates the weakest intensity and 5 level indicates the strongest intensity.


(1). Intensity display

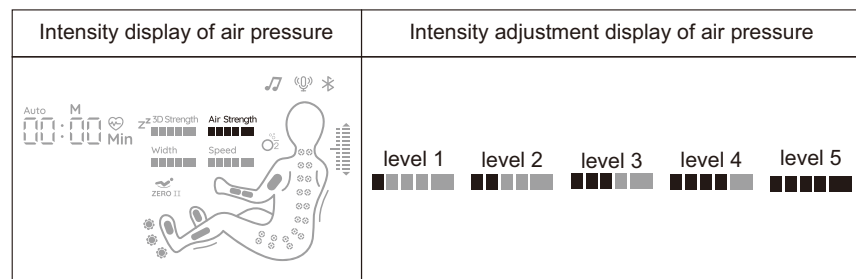

## VI. Auxiliary Function(Please display this function after starting)

1. Foot roller: Select the "foot roller" button of the remote controller to open or close the function of the foot roller according to the needs of the massage, and press continuously to cycle in fast, medium, slow and close modes.


(1). Foot roller display

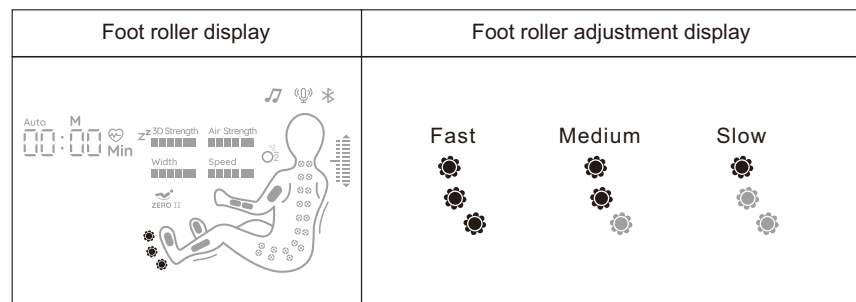

## Operation Instructions


2. 3D intensity: Select the "3D intensity" key of the remote controller, and then adjust the 3D intensity according to the needs of the massage. There are 1-5 levels of intensity to choose, 1 indicates the weakest intensity, and 5 indicates the strongest intensity.

(1). 3D intensity display

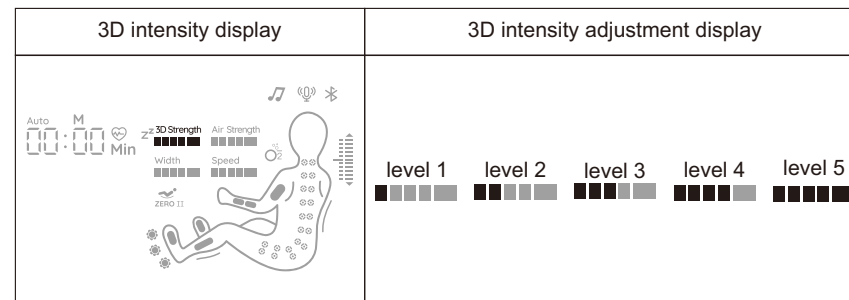

3. Shoulder position adjustment: Press the remote controller "up key or down key of shoulder position", and then adjust the shoulder massage position according to the massage needs. You can adjust it if you feel that it is not correct of shoulder position; Shoulder position adjustment A total of 11 adjustments.

(1). Shoulder position display

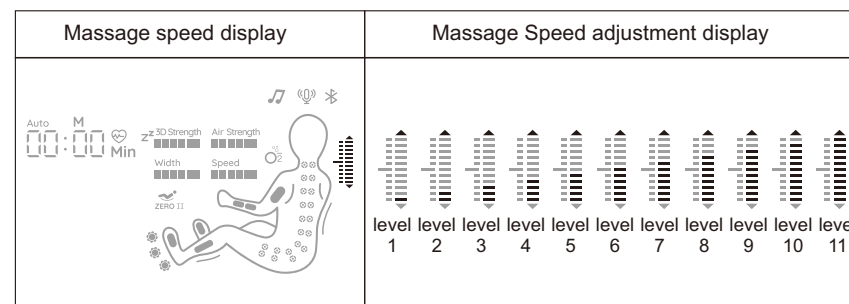


4. Volume adjustment: Long press the "shoulder position up key or down key" of the remote controller for 3 seconds, and then adjust it according to the volume of the music. The volume is adjustable from 1 to 10 levels.

5. Bluetooth: Long press the "timing/Bluetooth" key of the remote controller to turn on or off the Bluetooth function. When the Bluetooth function is turned on, the audio source device with Bluetooth function (such as mobile phone) can be paired and connected with the Bluetooth module of the massage chair, and the music played by the audio source device can be transmitted wirelessly to the massage chair audio system for music playback through Bluetooth.

(1). Enter the main menu of smart devices, open "Settings" - select "Bluetooth" option - open "Bluetooth" - Bluetooth automatically "search for new Bluetooth devices" to search.

# Operation Instructions

- The smart device will search for the corresponding Bluetooth model (such as IMCM-XXXX), click the pairing button, if you ask whether to pair with the device, select "pairing" to complete the pairing.
- Open the music player of the smart device and select the song to play.
- Bluetooth display.

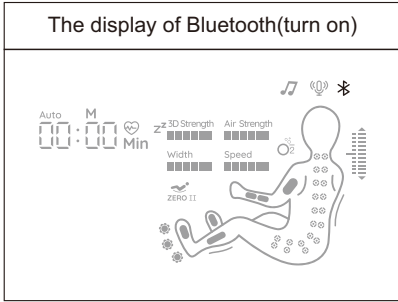

- Timing: Short press the "timing/Bluetooth" key, and then adjust the massage timing time according to the needs of the massage, each press, in 5 minutes superposition, not more than 40 minutes, if more than 40 minutes, return to the minimum time value.

Massage timing display	Massage timing adjustment display
	<div>05 Min 10 Min 15 Min 20 Min</div> <div>25 Min 30 Min 35 Min 40 Min</div>

- Heating: Short press the "heating/background music" button of the remote controller, and then turn on or off the heating function according to the massage needs. The heating parts include the the back and calf.


- Heating display

Heating display	Heating adjustment display
	<div>Calf heating</div> <div>Back heating</div>

# Operation Instructions

- Background music: Long press the "Heating/Background music" button of the remote controller, and then turn on or off the background music function according to the needs of the massage. After the background music function is turned on, after selecting the automatic program function, the background music will be automatically played in a loop.


- Background music display

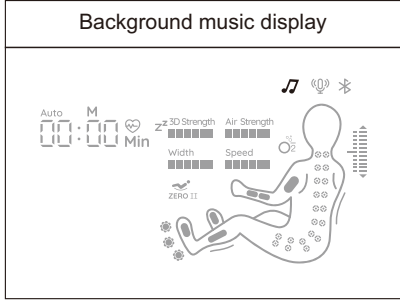

- Temperature: Short press the "temperature/mood lighting" button, and then adjust the heating temperature according to the needs; The temperature can be adjusted in normal temperature or low temperature. When the temperature is adjusted to normal, the icon of the heating part on the information interface does not blink, but blinks at low temperature.

- Mood lighting: long press the "temperature/mood lighting" key, and then open or close the mood lighting according to the massage needs.

- Voice control: Press the "Voice" button of the remote controller, press and hold to wake up the voice control function, and press and hold to turn on or off the voice control function. After opening the voice function, the massage chair can be controlled by voice according to the content of the voice entry.

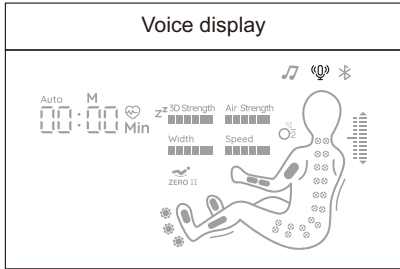

- When the massage chair is on, short press the "Voice " key on the remote to wake up the voice control function (if you cannot wake up the voice control, hold down the "Voice" key on the remote for 3 seconds to open the function). After the voice control is on, speak "Hi Alice" or "Hey Alice" near the speaker to wake up the voice control. When the voice assistant replies "I am here", it means that it has been awakened. After opening the voice control function, you can use the voice command to control the massage chair.

## Operation Instructions

- (2).Or short press voice button in the armrest shortcut key to wake up the voice assistant, and hold down the key for 3 seconds to turn on/off the voice assistant.
- (3).After waking up the voice assistant, please say the command within 6 seconds to achieve the functional control of the massage chair, and after the voice assistant responds to the command, you can continue to say the command, the interval between the two adjacent commands cannot exceed 6 seconds, otherwise you need to wake up the voice assistant again.

The detailed voice command table is as follows:


Note: Please use English when waking up the voice control function. The speed of speech should not be too fast, and the language command should be issued according to the voice command table on the back of remote controller.

Voice Control Entry		
Entry: "Hi Alice" or "Hey Alice". Answer: I am here.		
No.	Voice control entry	Answer entry
1	Massage close	Answer: Ok, Massage close
2	Massage on	Answer: Ok, Massage on
3	Stretch	Answer: Ok, Stretch
4	Full Body	Answer: Ok, Full Body
5	Comfort	Answer: Ok, Comfort
6	Neck&Shoulder	Answer: Ok, Neck&Shoulder
7	Waist&Buttocks	Answer: Ok, Waist&Buttocks
8	Air Massage	Answer: Ok, Air Massage
9	Relaxation	Answer: Ok, Relaxation
10	Energy	Answer: Ok, Energy
11	Fixed massager	Answer: Ok, Fixed massager
12	Move Down	Answer: Ok, Move Down
13	Move Up	Answer: Ok, Move Up
14	Change the other mode	Answer: Ok, Change the other mode
15	Open the air pressure	Answer: Ok, Open the air pressure
16	Close the air pressure	Answer: Ok, Close the air pressure
17	Open the foot roller	Answer: Ok, Open the foot roller
18	Close the foot roller	Answer: Ok, Close the foot roller
19	Up the seat position	Answer: Ok, Up the seat position
20	Down the seat position	Answer: Ok, Down the seat position

## Operation Instructions

### 12.Memory function

- (1).Storage: When massage chair is turned off, the "Memory (M in English)" on the display of remote will flash for 10 seconds, short press M1, M2, M3 during the flashing period will store the massage program before the end on its corresponding level.
- (2).Call: In the massage process, long press M1, M2, M3 will run the massage program which stored on the corresponding level, if the massage program on the corresponding level is empty, it will not run and emit a warning sound.
- (3).The information interface of remote controller will display time.

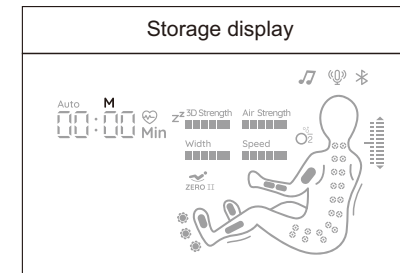

### X.Angle adjustment (Please use this function after massage chair is turned on)

- 1.Zero gravity button: A total of 2 level can be adjusted, corresponding to 2 angles. Press the Zero gravity button of the remote controller to adjust 2 zero gravity level or close zero gravity.

Zero gravity display	Zero gravity adjustment display
	<div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;"> <p><b>ZERO I</b></p> </div> <div style="text-align: center;"> <p><b>ZERO II</b></p> </div> </div> <p>Note: No icon is displayed when zero gravity is closed</p>



### 2.Full body angle

Button	Description
	Sitting down button: Press the sitting down button of the remote controller to adjust the angle of the massage chair to downward, the backrest mechanism drops, the calf mechanism rises, and releasing the button will stop the adjustment; When adjusting to the limit position, the adjustment stops and a "drop" tone is sounded.
	Sitting up button: Press the sitting up button of the remote controller to adjust the angle of the massage chair to upward, the backrest mechanism rises, the calf mechanism falls, and releasing the button will stop the adjustment; When adjusting to the limit position, the adjustment stops and a "drop" tone is sounded.



## Operation Instructions

### 3. Calf mechanism angle


Button	Description
	Calf mechanism drop key: Press the calf mechanism drop key of the remote controller to start downward adjustment of the angle, and release the key to stop adjustment; When adjusting to the limit position, the adjustment stops and a "drop" tone is sounded.
	Calf mechanism rise key: Press the calf mechanism rise key of the remote controller to start to adjust the angle, and release the button to stop the adjustment. When adjusting to the limit position, the adjustment stops and a "drop" tone is sounded.

### VI. Finish massage

During the massage:

1. Short press the power button of the remote controller, all massage functions are turned off, and the Angle of the massage chair is reset.
2. Long press the power button of the right armrest, all massage functions are turned off, and the Angle of the massage chair is reset.
3. Use voice command to shut down, all massage functions off, massage chair angle reset.
4. The timing function is over, all massage functions are closed, and the angle of the massage chair is no longer reset.

Cut off the power supply of the whole machine, attached figure (schematic diagram of the power supply of the whole machine).

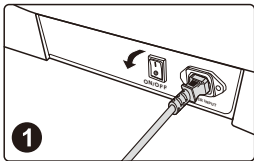

Power off switch  
("0" position)


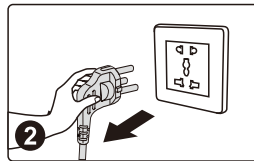


Diagram of unplugging  
the power

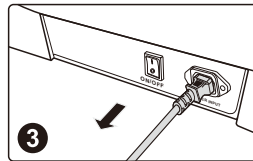

Unplug the power cable  
connecting the product

## Cleaning and maintenance

### Clean synthetic leather and plastic parts

Wipe the leather with a soft, dry cloth

- Do not use chemicals such as thinner, gasoline, alcohol to clean the machine.
- When using commercially available leather care products (rags), please follow their instructions.
- If synthetic leather is particularly dirty, wipe it by the following means.  
The soft cloth is immersed in water or hot water containing 3% to 5% neutral detergent, and then thoroughly wrung out.  
Wipe the leather surface with the soft cloth.  
Wipe off the cleaning solution with a cloth soaked in water and thoroughly wrung out.  
Wipe with a wrung out soft cloth.  
Let it air dry.
- If it is difficult to remove stains, please buy melamine foam soaked in neutral detergent and wipe the machine with it.
- Do not use a hair dryer to dry the surface.
- Because the color of the fabric will stain the surface of the synthetic leather, it is important to be careful when using denim and colored fabrics to wipe the machine.
- Do not keep the leather in contact with the plastic for a long time, as this will cause fading.
- Synthetic leather has the potential to discolor, so when applying hair dye, cover the contact with a towel.



## Troubleshooting

<b>Malfunction</b>	When the calf mechanism or backrest reaches a certain position, press the corresponding button and no longer change, and emit a continuous "beep" sound.
<b>Solution</b>	Due to the requirements of product structure design and ergonomics, this product has a restrictive protection design for the limit state of attitude adjustment. A warning sound will be sounded when the limit is reached at the same time.
<b>Malfunction</b>	After starting of the machine, the functions or some functions are not running.
<b>Solution</b>	After the machine is opened by pressing the key, it is necessary to manually select the automatic massage program or other working status, if no operation, the machine will automatically shut down after 20 minutes. If the manual mode is selected, the function that is not selected may be disabled. You need to set the working mode manually.
<b>Malfunction</b>	The machine makes leather friction sound when adjusting posture.
<b>Solution</b>	This product adopts the advanced anti-corrosion leather that meets the requirements of environmental protection. Due to the inherent characteristics of the leather material, the friction at the relative movement will produce a normal slight sound, which is a normal phenomenon. Note: If you hear fabric or leather tearing when the machine is running, please stop using it immediately and seek a designated professional for repair.
<b>Malfunction</b>	The massage roller does not reach the shoulder or neck position.
<b>Solution</b>	If the head does not touch the headrest, or the back does not touch the backrest, the shoulder position may be lower than the actual position during the body shape test. Sit in the deepest position of the seat, rest your head on the headrest, and re-operate from the beginning.
<b>Malfunction</b>	The machine has been damaged The power cable or plug is abnormally hot.
<b>Solution</b>	In order to prevent accidents, please be sure to contact online customer service.

Please feel free to contact online customer service if you have any questions about this product.

## Product specifications

Name: Massage Chair  
Configure: 11  
Rated voltage: 110-120V~  
Rated frequency: 60Hz Rated  
power: 160W  
Rated time: 20 min  
Safety structure: Class I