
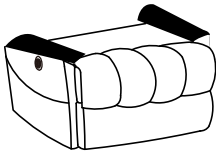
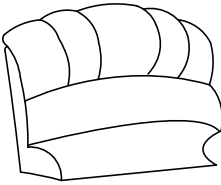
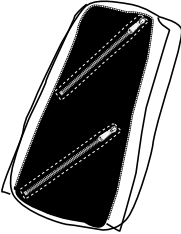
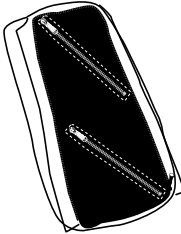
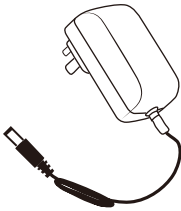
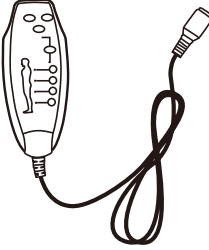
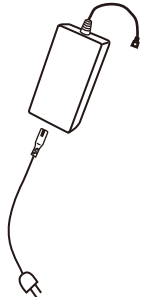
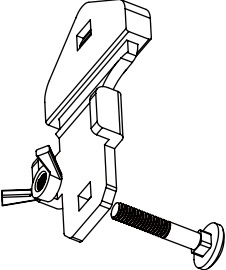



Massage Lift Chair Assembly Instruction

A 	B 	C 	D 	E 	F 	G 	H 
Chair base	Backrest	Left Armrest	Right Armrest	Power cord	Massage Remote	Power Source	Anti-fall bracket x2


STEP 1

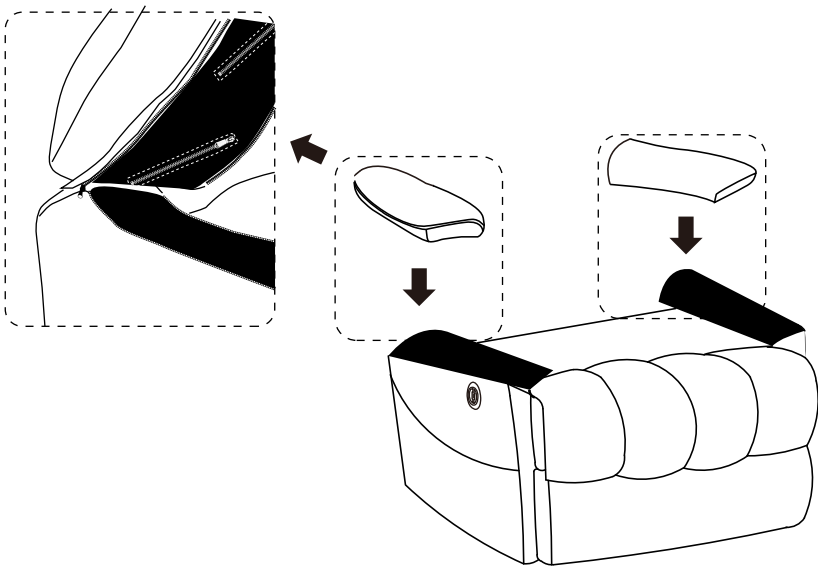



Zip the zipper to make sure both armrest cushions are well attached to the arm.

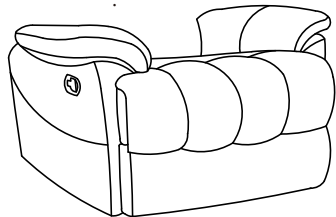

STEP 2

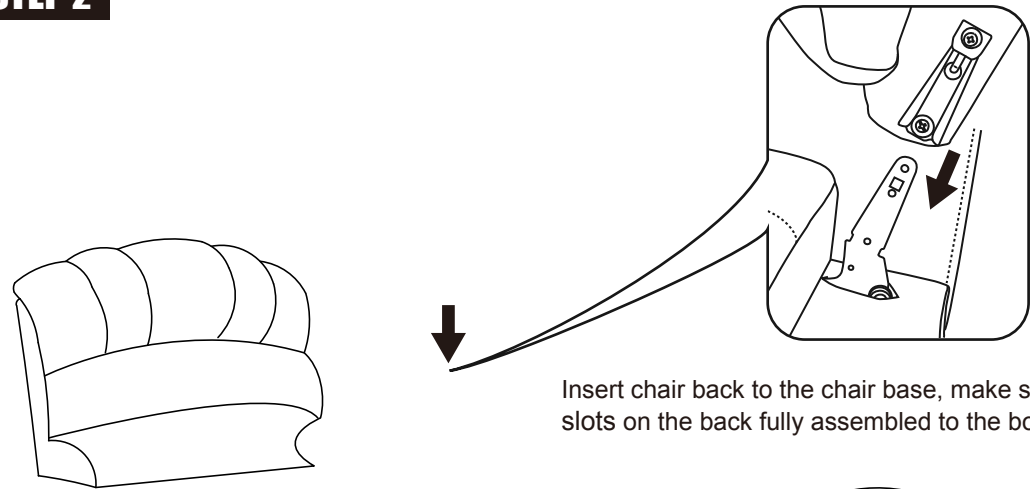

Insert chair back to the chair base, make sure the slots on the back fully assembled to the bottom part.

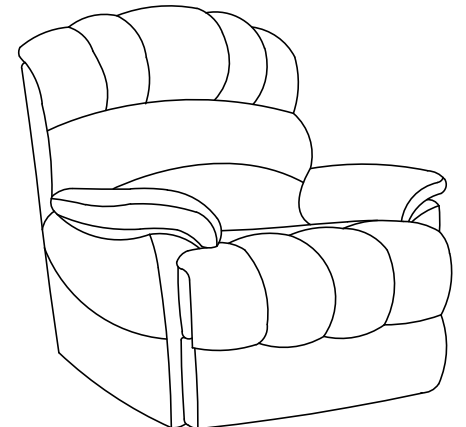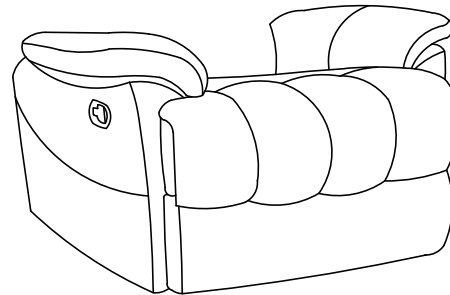


Massage Lift Chair Assembly Instruction

STEP 3

- No.1 and No.2 wire connect with the backrest and chair base.
- No.3 wire connects with backrest and power cord(E).
- No.4 wire connects with backrest and massage remote(F).
- No.5 wire connect the motor and the wire from Right Armrest, which use to connect the control panel.
- No.6 wire connect the motor and power source (G).
- No.7 butterfly nut up,screw bolt down (H).


NOTE

- 1: Two same color wires connect.
- 2: Plug power cord and power source into outlet, then you can use all function.

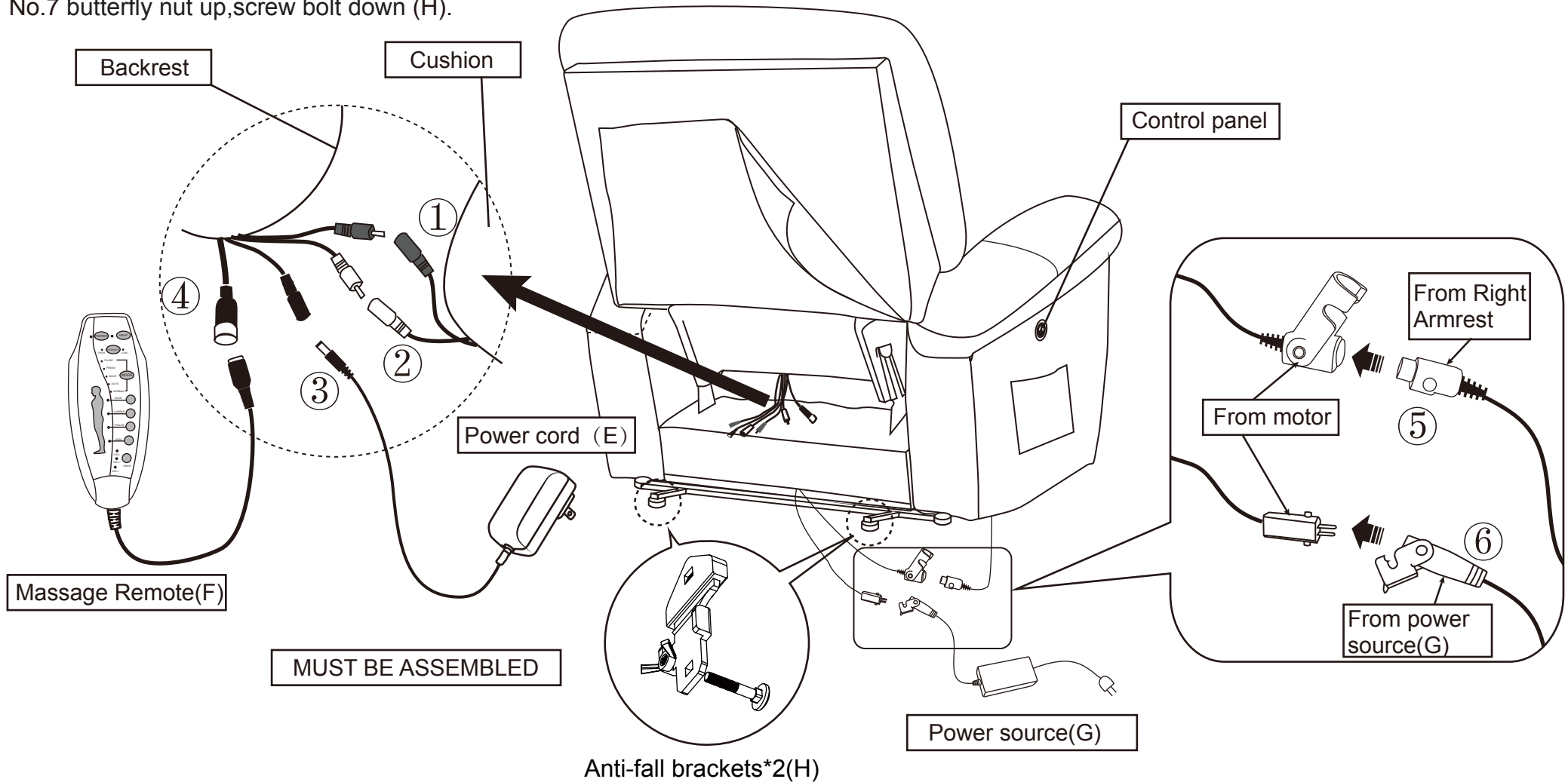


Massage Lift Chair Assembly Instruction

How to recline?

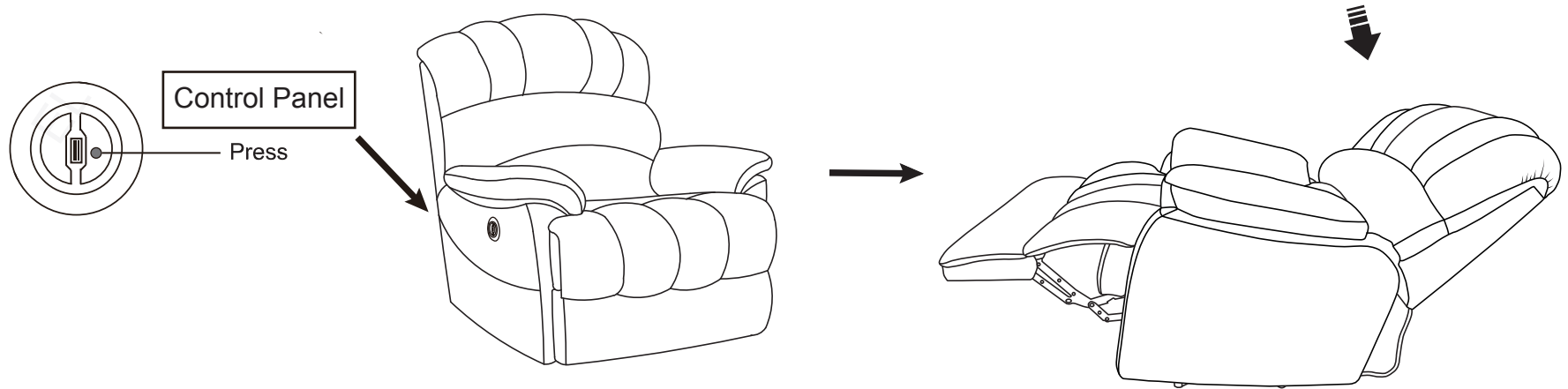

How to get footrest latch down and stand?

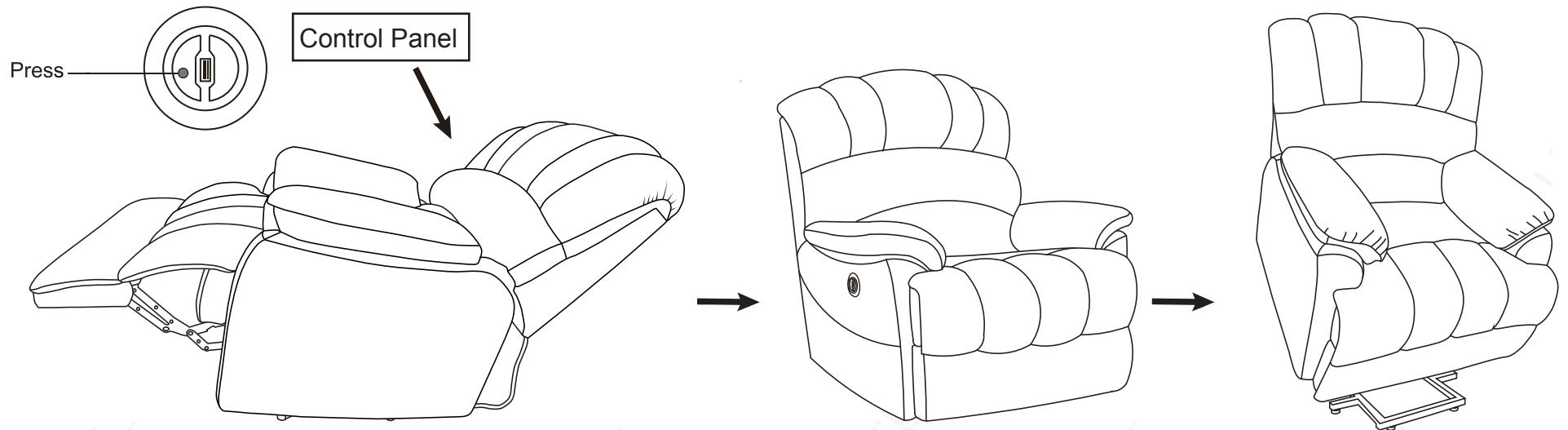

How to use remote control?

