
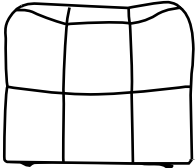
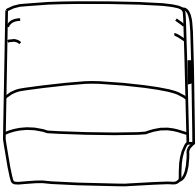
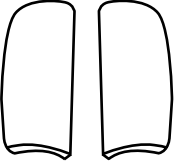
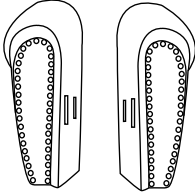
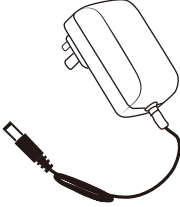
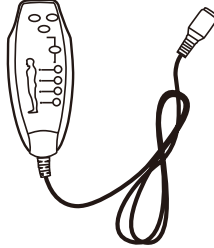
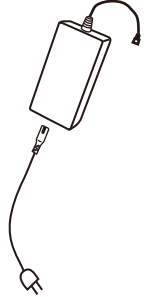
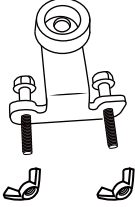



# Massage Lift Chair Assembly Instruction

							
Chair base	Backrest	2*Ears	Both Armrests	Power cord	Massage Remote	Power Source	Anti-fall bracket x2


## STEP 1

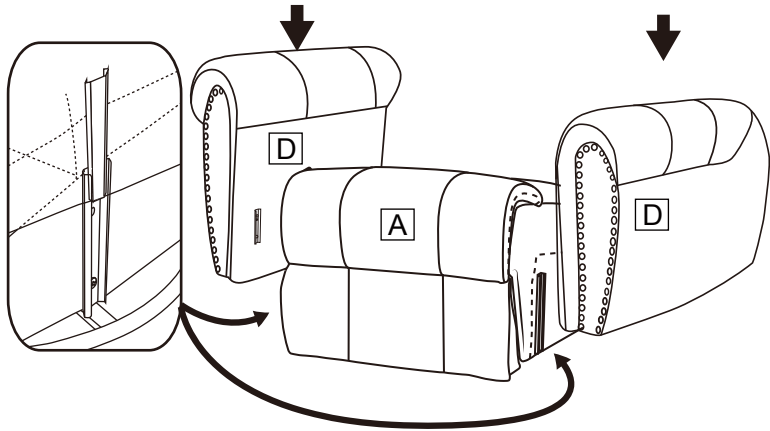




Insert two armrests to the chair base, make sure the slots on the armrests fully assembled to the bottom part.

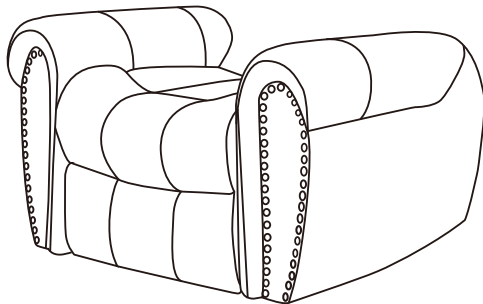

## STEP 2

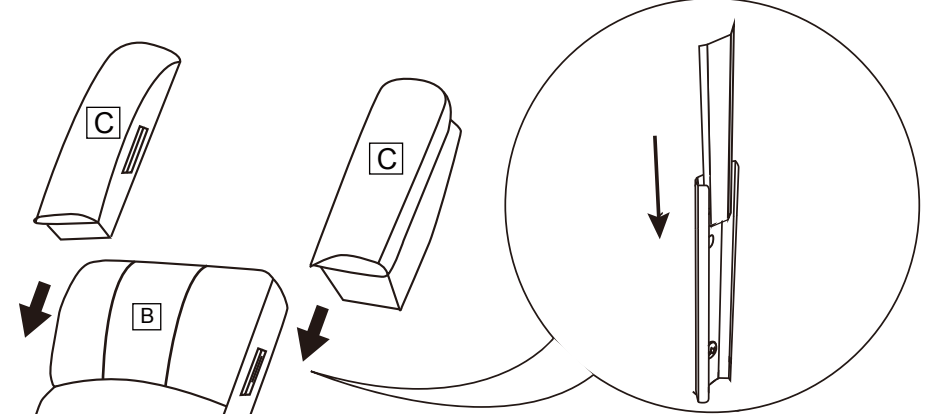


Insert chair back to the chair base, make sure the slotts on the back fully assembled to the bottom part.

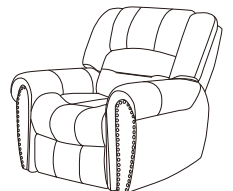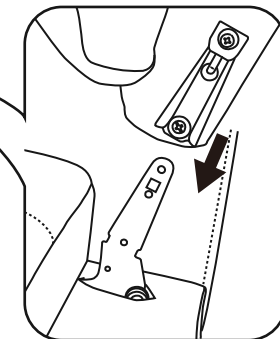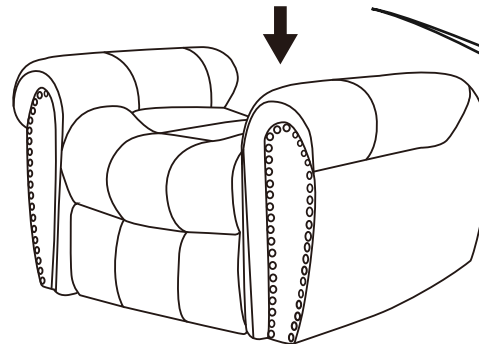




# Massage Lift Chair Assembly Instruction

## How to recline?

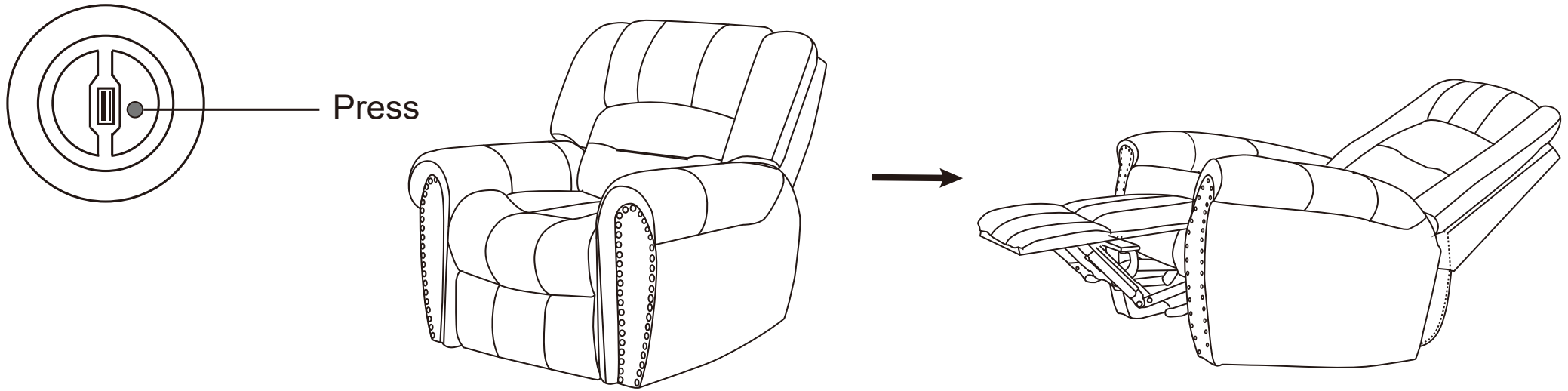

## How to get footrest latch down and stand?

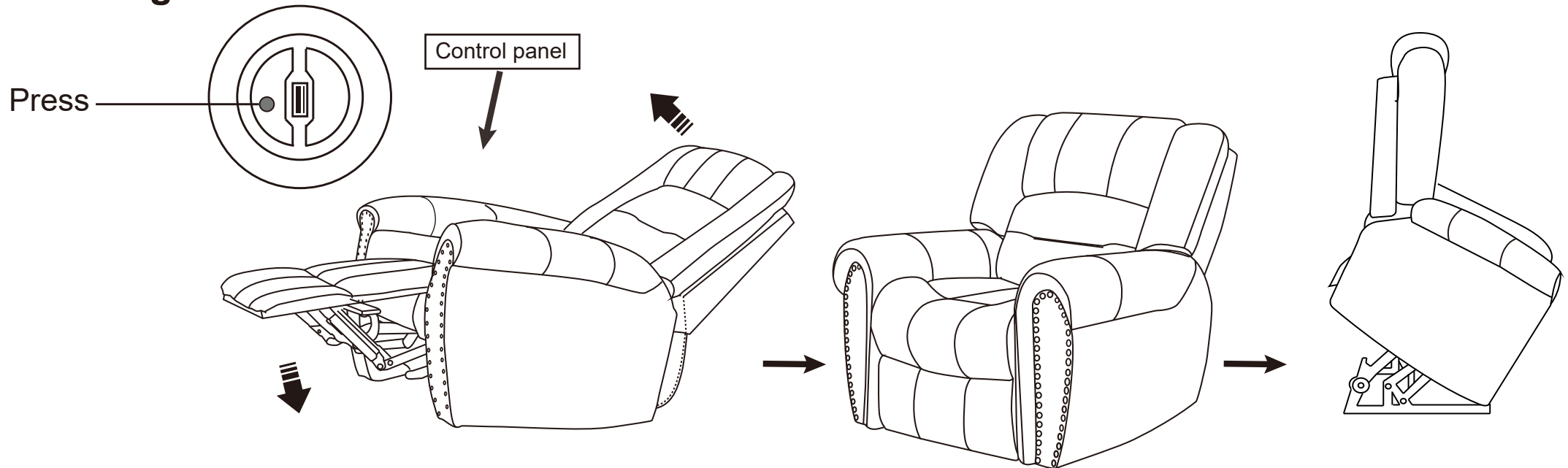

## How to use remote control?

