
# INVISIA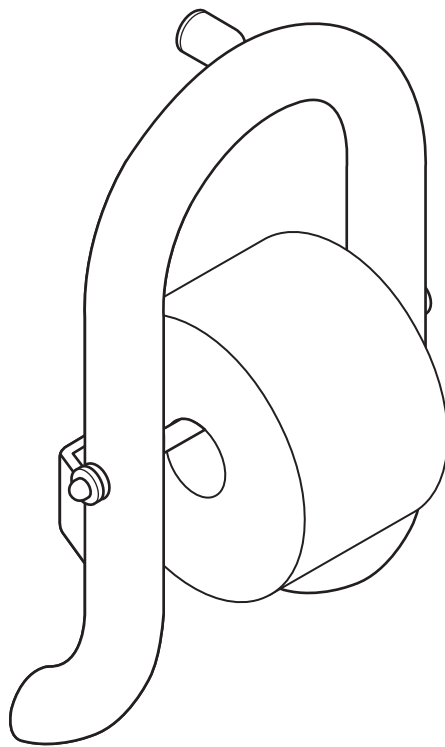

## **Wall Toilet Roll Holder**

INV-WTRH  
(DC156 - RevF)

[healthcraftproducts.com](http://healthcraftproducts.com)

# English

## OVERVIEW

Thank you for choosing the INVISIA Toilet Roll Holder. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your INVISIA Toilet Roll Holder is properly installed and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your INVISIA Toilet Roll Holder installed by a qualified contractor.

The INVISIA Toilet Roll Holder is a combination grab bar and towel bar designed to disguise safety in a bathroom accessory. It is intended to be used as a grab bar and its secondary function is to hold toilet paper. It is intended for moderate vertical and horizontal load-bearing, providing sitting and standing support for people with disabilities. It should be installed with appropriate fasteners to a structural wall. The axle (which holds a roll of toilet paper) is NOT intended to be a support product. The maximum user weight for this product is 500lbs/227kgs.

NOTE: Product is not intended to support full body weight. The INVISIA Toilet Roll Holder is not to be used in any other way than described above.

## LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, rusting due to water exposure, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon.

Specifications subject to change without notice.

Manufacturer: HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. 1-888-619-9992. Patent Pending. © 2026 HealthCraft Products Inc.

## DELIVERY CONTENTS

## FIGURE A.

- |                                  |                                  |
|----------------------------------|----------------------------------|
| 1. Grab Bar - Toilet Roll Holder | 7. Lock Washer, S/S (x1)         |
| 2. Wall Stem Assembly            | 8. Plastic Cap (x3)              |
| 3. #10-32 Set Screw              | 9. #10 Wood Screw, S/S (x1)      |
| 4. Paper Axle                    | 10. #10 Wood Screw CSK, S/S (x3) |
| 5. Center Punch                  | 11. 3/16" Bolt, S/S (x2)         |
| 6. 3/32" Hex Key                 | 12. Hollow Wall Anchor (x2)      |

## INSTALLATION WARNINGS

## FIGURE B.

1. Bar must be installed to walls that are structurally sound. If desired installation location does not have adequate structure, add wood blocking.
- 2a. CAUTION - Do not install if there is no structure behind drywall or if there is no structure for one side.
- 2b. For hollow wall scenarios, ensure center screw holes align with vertical wood stud and use appropriate fastener combination (Note: reduced weight capacity when not full structure).
3. CAUTION - Do not install into metal studs.
4. CAUTION - Do not install if structure behind walls is unknown.

## SUGGESTED PRODUCT LOCATION

## FIGURE C.

To determine the optimal mounting location, we recommend holding the bar in place and walking through the motions of a transfer (sitting, standing, reaching, etc.). ADA recommended bar height of 33-36" / 838-914mm.

## INSTALLATION

## FIGURE D.

1. Back off set screw from bar socket.
2. Allow wall stem to freely move in and out of bar socket.
3. Peel off label on back of wall stem to expose sticker.
4. Mark location of top wall stem.
5. Drill pilot hole appropriate for fastener.
6. Fasten the wall stem to the wall using supplied lock washer and fastener.
7. Place bar back onto the fastened wall stem.
8. Using wall plate as guide, mark location of bottom holes. Ensure level.
9. Drill pilot holes appropriate for fasteners.
10. Place bar back onto the fastened wall stems, and secure bar to wall stem by tightening set screws.
11. Secure the wall plate to the wall using appropriate fasteners (x3).
12. Cover exposed screw heads with push on plastic caps (x3).  
Inspect hardware regularly, and tighten if required.

## PRODUCT USAGE

## FIGURE E.

CAUTION - Grab bar and bathtub are slippery when wet.

## TECHNICAL DATA

## FIGURE F.

\*Complies with ADA 609.2 Cross Section - Grab Bars.

\*Complies with ADA 609.3 Spacing - Grab Bars.

\*Complies with ADA 609.8 Structural Strength - Grab Bars.

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