

Stair Stretcher Instructions for Use

I. Technical Parameters

1. Product Size (L×W×H): 90 × 51 × 91cm
2. Folded Size (L×W×H): 90×17×59cm
3. Package Size (L×W×H): 96×17×64cm
4. Net Weight: 8kg Gross Weight: 9.2kg
5. Load Capacity: ≤ 160kg

II. How to Use

For emergency personnel, this is the ideal facility for transferring patients up and down stairs. The two fixed rear wheels enable the patient to be moved with minimal

The two fixed rear wheels enable the patient to move over most surfaces with minimal effort. The stretcher comes with three restraint straps with quick-release buckles to ensure patient safety during transportation. Under the front of the stretcher, it has two flexible lifting bars. The stretcher can be folded, saving the ambulance space.

III. Application

Approved by clinical practice, this item is suitable for ambulances, and hospital battlefield gyms for transporting casualties and patients. It can also be used for general operations.

Precautions.

1. When using this stretcher to transport patients, please ensure that the safety belt is securely locked to ensure safety
2. In general operation, please be careful not to tear the surface of the stretcher.

IV. Maintenance

1. Maintain daily cleanliness (including sterilization).
2. Check frequently if the parts are loose.

V. Storage and Transportation

1. Store this product in a moisture-proof and corrosion-proof place.
2. Ordinary transport vehicles can carry the product.

VI. Packaging.

1. Packed in a highly durable carton with a wooden bottom.
2. Package: Inspection certificate, instruction manual, and packing list.