





1301 Westinghouse Blvd Suite I
Charlotte, NC. 28273

Email: support@csibathware.com
Phone: 704-843-9292
Fax: 704-943-0675

UNIQUE TOUCH INFINITY GRAB BAR

- Material: 100% Optical Quality Acrylic
- Available Colors:
 - Translucent Clear
 - Opaque Snow
 - Opaque Black
- Dimensions:
 - 12" Bar
 - Center to Center = 11.97" (304 mm)
 - Total Length = 13.23" (336 mm)
 - 16" Bar
 - Center to Center = 15.95" (405 mm)
 - Total Length = 17.2" (437 mm)
 - 24" Bar
 - Center to Center = 24" (610 mm)
 - Total Length = 25.25" (641 mm)

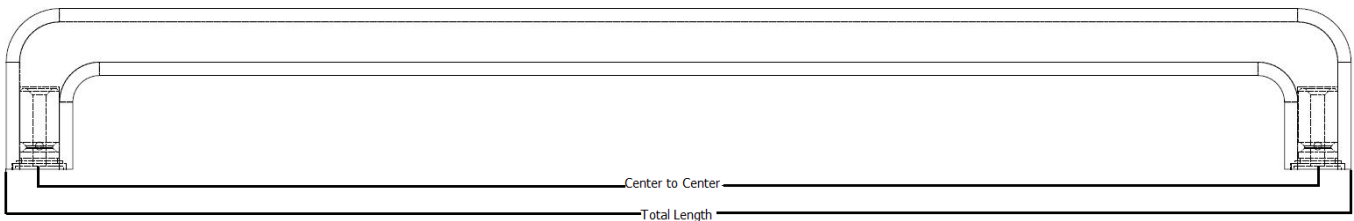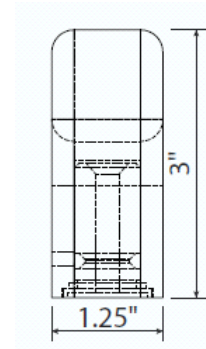

IMPORTANT INSTRUCTIONS

Installation must ALWAYS comply with your state and local codes



READ AND SAVE THESE INSTRUCTIONS



Read carefully before attempting to assemble, install, operate or maintain this product. Protect yourself and others by observing all safety information. Failure to comply with instructions could result in personal injury and/or property damage! Retain instructions for future reference

COMPLIANCE

To prevent serious injuries, do not use this bar for individuals weighing more than 300 lbs.

CARE

The Unique Touch grab bar is 100% cast acrylic. Clean using a soft cloth and a mild soap containing no abrasives or solvents. For minor touch ups of scratches, use a polishing compound made for acrylic surfaces.

WARNING

Concealed in-wall support backing is ALWAYS REQUIRED for this product (not included and must be selected and installed by a licensed professional).

Avoid damage to enclosures, walls, tiles, fasteners and support structure by not over tightening the fasteners.

CAUTION

Personal injury and or property damage may result from improper installation practices such as inadequate support and fasteners.

SAVE THESE INSTRUCTIONS



1301 Westinghouse Blvd Suite I
Charlotte, NC. 28273

Email: support@csibathware.com

Phone: 704-843-9292

Fax: 704-943-0675

INSTALLATION INSTRUCTIONS:

WARNING


READ ALL SAFETY WARNINGS BEFORE INSTALLING THE GRAB BAR

1. Confirm contents of included hardware as all follows:
 - 2 Metal Inserts
 - 2 Rubber Washers
 - 2 Stainless Steel Screws
 - 2 Set Screws
 - 1 Allen Wrench
2. Turn bar over and view the two mounting holes - one at each end of grab bar. Measure the distance center to center of these holes. This measurement should correspond to the measurement shown on the product packaging. Choose the location for the grab bar on the wall and using a measuring tape, place a mark indicating the center of each hole. If installing the bar in a horizontal *or* vertical position, use a level to ensure accuracy and be sure to recheck center-to-center measurements before drilling holes.
3. Drill two 1/8" holes having the same center-to-center distance as the bar.
4. Place rubber washer over narrow end of metal insert
5. Roll rubber washer down metal insert until it is seated in the shallow groove.
6. Insert stainless steel screw into narrow end of the metal insert.
7. Insert screw into pre-drilled hole and tighten until assembly is firmly supported against the wall.
8. Repeat steps 3-6 for second metal insert and pre-drilled hole.
9. Apply bead of clear sealant around bottom edges of grab bar. It is very important to seal the space between the bar and the surface it is attached to with silicone if it is in contact with water.
10. Install the grab bar on the metal inserts with the set screw holes facing the floor.
11. Insert set screw (1) in the hole using the included allen wrench until the screw touches the insert.

STOP SCREWING WHEN THE SCREW TOUCHES THE INSERT.

Do not force the screw or over tighten - the acrylic will crack.

Installation on hard wall or in a stud

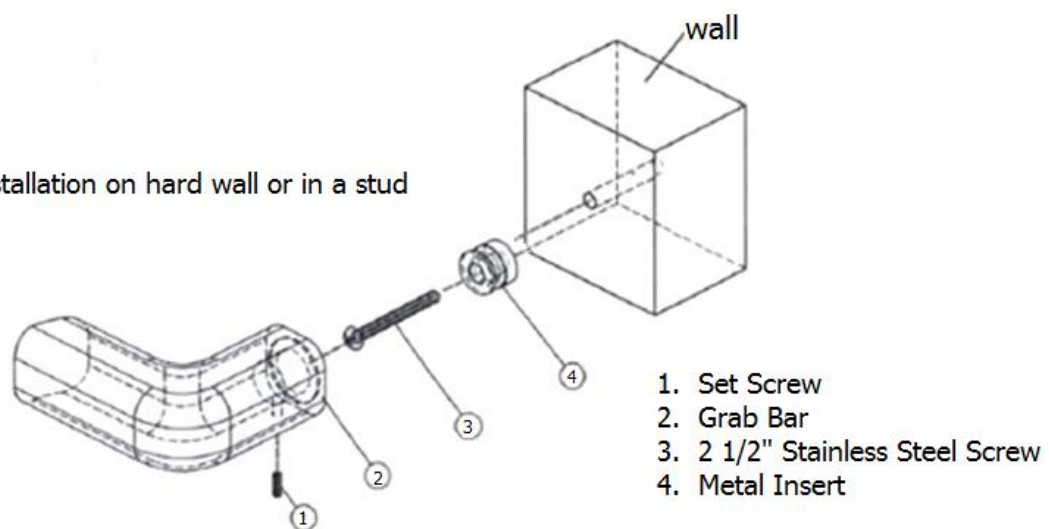



WE APPRECIATE YOUR BUSINESS!

INFINITY