

# **VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

Technical Support and E-Warranty Certificate [www.vevor.com/support](http://www.vevor.com/support)

## **TUB TRANSFER BENCHES**

We continue to be committed to provide you tools with competitive price. "Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

# VEVOR<sup>®</sup>

TOUGH TOOLS, HALF PRICE

TUB TRANSFER BENCHES

MODEL:FBL650328





## NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:


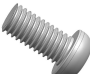








**Technical Support and E-Warranty Certificate**  
**[www.vevor.com/support](http://www.vevor.com/support)**

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Symbol	Symbol Description
	<b>Warning:</b> To reduce the risk of injury, the user must read the instructions manual carefully.
	This symbol, placed before a safety comment, indicates a kind of precaution, warning, or danger. Ignoring this warning may lead to an accident. To reduce the risk of injury, fire, or electrocution, please always follow the recommendation shown below.

## PART LIST


When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.

NO	PICTURE	PART NUMBER	QTY	NO	PICTURE	PART NUMBER	QTY
①		Seat frame	1	⑥		screw	4
②		Chair foot	4	⑦		Armrest	2
③		Backrest	1	⑧		Shower buckle	1
④		trestle	2	⑨		Shim	4
⑤		Seat frame tube	2	⑩		safety belt	2


## SEMBLY INSTRUCTIONS

### 1. Installation Steps


1.Thread the two support bracket tubes ⑤ through the two tubes on the front and rear sides of the bracket ① in sequence (note that the two holes in the middle of the bracket tubes are facing the outside of the product).

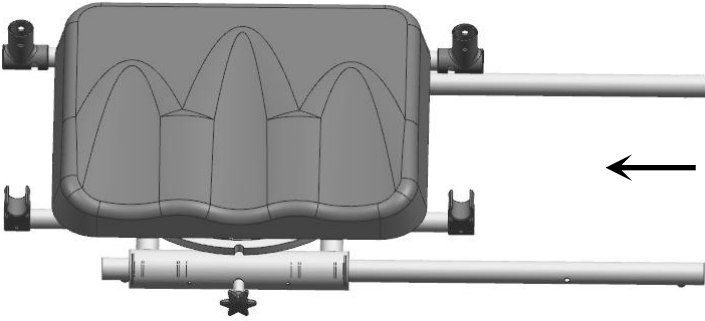


2.Align the pipe openings of two brackets ④ with the pipe openings of bracket pipe ⑤ and insert them (ensure that the holes of the bracket pipe openings are aligned with the holes of the bracket pipe openings).

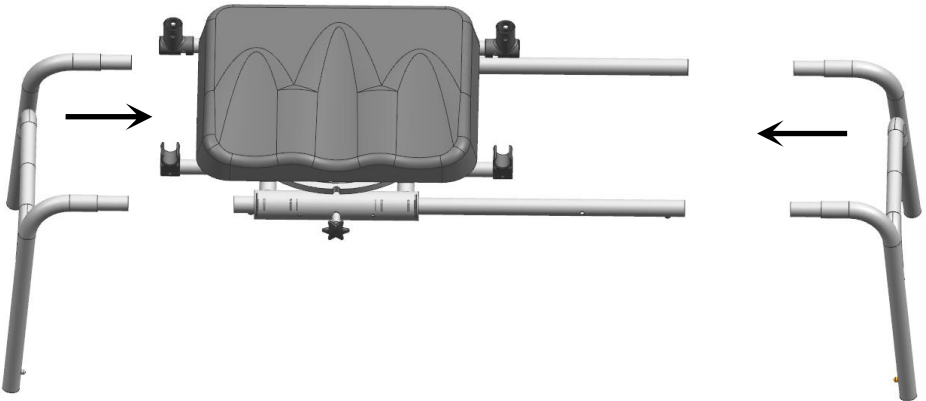


3. Secure bracket ④ and seat tube ⑤ with four screws ⑥ and washers ⑨ in sequence.

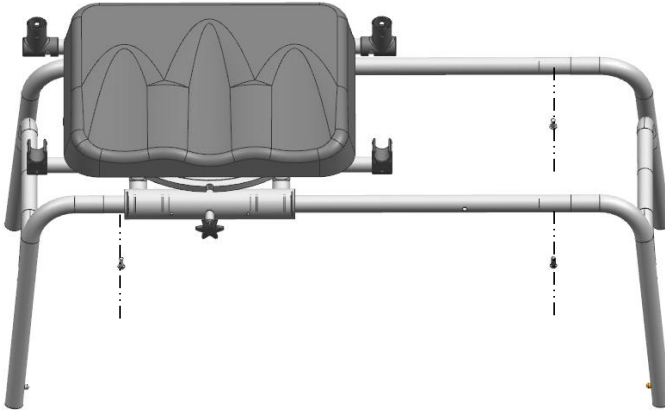


4. Invert the assembled main frame on a flat surface and insert the four chair legs ② into the bracket ④ in sequence (ensuring that the marble on each chair leg pops out normally).

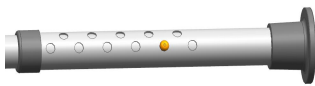

The chair legs are adjustable in 11 gears, and users can adjust the position of the chair leg ball hole to meet the height requirements of the seat.

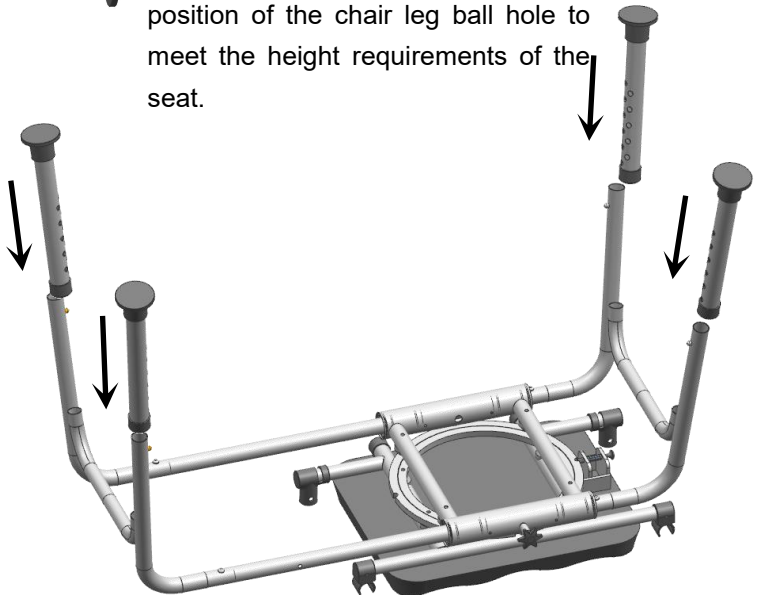

5. Place the main frame on a flat surface and insert the two handrails ⑦ into the plastic on both sides of the frame (ensuring that the marbles on each armrest pop out normally).

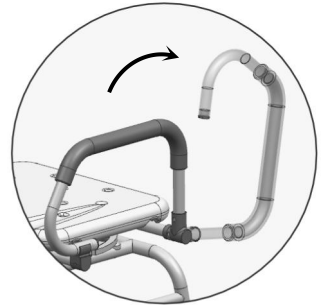


When the armrest is not needed, you can use force to pull it up and rotate it back in the direction of the arrow.

6. Align the backrest ③ with the pipe opening on the back of the seat frame ①, and then insert it (ensure that the marble of each backrest pipe pops out normally).




Two seat belts can be fixed to each of the two backrest tubes to secure the user's body.

7. After the product assembly is completed, a pull rod can be seen below the right side of the seat plate. When pulling the pull rod, the direction of the seat plate can be adjusted by rotation. After releasing the pull rod, the seat plate will rotate in one direction and fix its direction by itself.

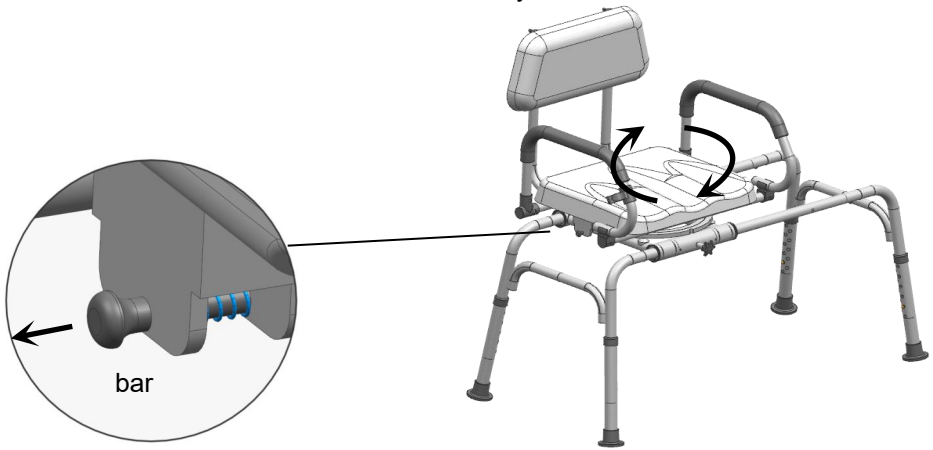

There is a knob below the front of the seat plate, which can slide on the main frame when pulled. After releasing the knob, the seat plate will automatically fix its position when it slides to the other end.

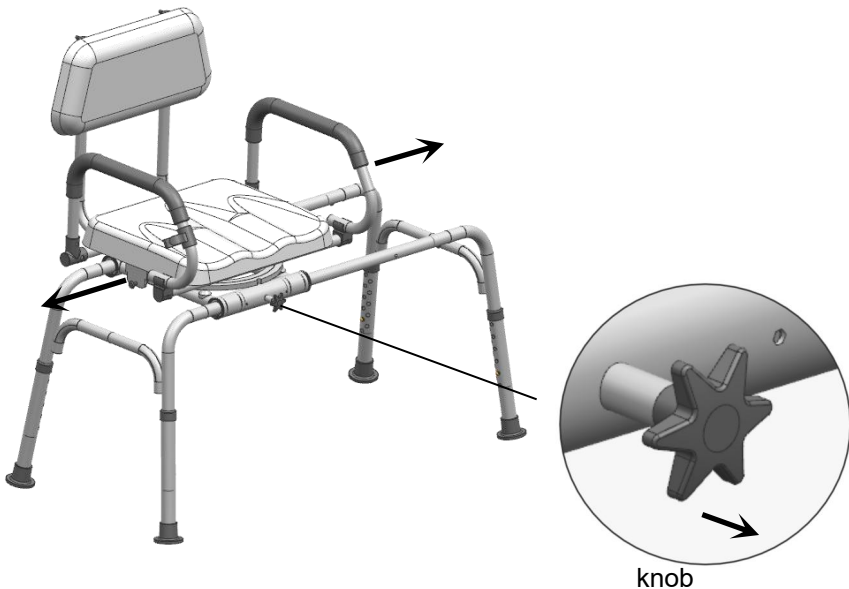

## SUBSEQUENT CONSULTATION: READ CAREFULLY

### Warning:

- \*Put the chair stably on a flat floor
- \*Building the chairs in good condition.
- \*Stay away from open flames.
- \*Avoid the overload at the chair.

### Precautions before assembly:

- \*Prepare a clean and dry mounting area.
- \*Verify that the product is not damaged.
- \*Check that none of the pieces are missing.

### Maintenance:

- \*Clean with a soft fabric.
- \*Regularly remove leaves and or water collected on the canvas.
- \*Check the seating regularly.


### Storage conditions:

- \*The fabric should be stored completely dry, in a clean and dry place to avoid the risks of mold.

Caution with the risks of pinching or cuts during assembly.

The manufacturer declines all responsibility in case of damages due to a bad installation.

### Weight limited

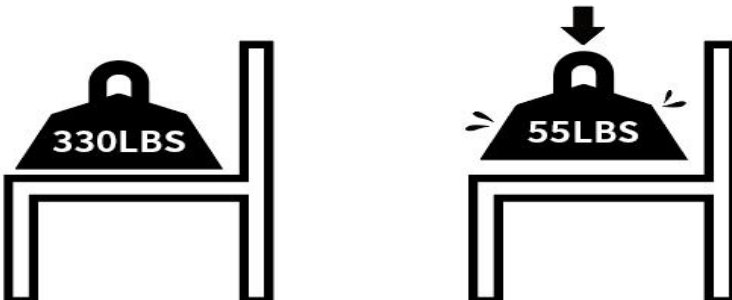

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd  
Address: 2F-2, 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin  
Community, Xiaolan, Zhongshan, Guangdong, 528414, CHINA

**Imported to AUS:** SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD  
NSW 2122 Australia

**Imported to USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim  
Place, Rancho Cucamonga, CA 91730



YH CONSULTING LIMITED.  
C/O YH Consulting Limited Office 147, Centurion  
House, London Road, Staines-upon-Thames,  
Surrey, TW18 4AX



E-CrossStu GmbH  
Mainzer Landstr.69,  
60329 Frankfurt am Main.

**VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

**Technical Support and E-Warranty Certificate**

**[www.vevor.com/support](http://www.vevor.com/support)**