
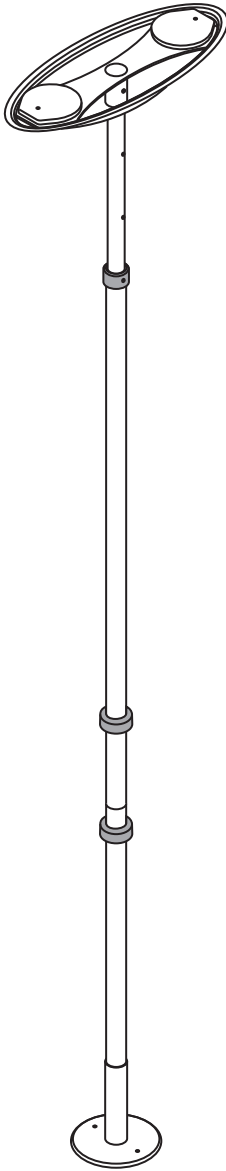
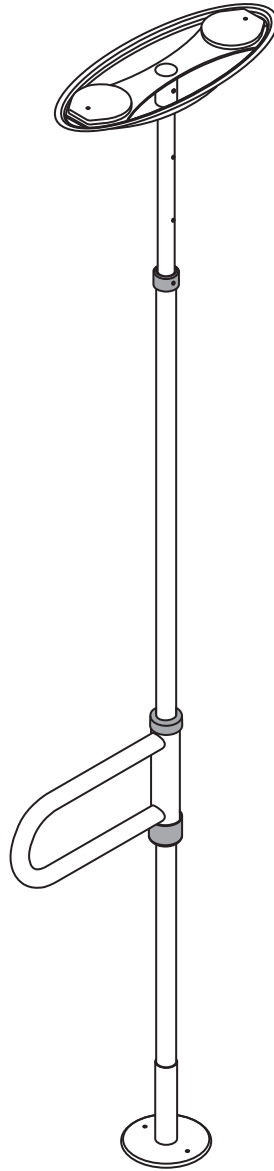
STEP-BY-STEP  
INSTALLATION  
VIDEOS

**uPole™  
only**  
UPOLE-P-CW

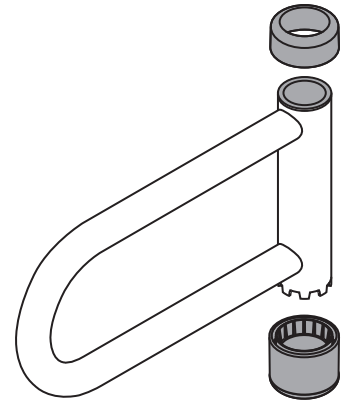**uPole™ +  
Pivot Bar**  
UPOLE-PB-CW

**Pivot Bar  
Add On Kit**  
UPOLE-BKIT-CW

## ⚠️ WARNING FOR BED SAFETY

### ⚠️ WARNING

#### ENTRAPMENT HAZARD

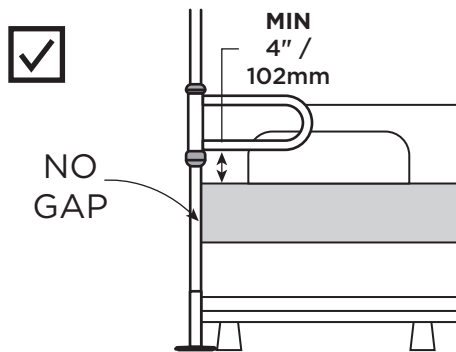
There is a risk of entrapment associated with all bedside mobility aids. Entrapment occurs when an individual becomes wedged between a bed rail and another object (example: mattress). Entrapment can result in serious injury and death. Read and understand the enclosed Entrapment Prevention Guidelines document. See the additional Entrapment Prevention Guidelines:

- [www.fda.gov/bedsafety](http://www.fda.gov/bedsafety)
- <https://www.cpsc.gov/safety-education/safety-guides/furniture-furnishings-and-decorations/adult-bed-rails>


PRIOR TO INSTALLATION, review all of the product specifications and instructions to ensure that it is safe to use with your bed and mattress. **Stop using immediately if product or bed shifts from its original position, if product is damaged, broken or missing parts.**

PRIOR TO EVERY USE, ensure the pole is installed correctly in one of the specified locations:

(a) No gap between the pole and the side of the mattress. Ensure a minimum distance 4"/102mm between the lower horizontal bar and mattress height.

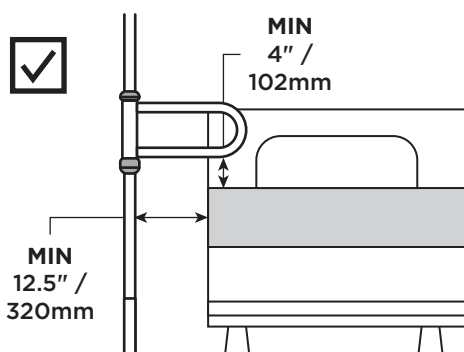(b) A gap of at least 12.5"/320mm between the pole and the side of the mattress. Ensure a minimum distance 4" / 102mm between the lower horizontal bar and mattress height.

### ⚠️ WARNING

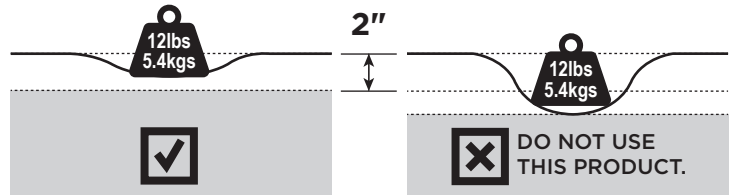
#### ENTRAPMENT HAZARD

NEVER use product without properly securing beside the bed. Incorrect installation can allow product to move away from mattress, bed frame and/or head or foot boards, which can lead to entrapment and death.

### ⚠️ WARNING

#### ENTRAPMENT HAZARD

(a) There is an increased risk of entrapment when using a safety product beside a bed with a soft mattress. Test mattress by applying 12lbs (5.4kg) of weight on the edge of the mattress. If the mattress can compress by 2" (51cm), do not use this product.

(b) If product is used in a care facility, follow the installations policies of the institutions. If this product does not comply with those policies, do not install this product.

(c) When changing bed linens, always confirm that product is installed correctly as per installation instructions. Re-position to eliminate a gap between product and side of mattress.

For questions about product safety or installation, or in case of a serious event with this product, contact a the HealthCraft at 1-888-619-9992 immediately. In addition to contacting the manufacturer directly, consumers should report problems to the CPSC at this website: <https://www.saferproducts.gov/> or call 1-800-638-2772, or to the FDA at 1-800-332-1088.

### ⚠️ WARNING

#### SUFFOCATION, STRANGULATION, ENTRAPMENT HAZARD

If product is installed incorrectly or moves from its initial position gaps can occur which can entrap and cause fatality. People with Alzheimer's disease, dementia or other neurological conditions, or those who are sedated, confused, or frail, are at increased risk of entrapment, suffocation and strangulation.

- NEVER use unless product is tight against mattress, without gaps, and at least 12-1/2 in. from headboard and footboard.
- NEVER use with children.
- NEVER use on toddler, bunk, water, electrical or inflatable beds, or on beds with mattress toppers or soft compressible pads.

This product is under the jurisdiction of the FDA; to report problems or injuries, call 1-800-332-1088.

### ⚠️ WARNING

#### ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS

Gaps in and around this product can entrap and kill. People with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, and are at increased risk of entrapment and strangulation. People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured beside the bed. If product can move away from bed or mattress, it can lead to entrapment and death.

**Complies with ASTM F3186-24** - Standard Specification for Adult Portable Bed Rails and Related Products.

# English

## OVERVIEW

Thank you for choosing the uPole System from HealthCraft. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your uPole System is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your uPole installed by a qualified contractor.

The uPole System is intended for moderate vertical load bearing to provide sitting and standing support for people with disability. The floor to ceiling range is 7-10 feet, and maximum user weight capacity 300lbs. Product is not intended to support full body weight. The uPole System is not to be used in any other way than described above.

The uPole is a high quality, commercial grade floor to ceiling safety pole. The uPole must be installed between a structural floor and ceiling by turning a jackscrew at the bottom of the pole. The pressure created by the jackscrew is strong enough for the uPole to support the user, yet it does not require drilling holes into the floor or ceiling. This is ideal for relocating or removing the pole. *NOTE: The top and base plates each have two holes that may be used to fix the plate directly to the ceiling joist(s) and the floor if necessary. Do not install the pole where it is directly exposed to water. See Page 10.*

The uPole Pivot Bar is a high quality, commercial grade pivoting and locking horizontal bar that connects to the uPole (floor to ceiling safety pole). The horizontal bar can be lifted, pivoted and lowered to the next locking position around the uPole. In any of the 8 lowered position (every 45 degrees) the Pivot Bar can be used to support the user.

## LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, rusting due to water exposure, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon.

Specifications subject to change without notice.

Manufacturer: HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. 1-888-619-9992. Patent Pending. © 2026 HealthCraft Products Inc.

## PARTS - UPOLE

FIGURE A.

- |                    |                                 |
|--------------------|---------------------------------|
| 1. Extension Pole  | 10. Extension Bushing           |
| 2. Top Pole        | 11. Top Bushing - uPole (x2)    |
| 3. Bottom Pole     | 12. Bottom Bushing - uPole (x2) |
| 4. Cover Sleeve    | 13. C-Clip                      |
| 5. Jam Nut         | 14. Cotter Pin (x2)             |
| 6. Extension Screw | 15. Clevis Pin (x3)             |
| 7. Plastic Washer  | 16. Wood Screw (x4)             |
| 8. Top Plate       | 17. Flat-Head Screw (x4)        |
| 9. Base Plate      | 18. Hex Key                     |

## PARTS - PIVOT BAR

- |                                |                        |
|--------------------------------|------------------------|
| 19. Pivot Bar                  | 22. Collar Cover       |
| 20. Top Bushing - Pivot Bar    | 23. Collar Bushing     |
| 21. Bottom Bushing - Pivot Bar | 24. Castellated Collar |

## INSTALLATION WARNINGS

FIGURE B.

1. Use caution while maneuvering pole during installation.
2. Standard pole can be installed anywhere with structural ceilings.
3. Standard pole can be installed directly under 1 ceiling joist.
4. Standard pole can be installed directly under 2 ceiling joists.
5. If subject to heavy usage, pole must be secured to ceiling joist(s) and floor using provided wood screws.
6. CAUTION - Damage may occur if extension screw is overtightened.
7. CAUTION - Do not install the pole with drop ceilings.
8. CAUTION - Do not install the pole on loose floor coverings.
9. CAUTION - Do not install the pole where it is directly exposed to water.
10. CAUTION - Do not install the pole with angled ceilings.

## SUGGESTED UPOLE PLACEMENT

FIGURE C.

1. BEDROOM - Locate pole at least 12.5"/320mm from headboard or footboard, adjacent to waistline and either (a) against the mattress - no gaps, or (b) 12.5"/320mm away from mattress.
2. BATHTUB - Locate pole base against tub, approximately half way along its length.
3. SEATING (TOILET or CHAIR) - To provide clearance when standing, locate pole 3-10" / 76mm-254mm forward of the knees, and 2-6" / 51mm-154mm to the side of knees when sitting.

## SUGGESTED PIVOT BAR POSITIONS

- |   |                                     |
|---|-------------------------------------|
| 4. BEDROOM                                  | 5. BATHROOM or 6. SITTING           |
| A = side support / out of the way           | A = side support / out of the way   |
| B = in-bed support                          | B = standing or transfer support    |
| C = sitting support                         | C = standing support                |
| D = standing support                        | D = standing or transfer support    |
| E = transfer support to walker / wheelchair | E = standing or transfer support    |
|   | F = transfer support in/out bathtub |

\*TOOLS REQUIRED: tape-measure, level, screwdriver, tape

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

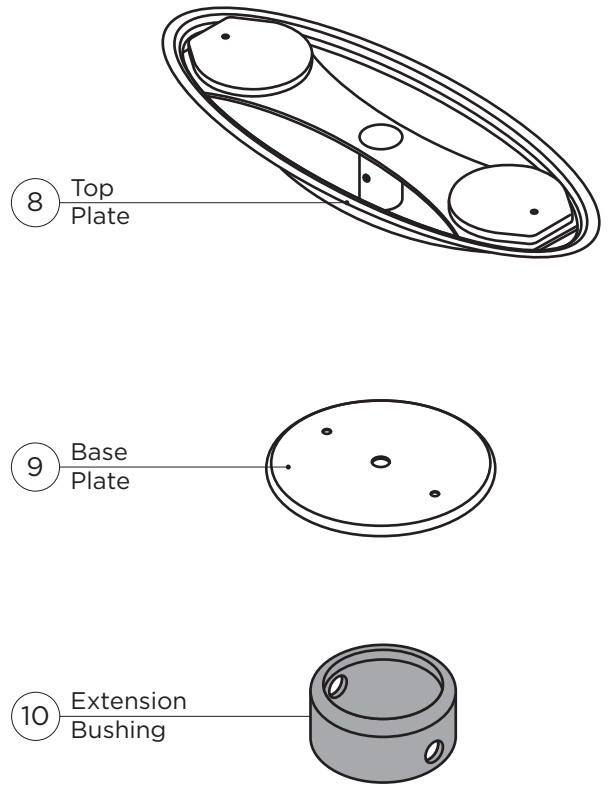
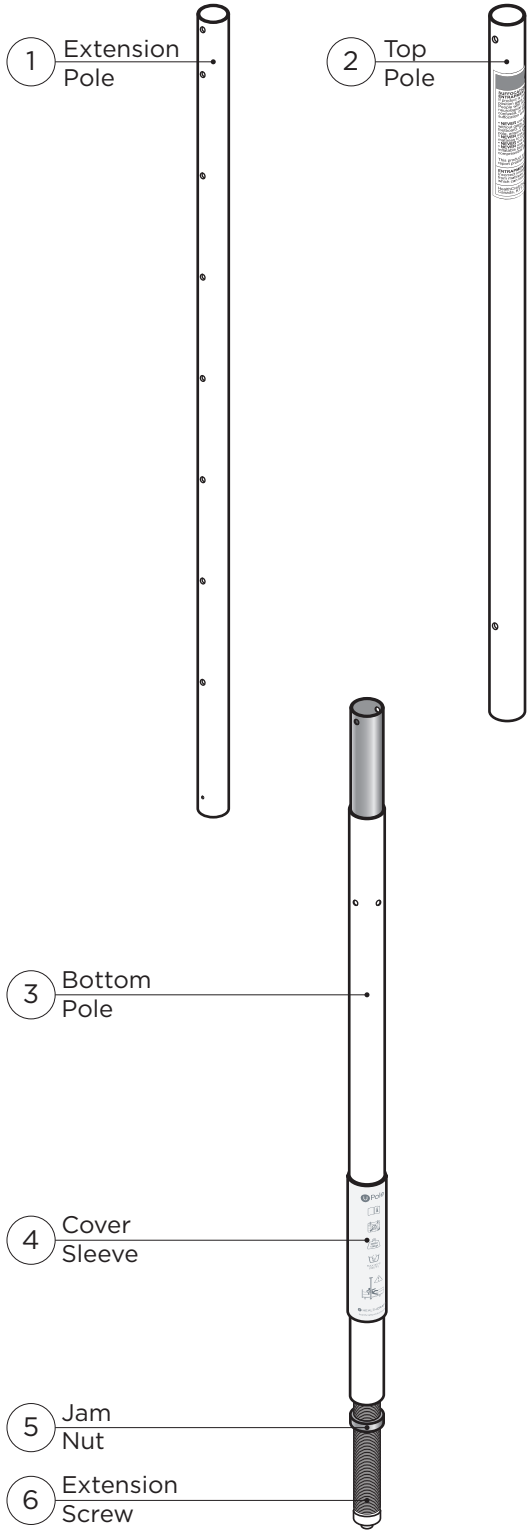
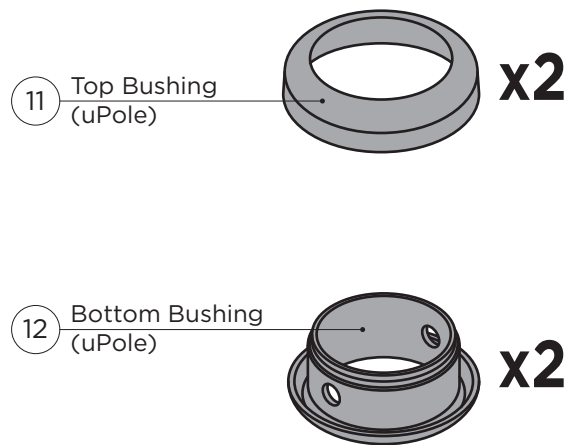
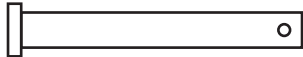
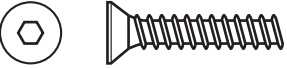
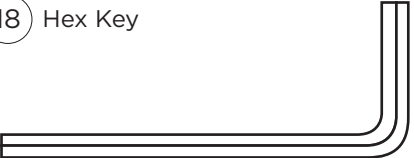
# A - PARTS
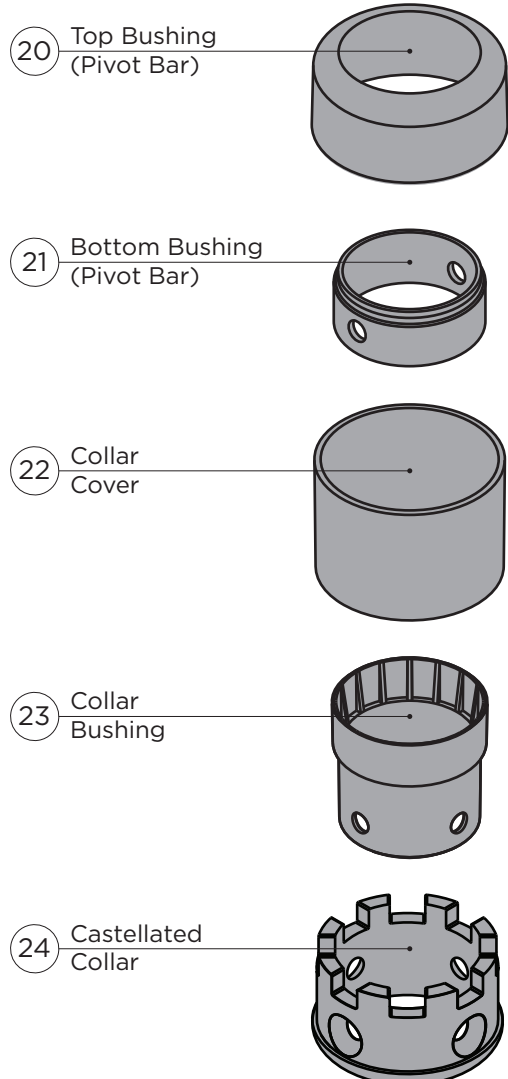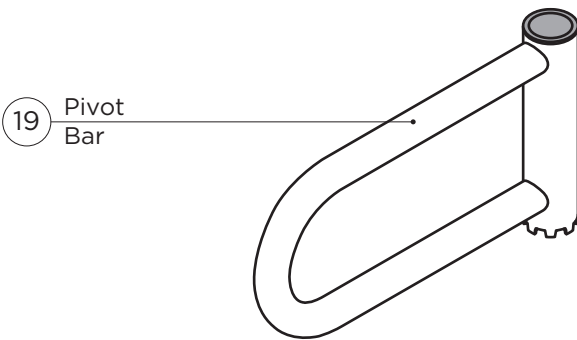
## Hardware

<p>7 Plastic Washer</p> 	<p>13 C-Clip</p> 	<p>14 Cotter Pin</p>  <p><b>x2</b></p>
<p>15 Clevis Pin</p>  <p><b>x3</b></p>	<p>16 Wood Screw *Optional</p>  <p><b>x4</b></p>	
<p>17 Flat-Head Screw</p>  <p><b>x4</b></p>	<p>18 Hex Key</p> 	

## Pivot Bar


# B - INSTALLATION WARNINGS

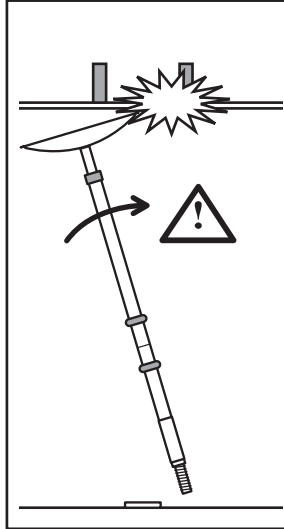
## 1


Use caution while maneuvering pole during installation.

## 2

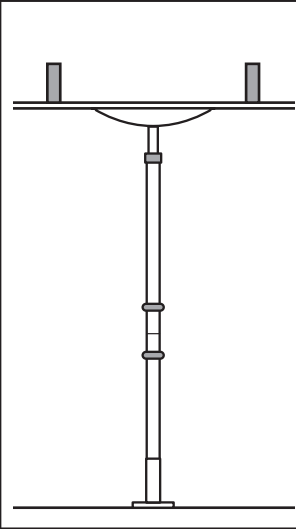
Standard pole can be installed anywhere with structural ceilings.

## 3

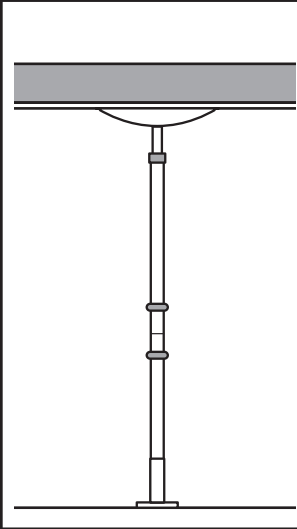
Standard pole can be installed directly under 1 ceiling joist.

## 4

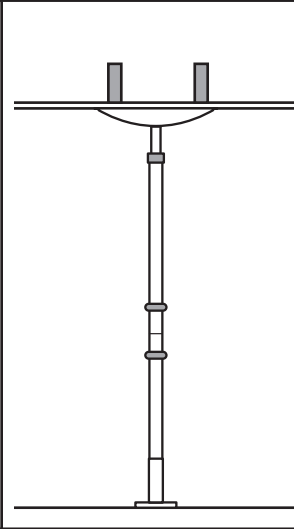
Standard pole can be installed directly under 2 ceiling joists.

## 5\*

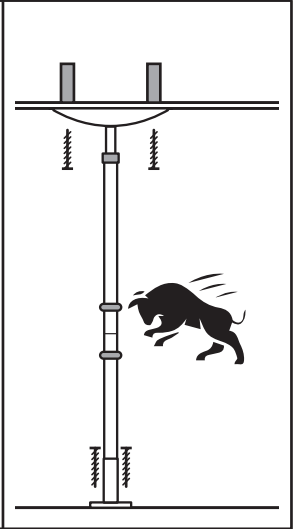
If subject to heavy usage pole must be secured to ceiling joist(s) and floor using provided wood screws (part #16).

## 6

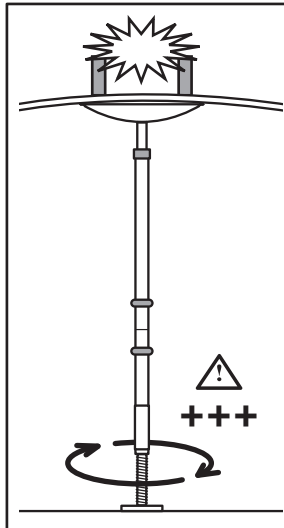
CAUTION - Damage may occur if extension screw is overtightened.

## 7

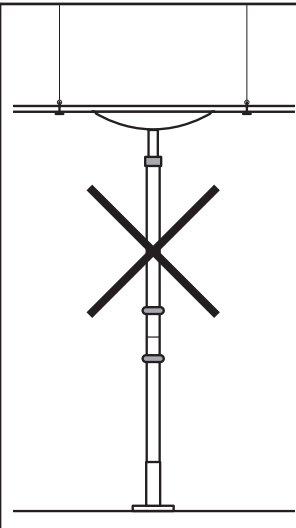
CAUTION - Do not install the pole with drop ceilings.

## 8

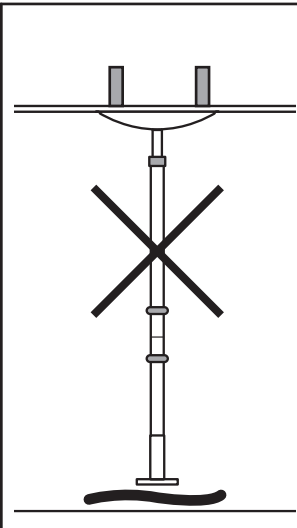
CAUTION - Do not install the pole on loose floor coverings.

## 9

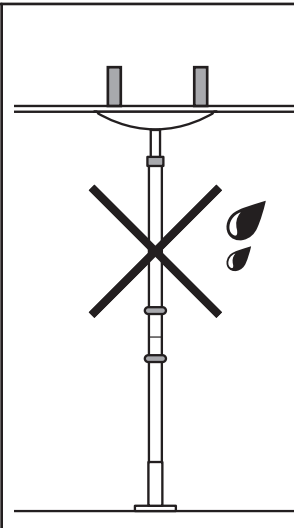CAUTION - Do not install the pole where it is directly exposed to water.


## 10

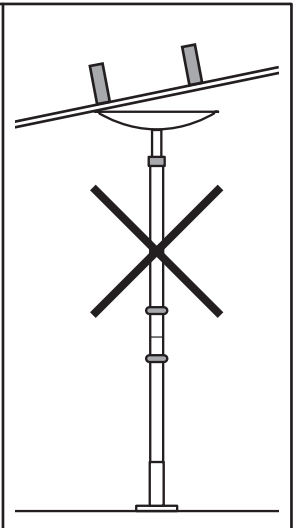
CAUTION - Do not install the pole with angled ceilings.

# C - SUGGESTED PLACEMENT AND POSITIONS

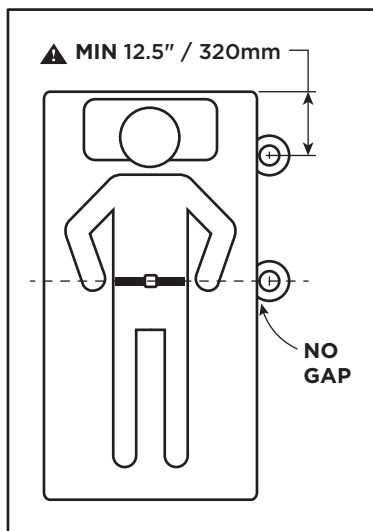
## 1a


## 1b
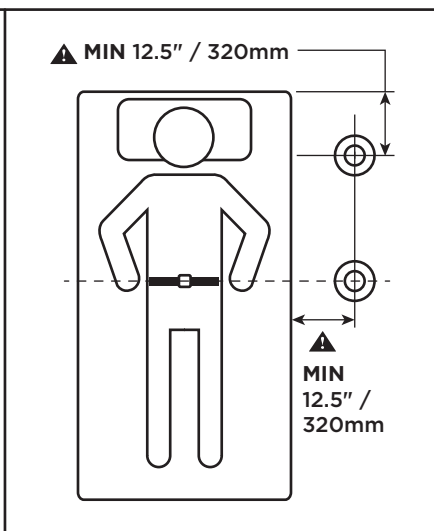
## 2
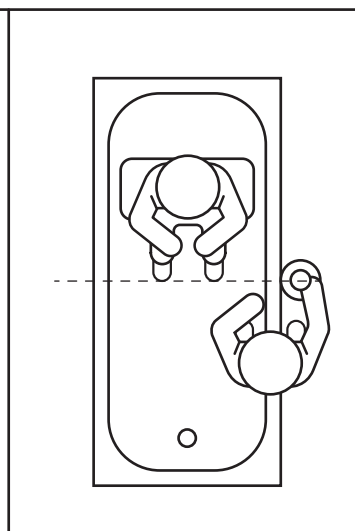
## 3


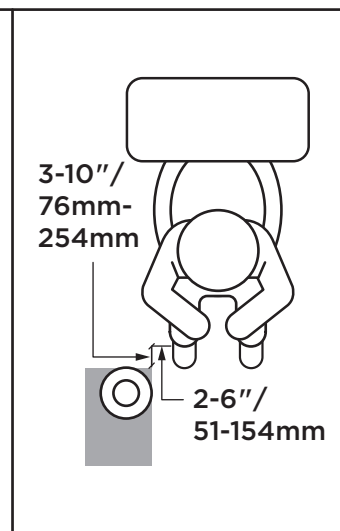
**BEDROOM** - Locate pole at least 12.5"/320mm from headboard or footboard, adjacent to waistline, and either (a) against the mattress - no gaps, or (b) at least 12.5"/320mm away from mattress.


**BATHTUB** - Locate pole base against tub, approximately half way along its length.

**SEATING (TOILET or CHAIR)** - To provide clearance when standing, locate pole 3-10" / 76mm-254mm forward of the knees, and 2-6" / 51mm-154mm to the side of knees when sitting.

## 4
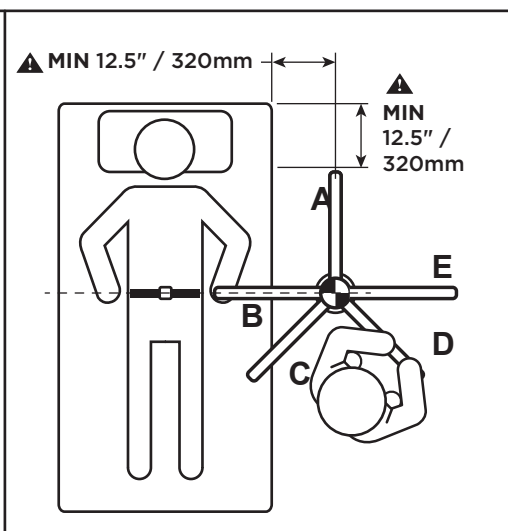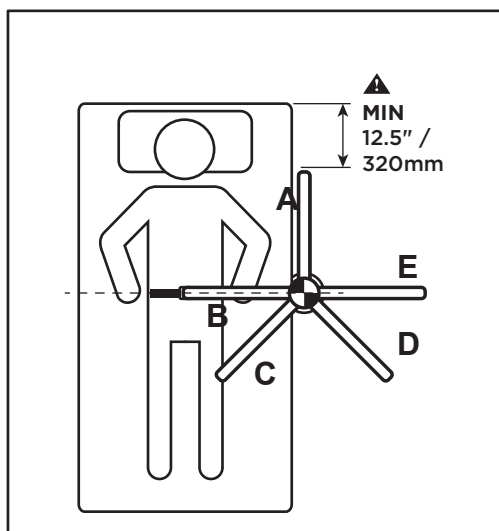
## 5/6

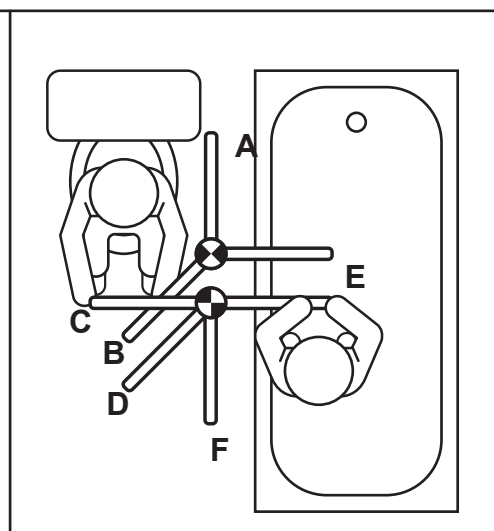
**BEDROOM**

- A = side support / out of the way
- B = in-bed support
- C = sitting support
- D = standing support
- E = transfer support to walker / wheelchair

**BATHROOM or SITTING**

- A = side support / out of the way
- B = standing or transfer support
- C = standing support
- D = standing or transfer support
- E = standing or transfer support
- D = transfer support in/out bathtub

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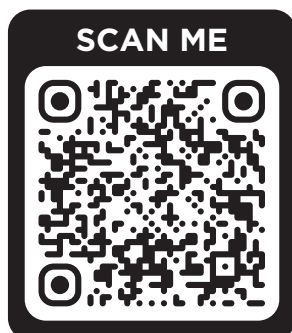
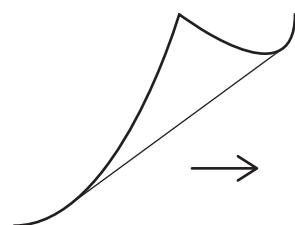
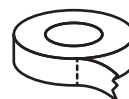
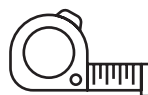
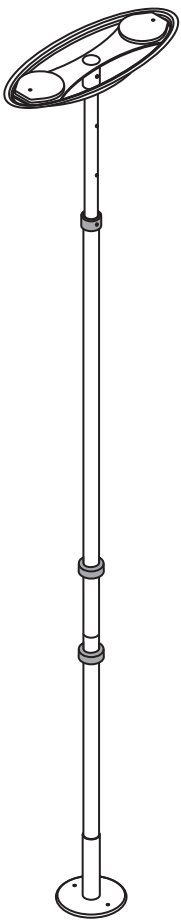
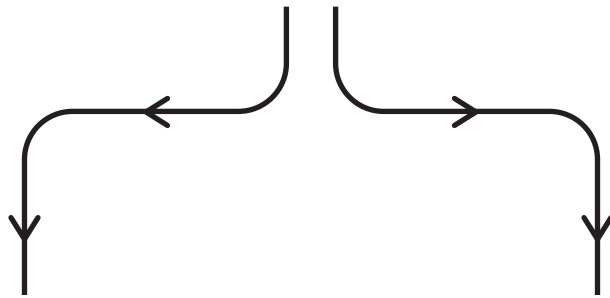
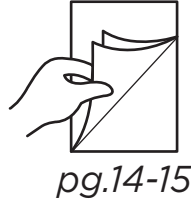
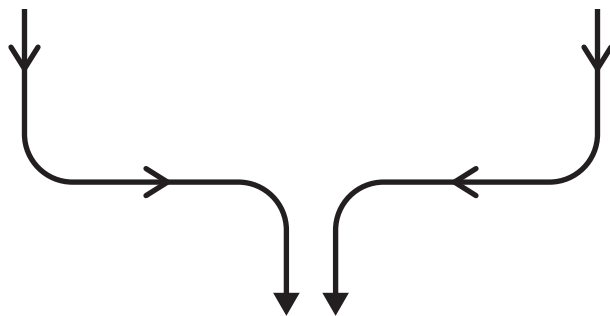
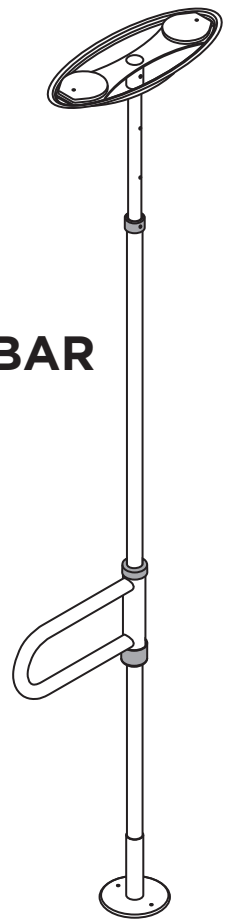
## FIG G. INSTALLATION

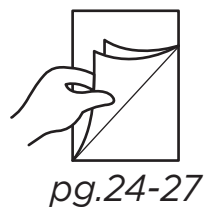
## D - POLE HEIGHT SETUP

**1** Determine height of ceiling and note corresponding hole on Extension Pole.

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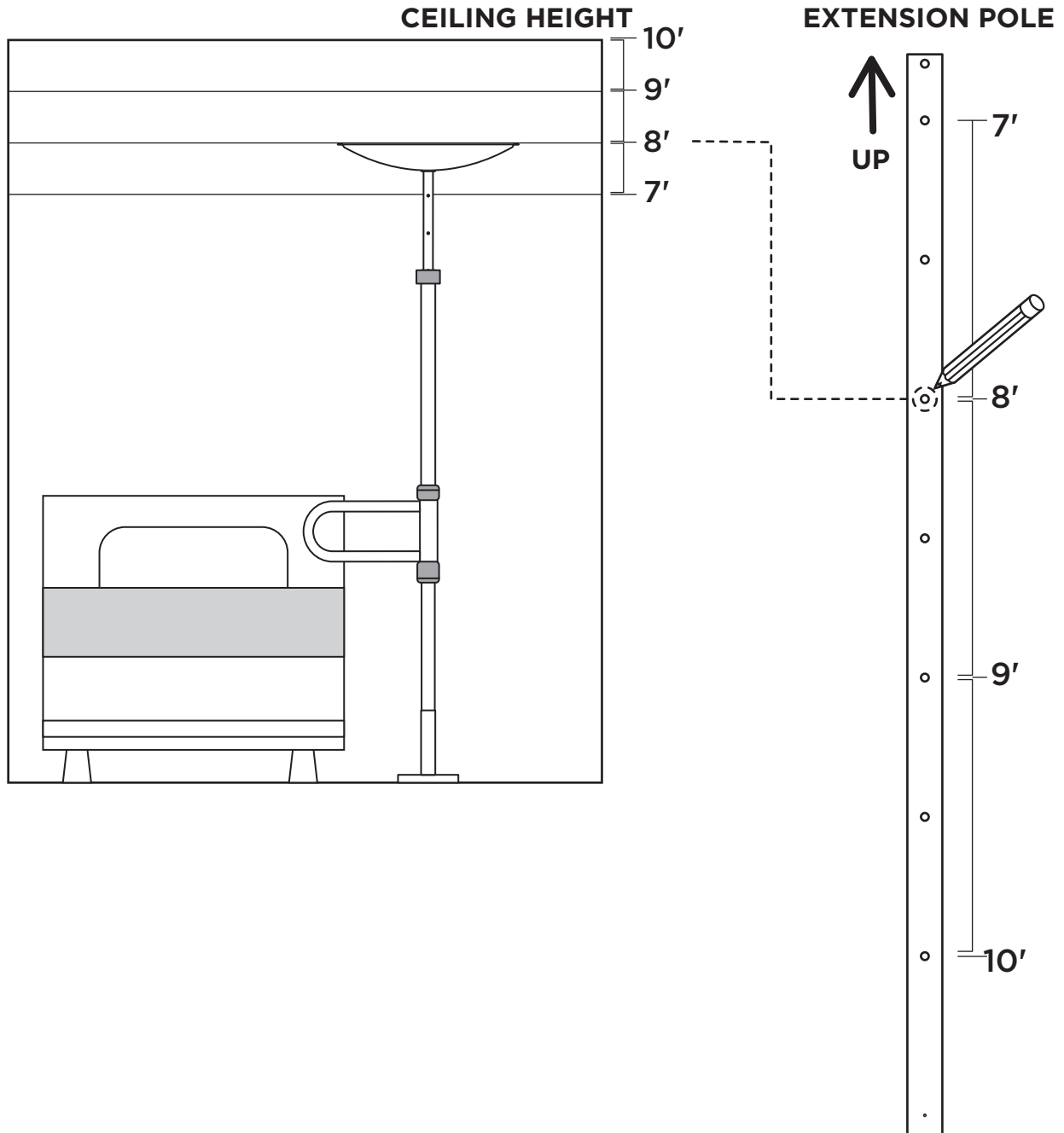
## D - POLE HEIGHT SETUP

### 2 Align Extension Bushing with through-hole on Top Pole.

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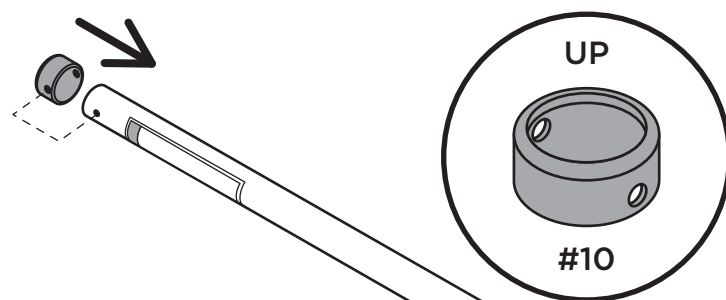
### 3 Slide Extension Pole into Top Pole; ensure correct UP orientation of Extension Pole.

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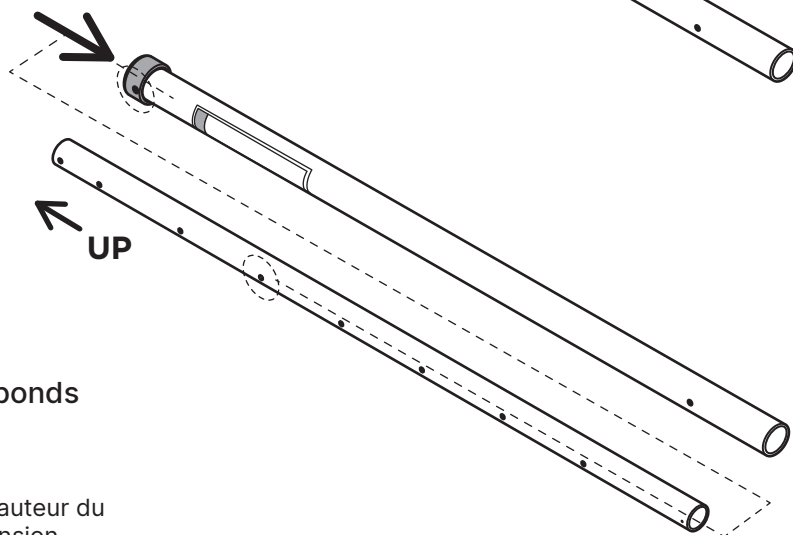### 4 Align hole on Extension Pole that corresponds to ceiling height with through-hole on Extension Bushing.

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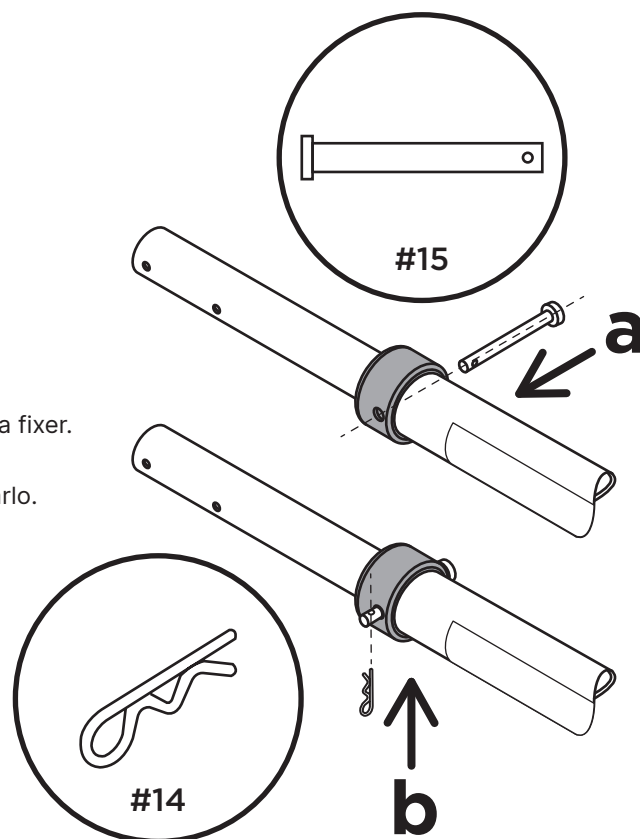
### 5 (a) Insert Clevis pin to secure bushing and (b) Insert Cotter pin into Clevis pin to secure.

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\*Continue to **pg.16** for uPole only assembly


\*Skip to **pg.20** for uPole + Pivot Bar assembly

## E - UPOLE ONLY ASSEMBLY

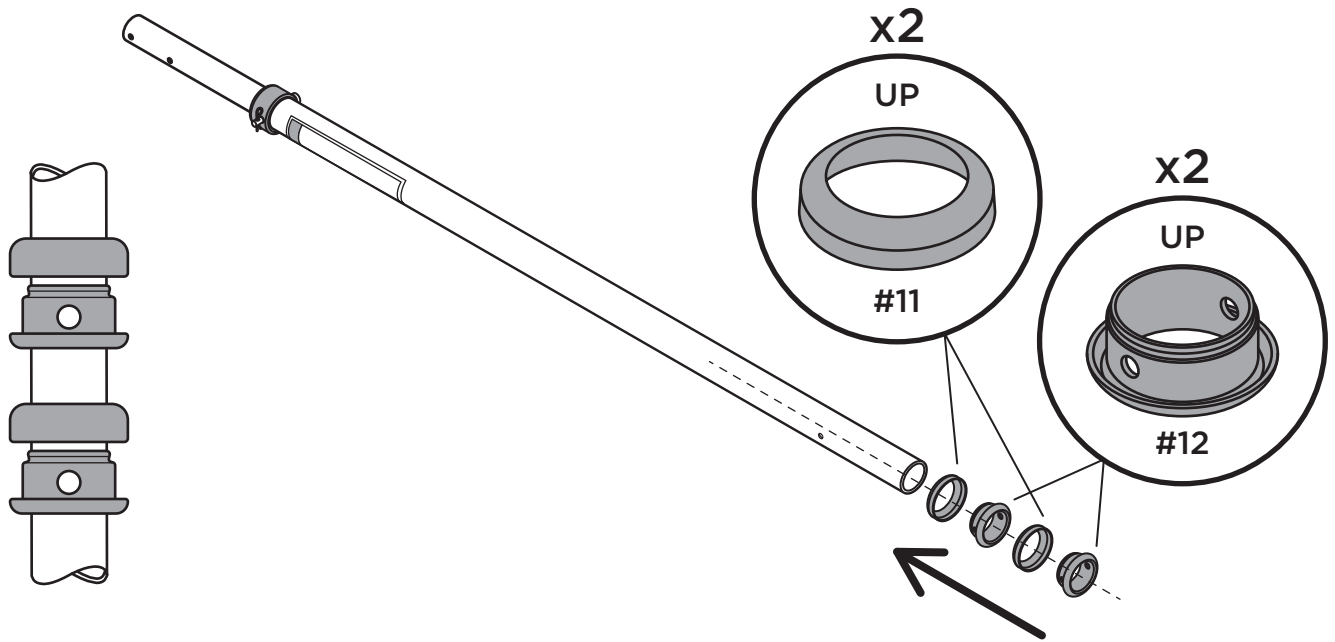- 1** Slide all Top and Bottom Bushings (for uPole only) onto Top Pole. Ensure correct order and UP orientation for all bushings.

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
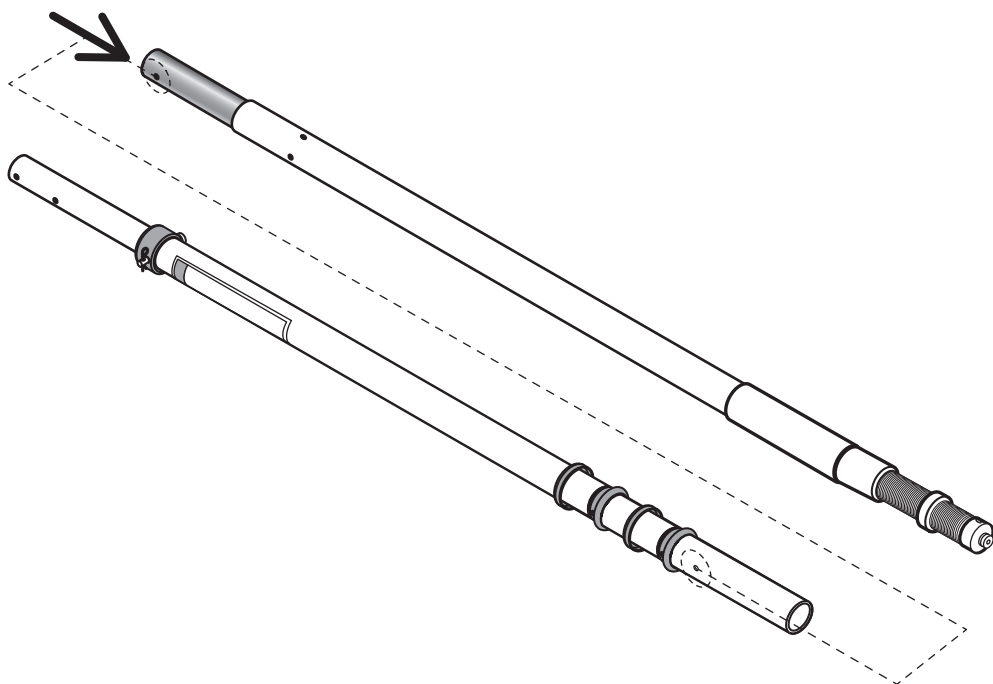
## E - UPOLE ONLY ASSEMBLY

### 3 (a) Align Bottom Bushing with through-hole on Top Pole (and Bottom Pole) and (b) Insert Clevis pin to connect Top and Bottom Poles together.

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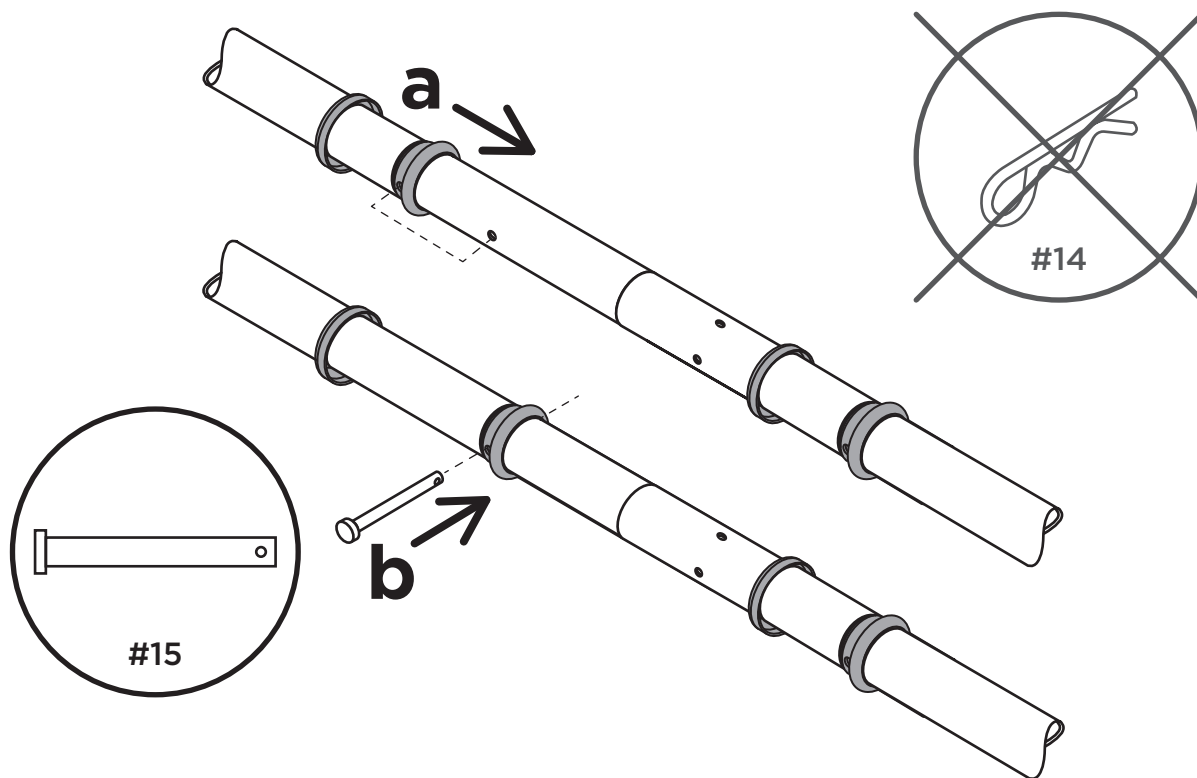
### 4 Slide Top Bushing down to conceal Clevis pin and SNAP to Bottom Bushing.

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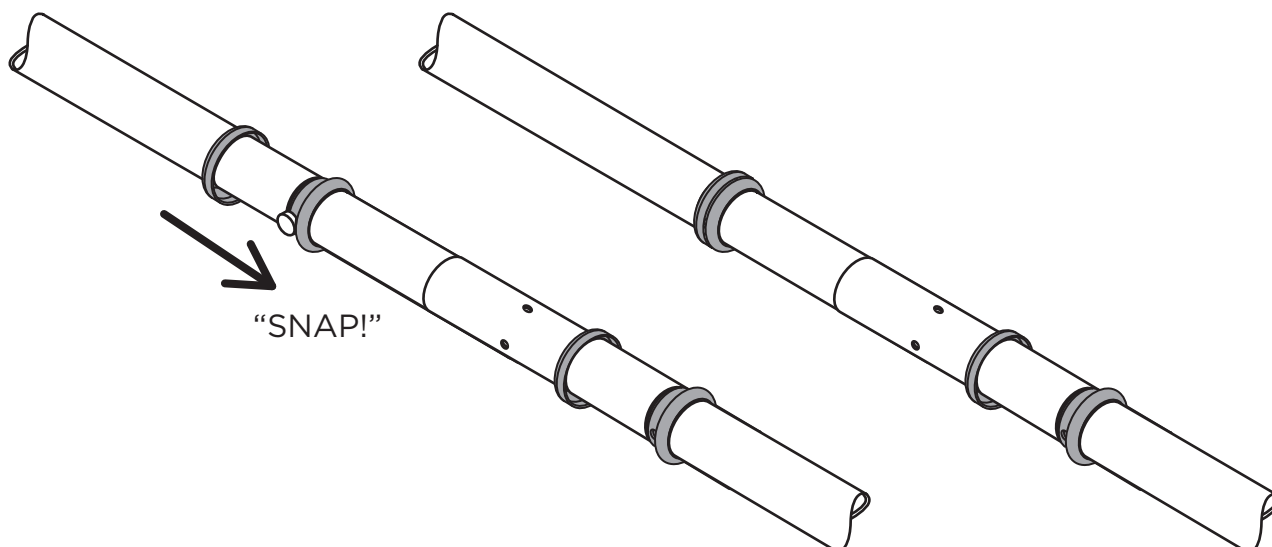
## 5 (a) Align Bottom Bushing with threaded holes on Bottom Pole and (b) Install Flat-head screws (x2) with Hex key to secure Bottom Bushing to Bottom Pole.

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
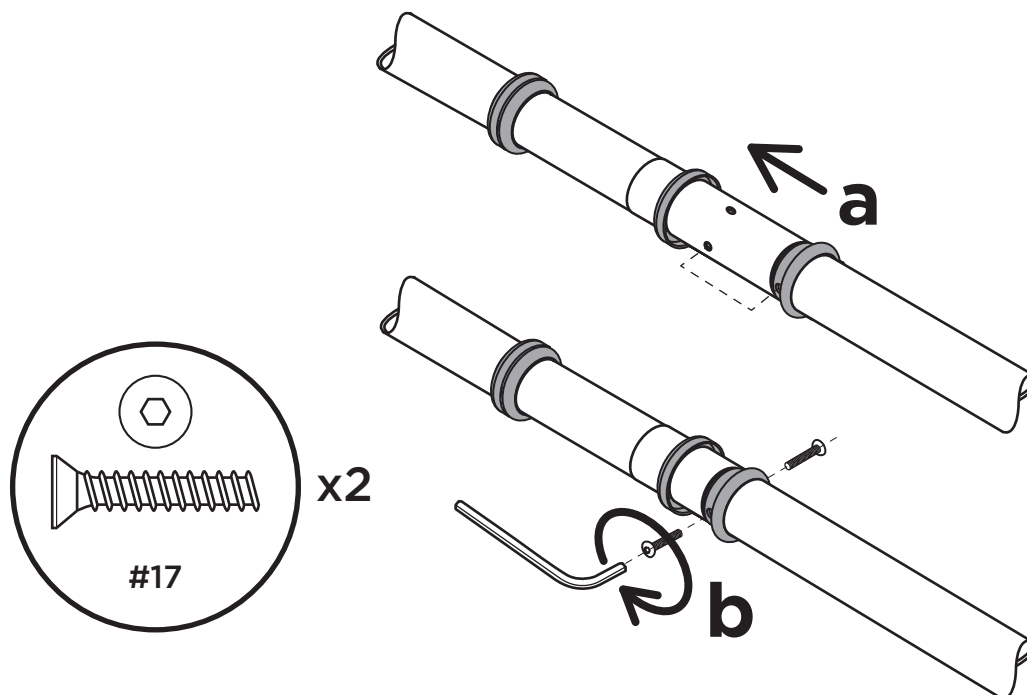
## 6 Slide Top Bushing down to conceal screws and SNAP to Bottom Bushing.

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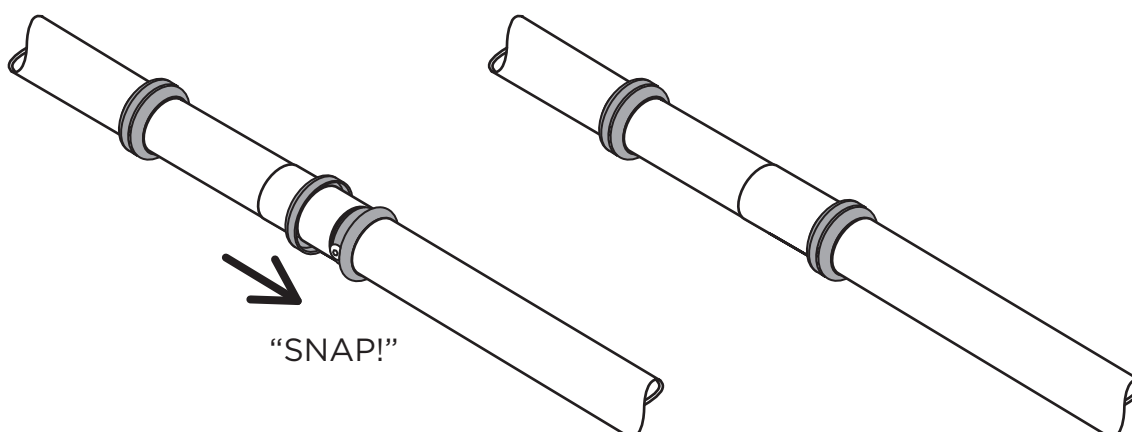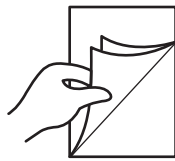
*pg.24-27*

## F - UPOLE + PIVOT BAR ASSEMBLY
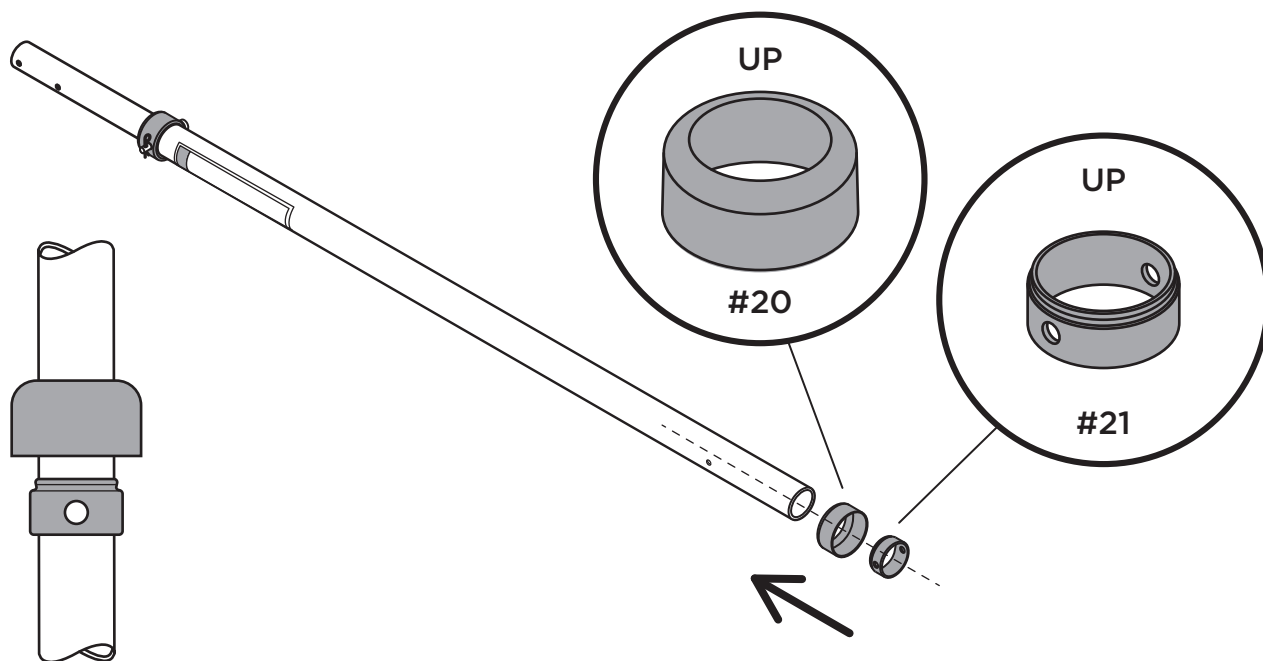
### 1 Slide Top Bushing and Bottom Bushing (for Pivot Bar) onto Top Pole. Ensure correct UP orientation.

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
### 2 (a) Fit Collar Bushing inside Castellated Collar and (b) Slide both onto Bottom Pole and align all holes.

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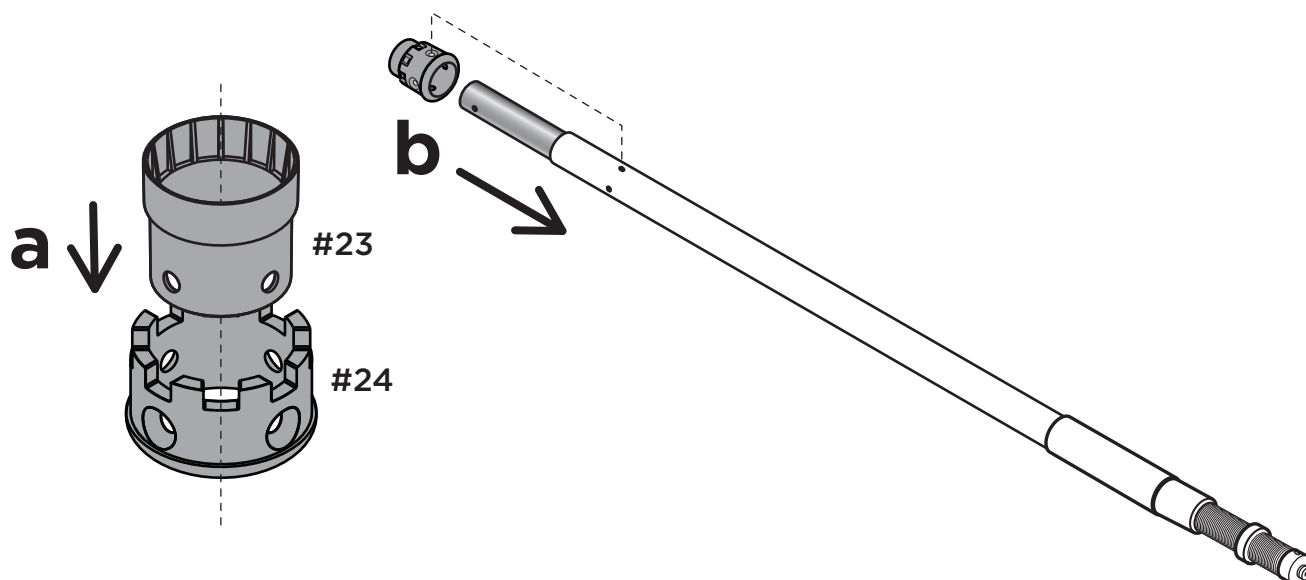
## F - UPOLE + PIVOT BAR ASSEMBLY

- 3** (a) Insert 4 flat-head screws into threaded holes, and once all screws are engaged,  
(b) Fully tighten screws with Hex key to secure the Castellated Collar.

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
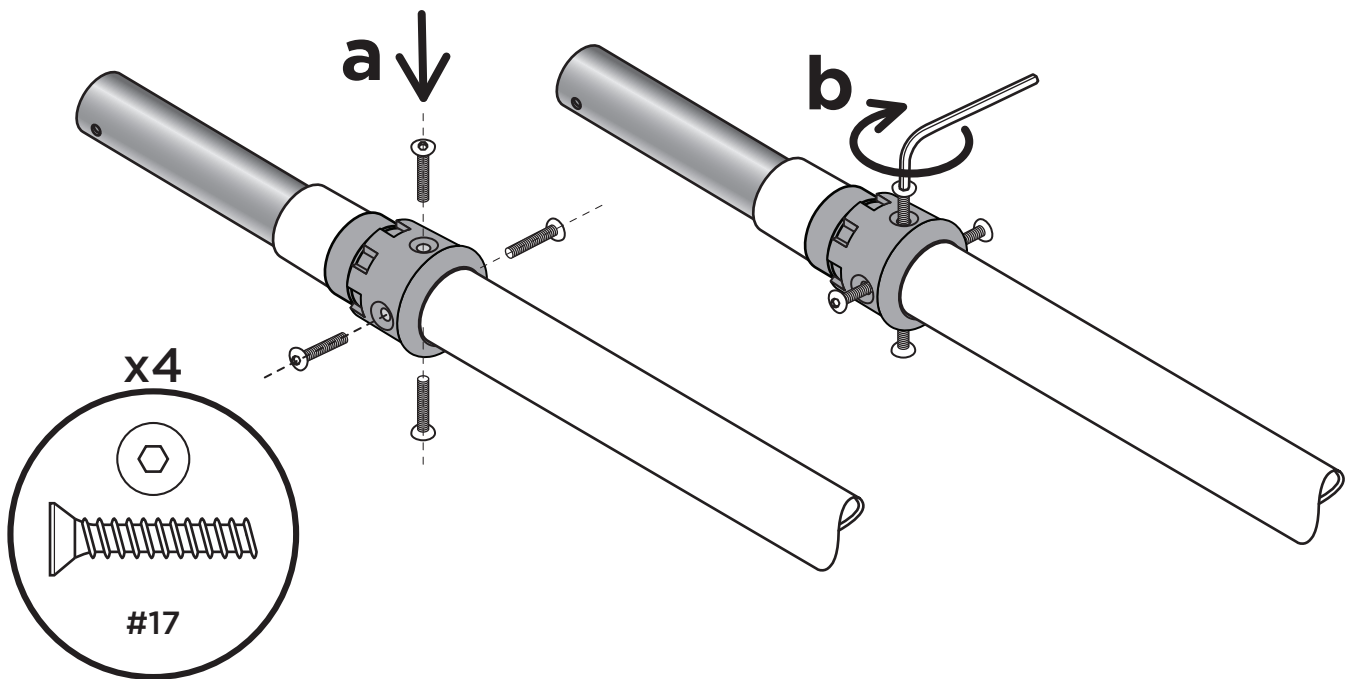
- 4** Slide Collar Cover onto Castellated Collar to conceal screws.

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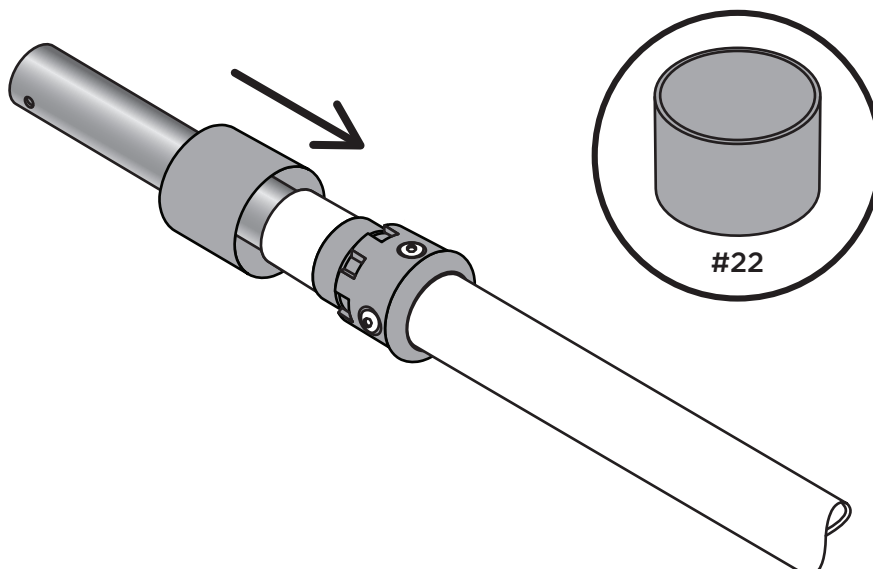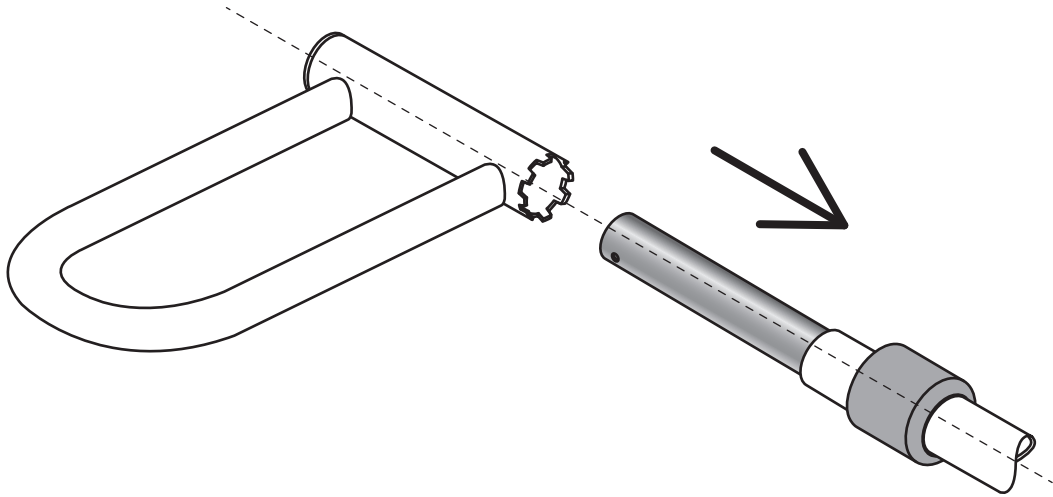
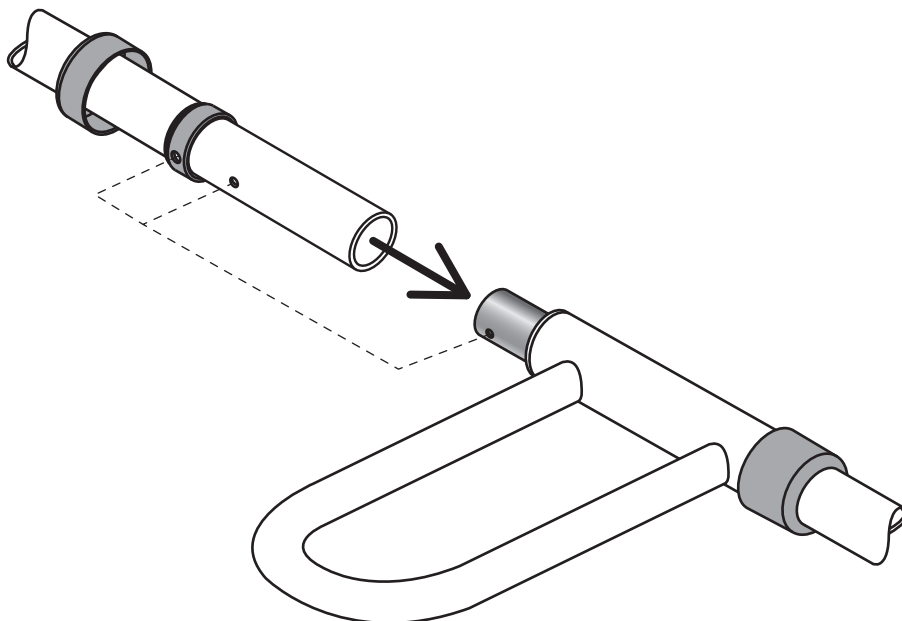
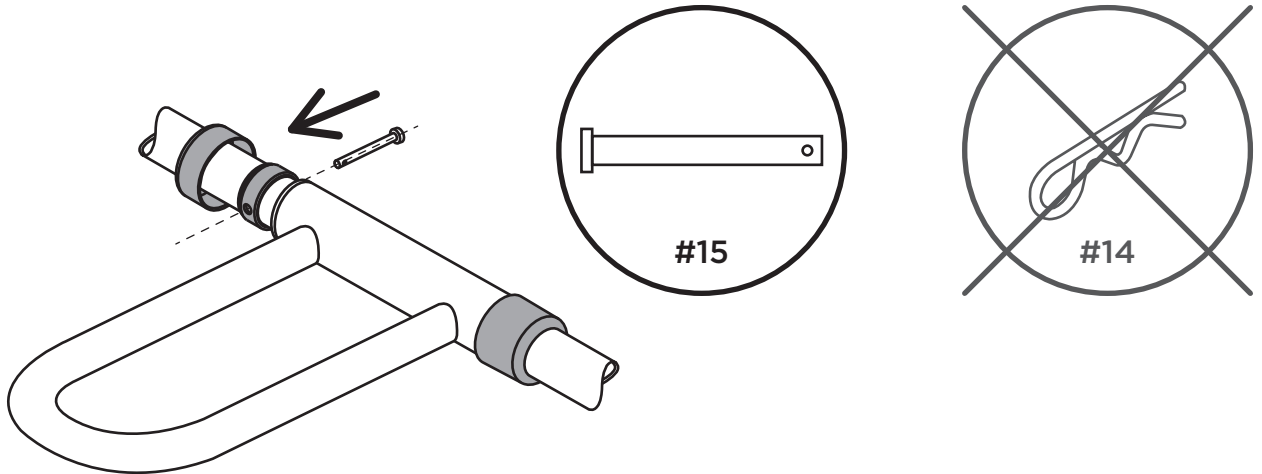
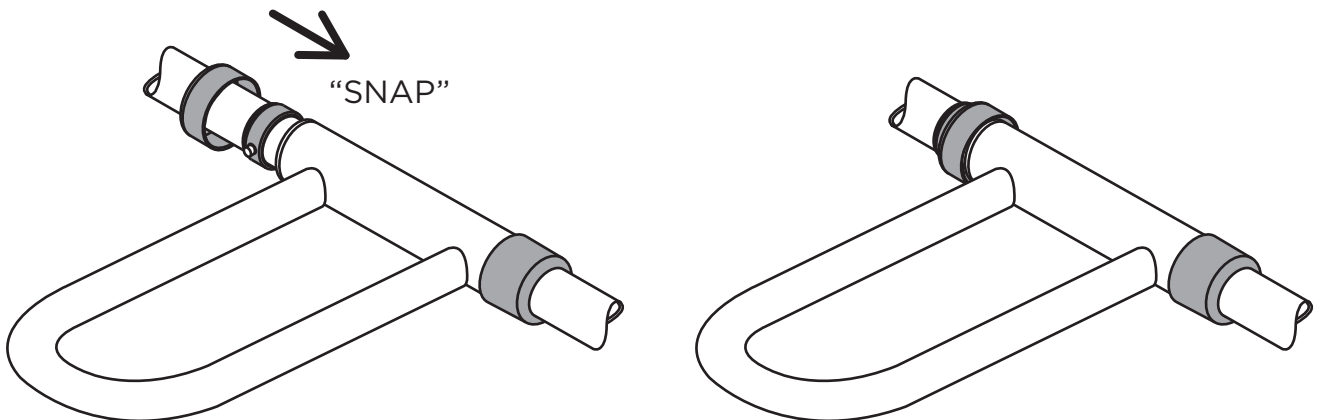
\*Continue to pg.24 to finish installation

## G - INSTALLATION
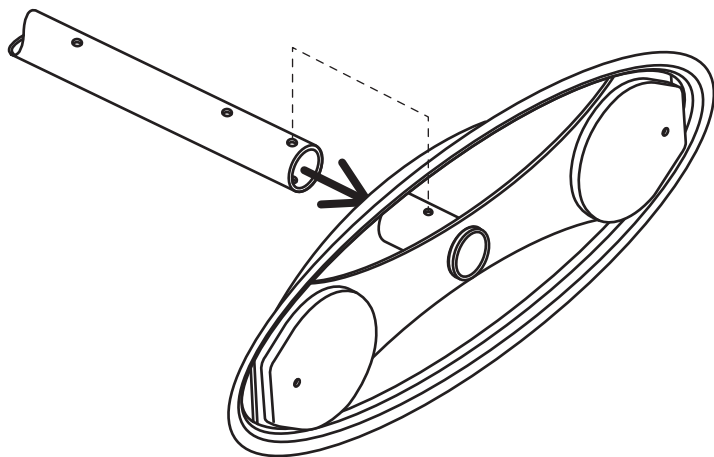
### 1 Place Top Plate onto Extension Pole and Align holes.

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
### 2 Insert clevis pin to connect the Top Plate to the Pole.

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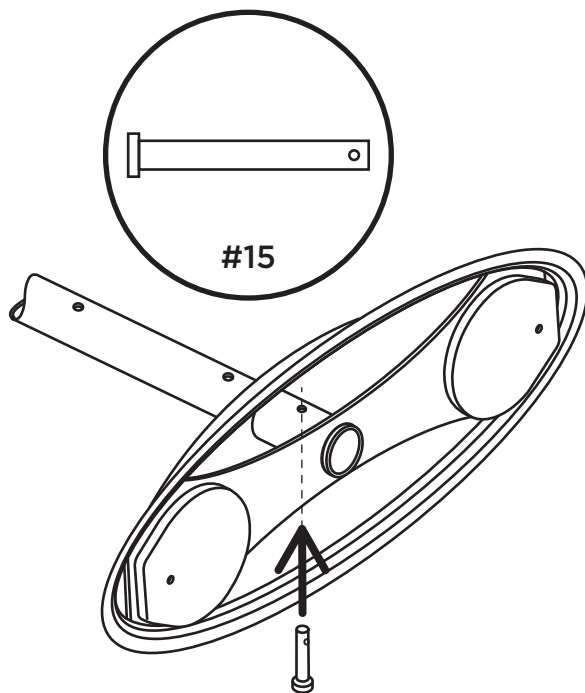
### 3 Insert cotter pin into clevis pin to secure.

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
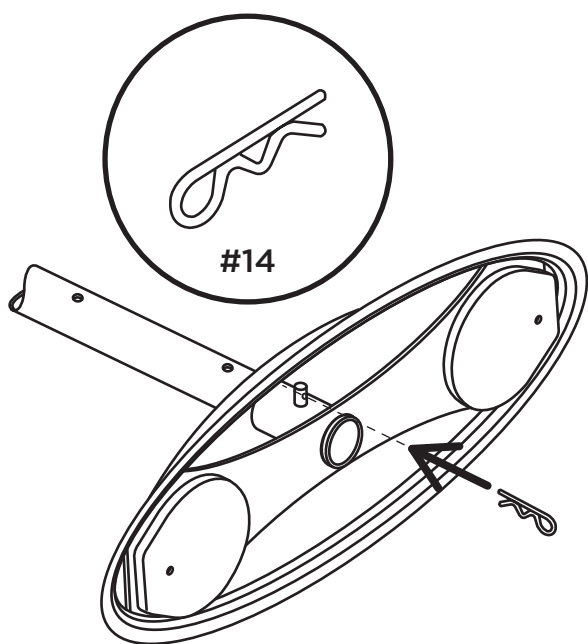
### 4 Place plastic washer onto stem of Extension Screw (on Bottom Pole).

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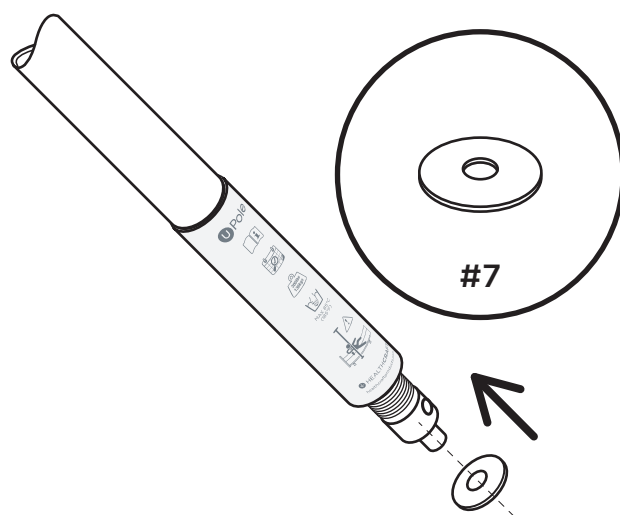## G - INSTALLATION

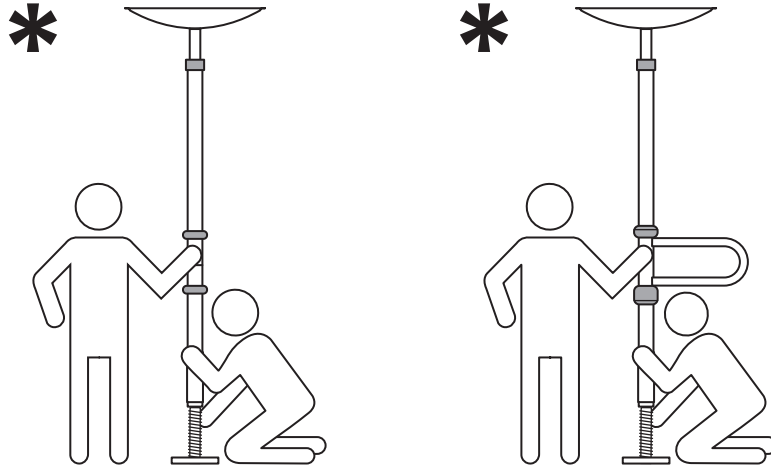**\*NOTE :** For steps 5–9, it is recommended to have a second person assist with installation.

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**\*NOTE :** See pg.10-11 Fig.B-C for installation warnings and suggestions

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
### 5 Place Base Plate on floor at desired location\*, tilt pole until upright, and insert into the Base Plate hole.

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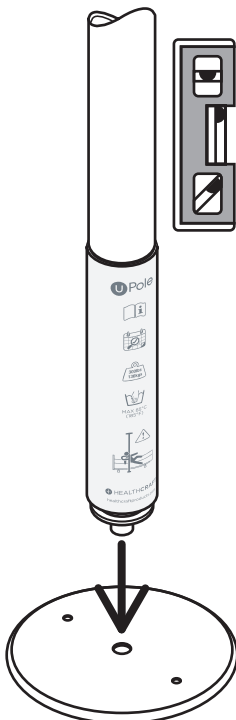
### 6 a) Lift Cover Sleeve and tape to pole, then b) Insert shaft of screwdriver through hole of Extension Screw.

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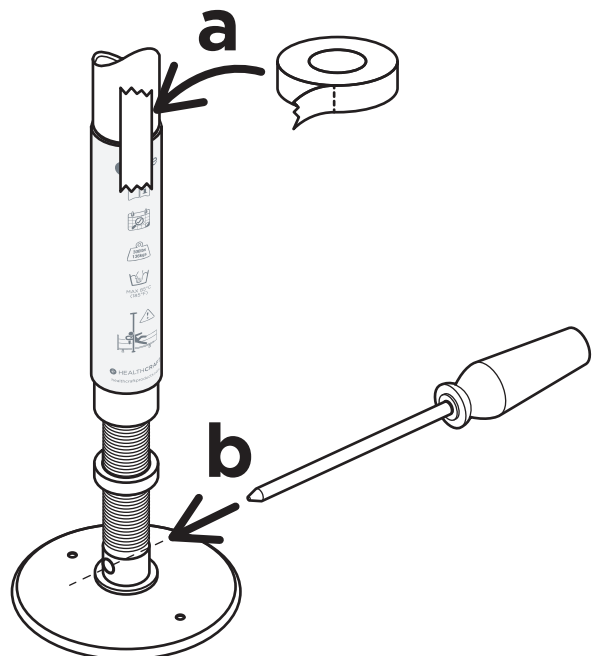
## G - INSTALLATION

### 7 Hold the pole upright and turn screwdriver clockwise to raise pole.

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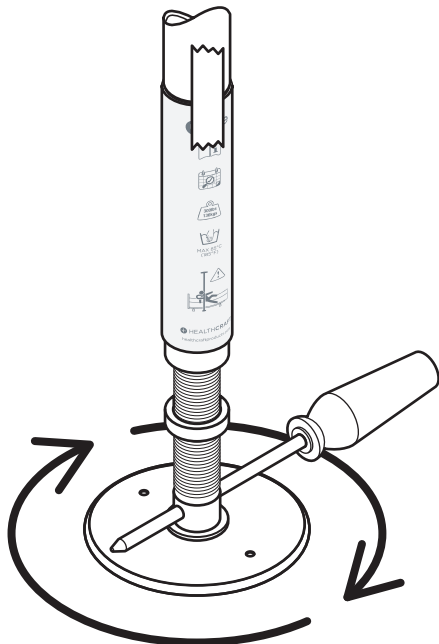
### 8 Ensure pole and Top Plate are level as they contact the ceiling.

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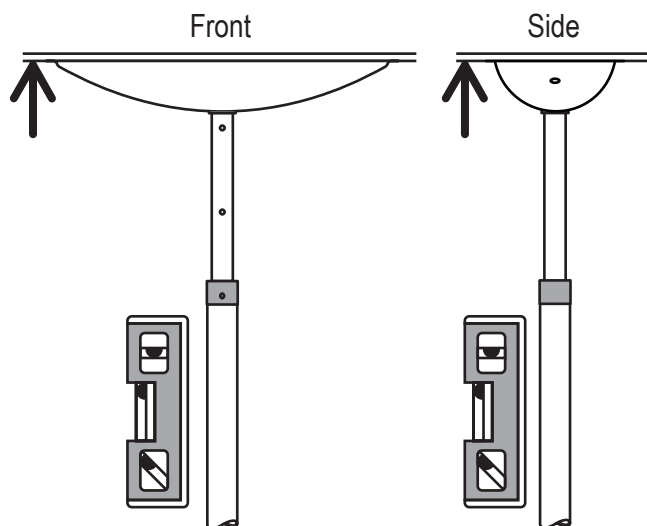
### 9 Continue turning screwdriver until pole is **COMPLETELY SECURE**.

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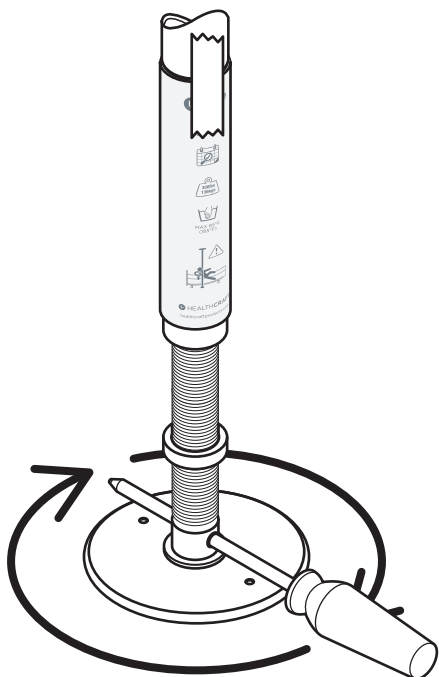
### 10 Turn Jam Nut counter-clockwise until it is tight against bottom of pole.

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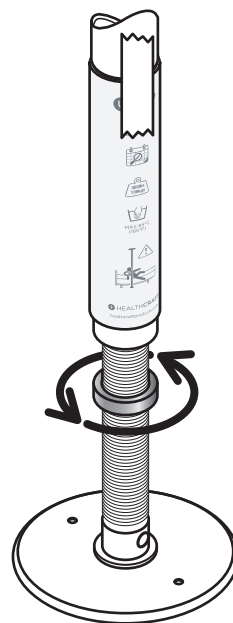
## G - INSTALLATION

### 11 Crank screwdriver another 1/4 turn counter-clockwise to lock Jam Nut.

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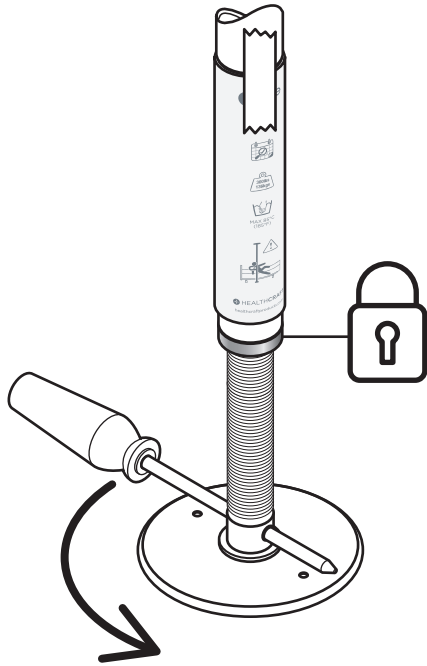
### 12 Cover exposed hole on the Extension Screw with C-Clip (SNAPS into hole).

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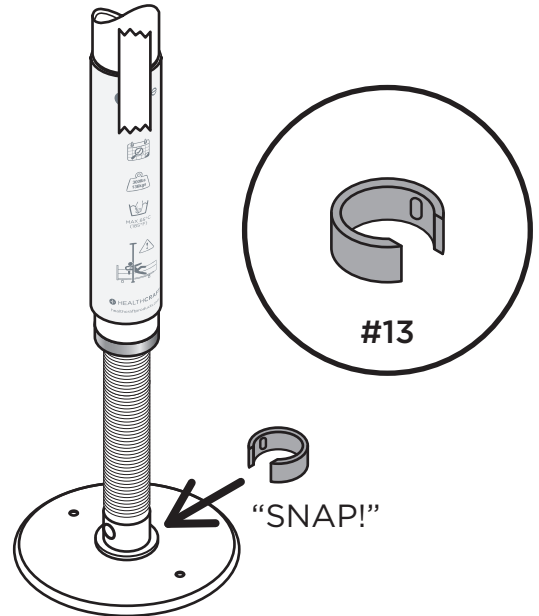
### 13 Lower Cover Sleeve to conceal Extension Screw.

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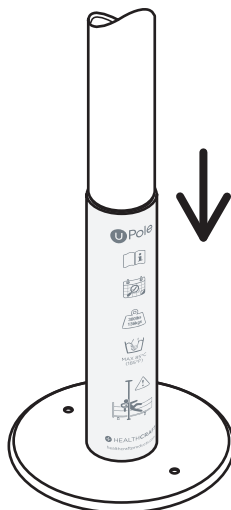
### 14 Inspect pole monthly - tighten Extension Screw if needed.

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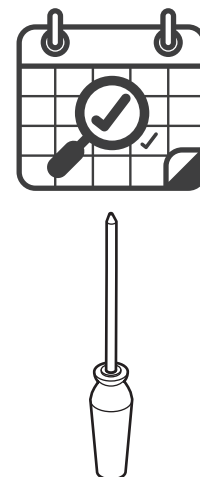# **INSTALLATION COMPLETE SAVE THESE INSTRUCTIONS**

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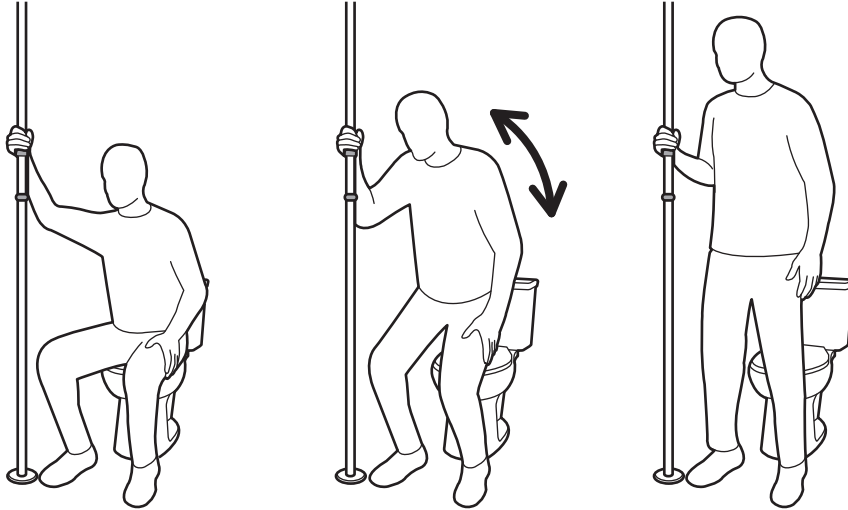
*\*If repositioning of uPole is required, follow Fig.G steps 13-5  
(Reverse order and actions), then 5-13 to re-install.*

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
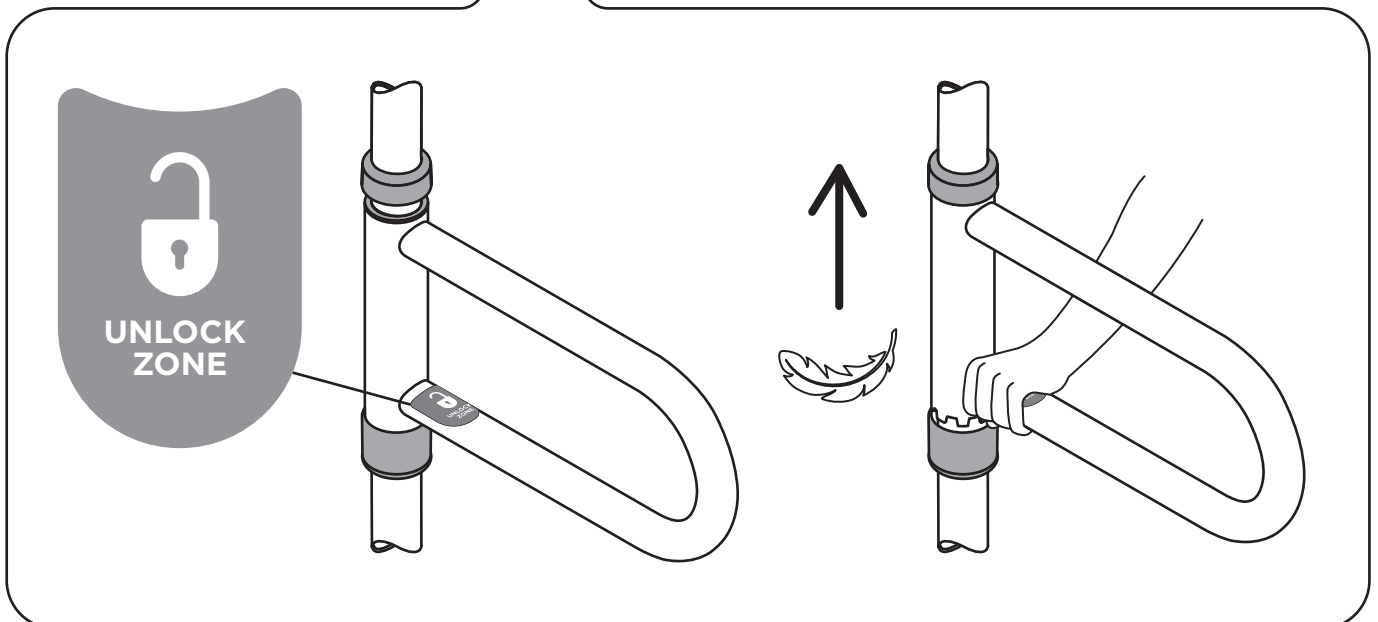
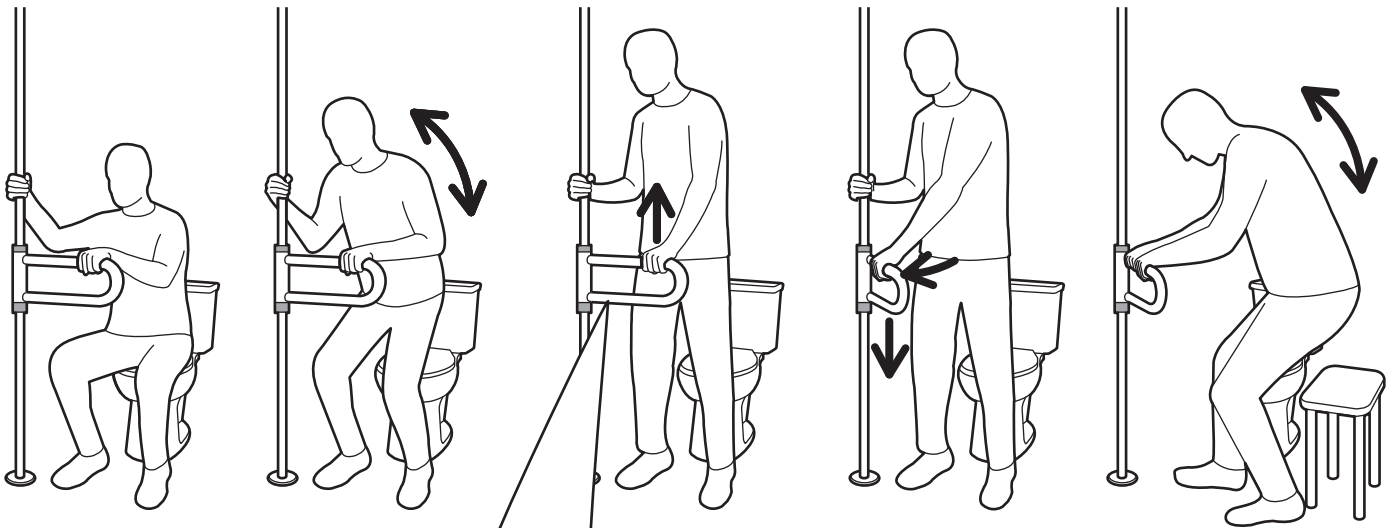
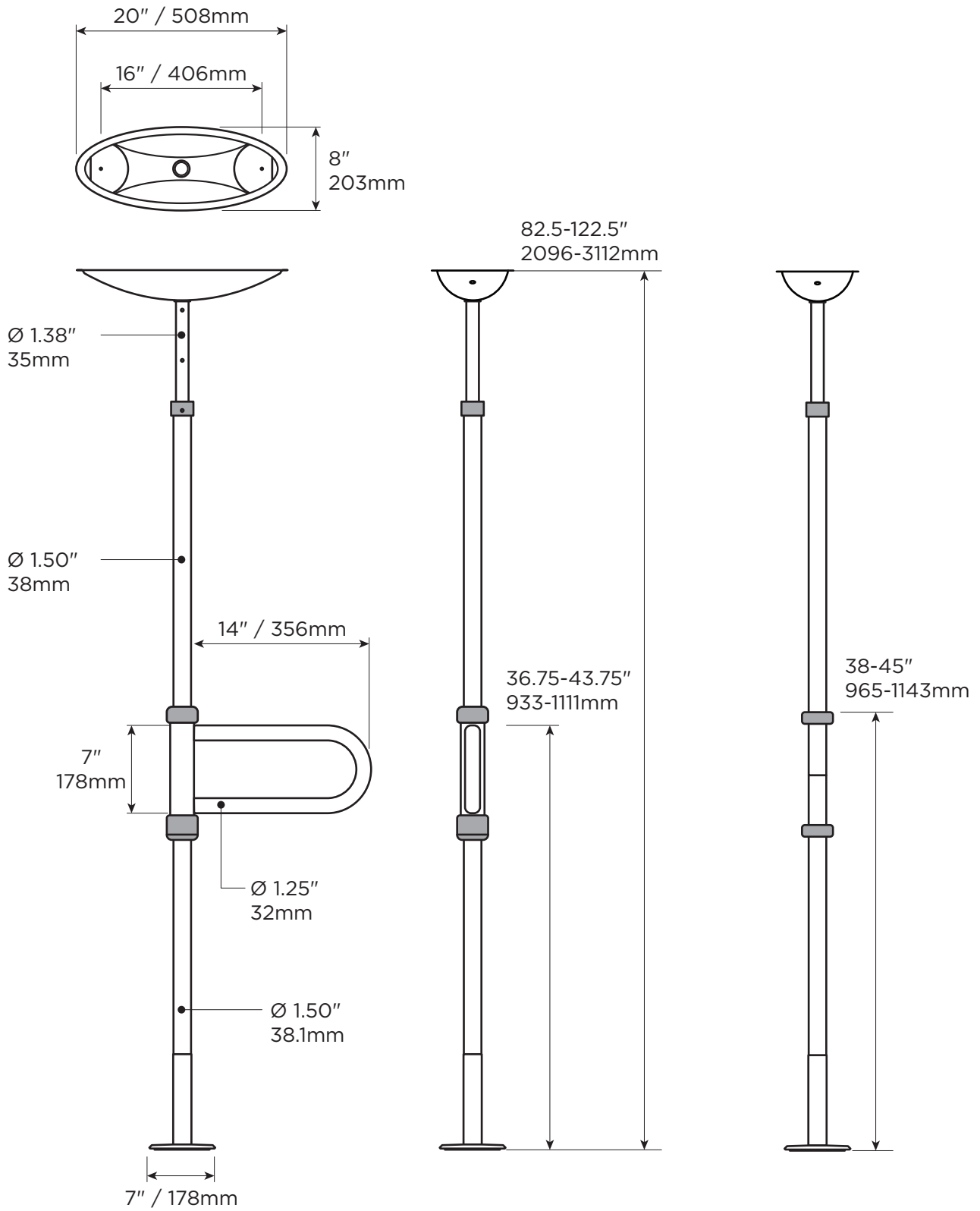
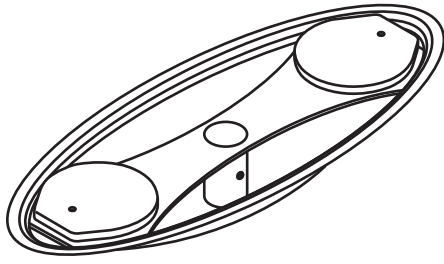
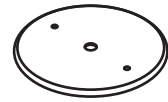
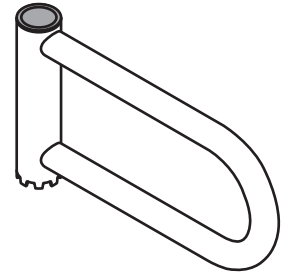
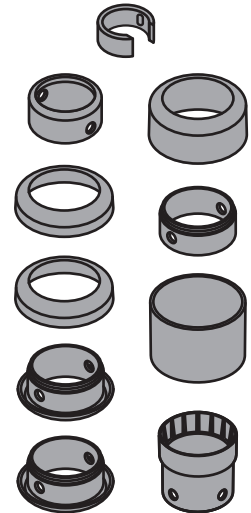
**PA**  
0.15lbs / 0.07kg

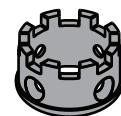**FE**  
0.30lbs / 0.14kg

[HEALTHCRAFTPRODUCTS.COM/UPOLE-INSTALLATION](https://HEALTHCRAFTPRODUCTS.COM/UPOLE-INSTALLATION)

STEP-BY-STEP  
INSTALLATION VIDEOS

**Need assistance? Call us!**  
**1 (888) 619-9992**

HealthCraft Products Inc.  
2790 Fenton Road  
Ottawa, Canada  
K1T 3T7