
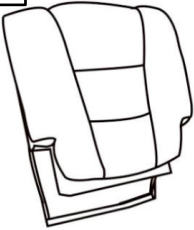
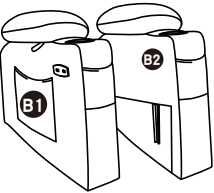


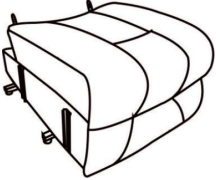


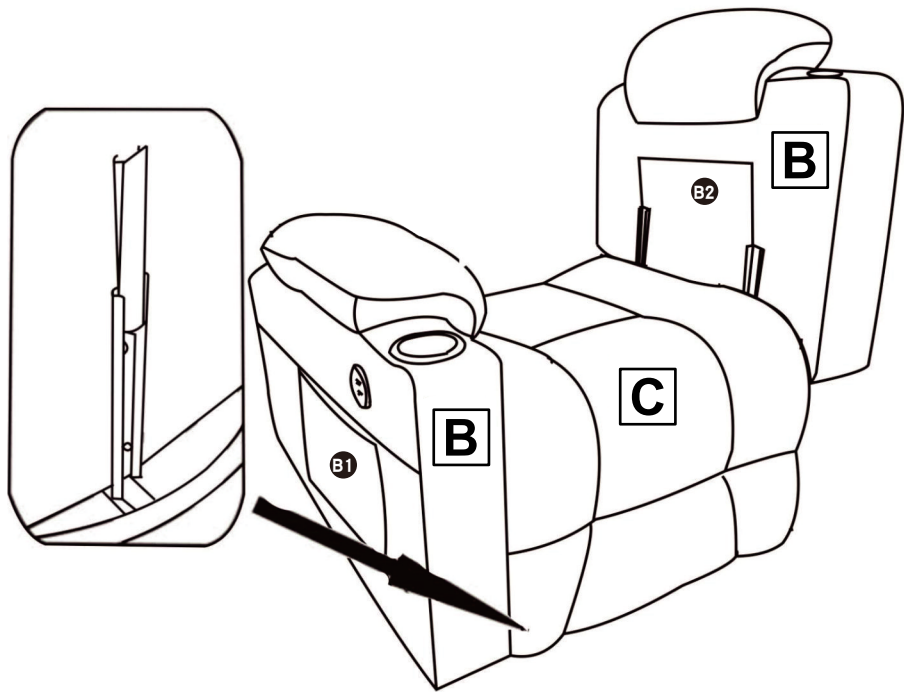



Lift Recliner Assembly Instructions

A  Backrest	B  Armrests x2	C  Base	D  Lift Remote	E  Power Cord	F  Parts x2	G  Parts x4	H  Massage Remote
---	--	---	---	---	---	---	---

STEP 1




STEP 2

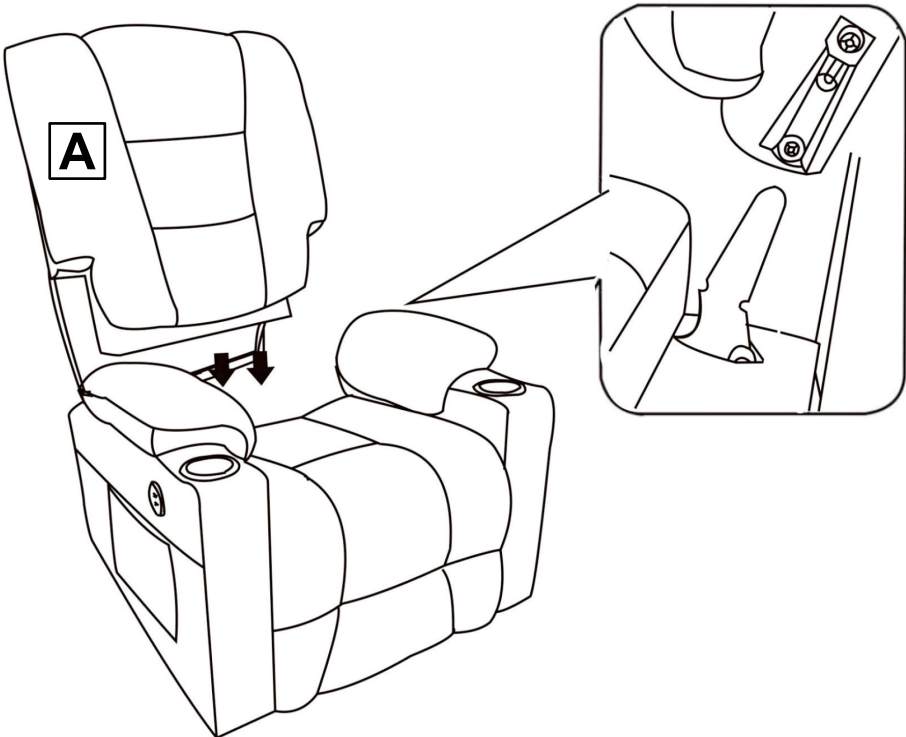

Lift Recliner Assembly Instructions

STEP 3



Schematic diagram of wiring connection for the chair.

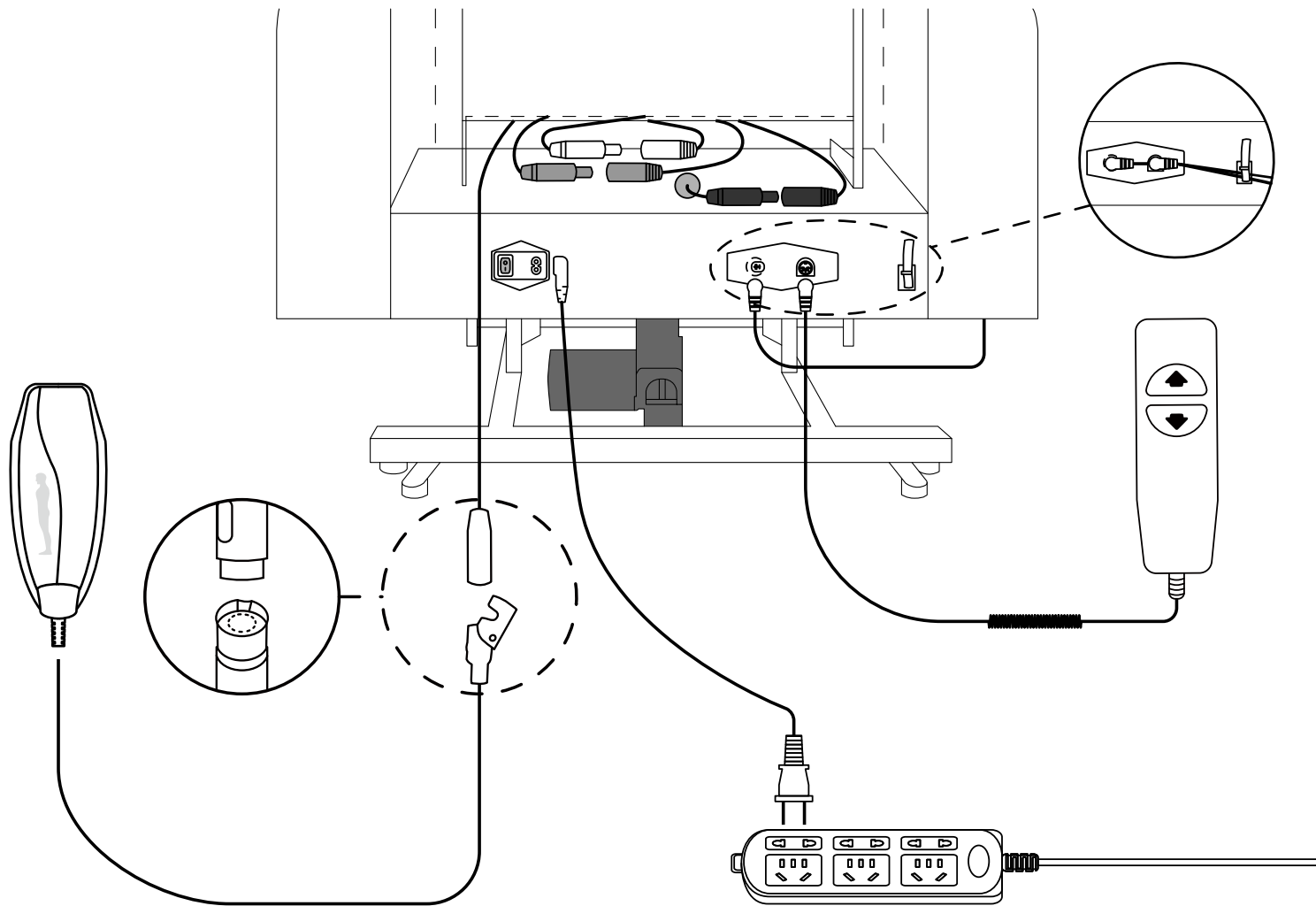

Lift Recliner Assembly Instructions


STEP 4

Assemble the anti-back-tipping part.

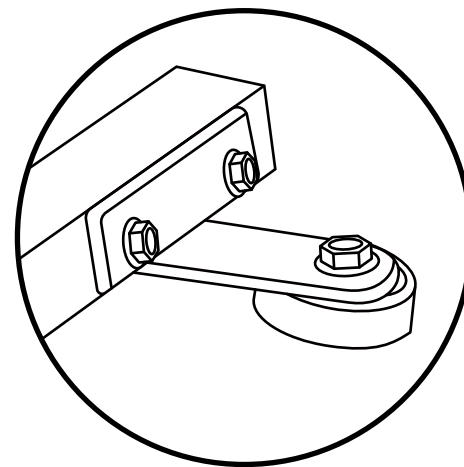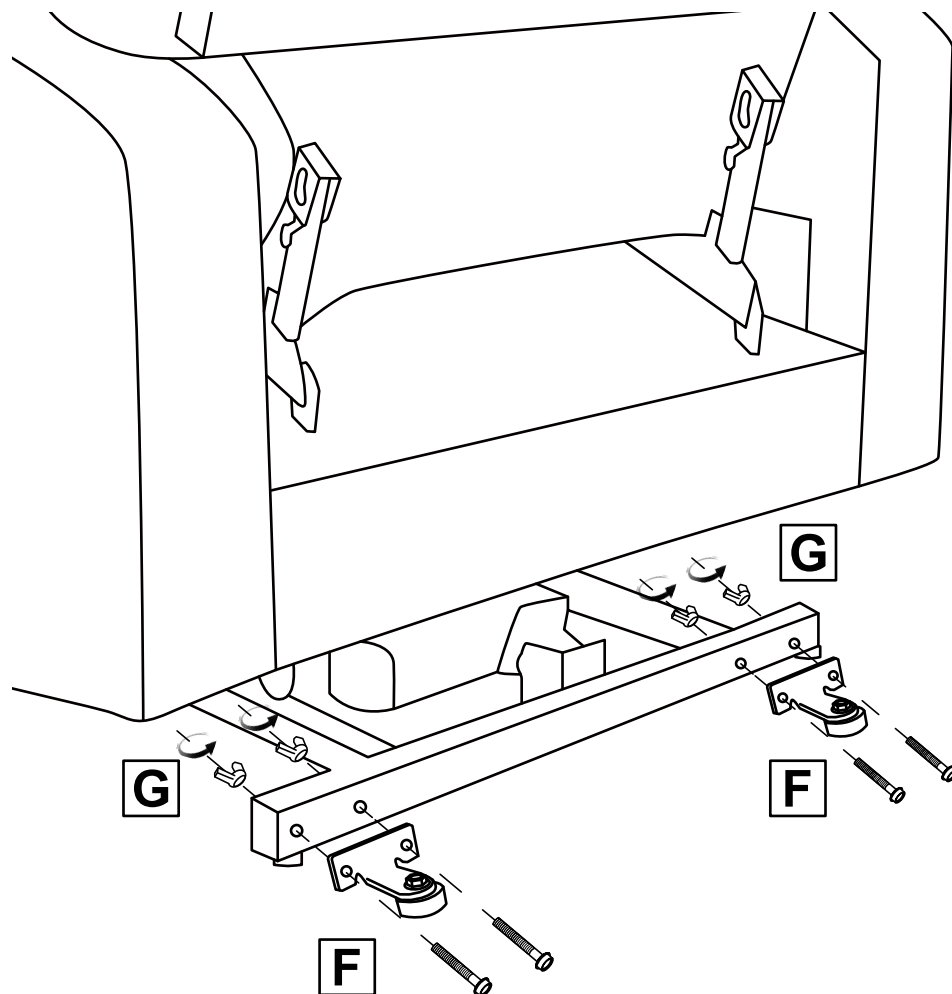

Lift Recliner Assembly Instructions

How to use the lift remote?

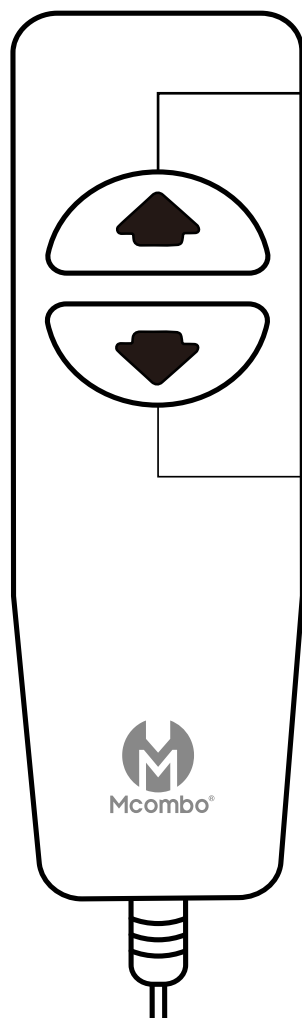

^ LIFT (UP)

Press and hold the UP button to raise the chair to the full-lift position.

Press and hold the UP button to return to the seated position from the reclined position.

▼ LIFT (DOWN)

Press and hold the DOWN button to lower the chair to the seated position.

Press and hold the DOWN button to raise the footrest and recline the chair back.

Lift Recliner Assembly Instructions

How to use the massage remote?

